



All capacitors are uF.
Copyright GTR LABS 2001 All rights reserved.

Automatic Exposure Control Interface/ High Speed Starter Interface Board	
AHB	
Rev 2	ID AHB.ckt
Date: 3/29/01	Page: 1 of 1

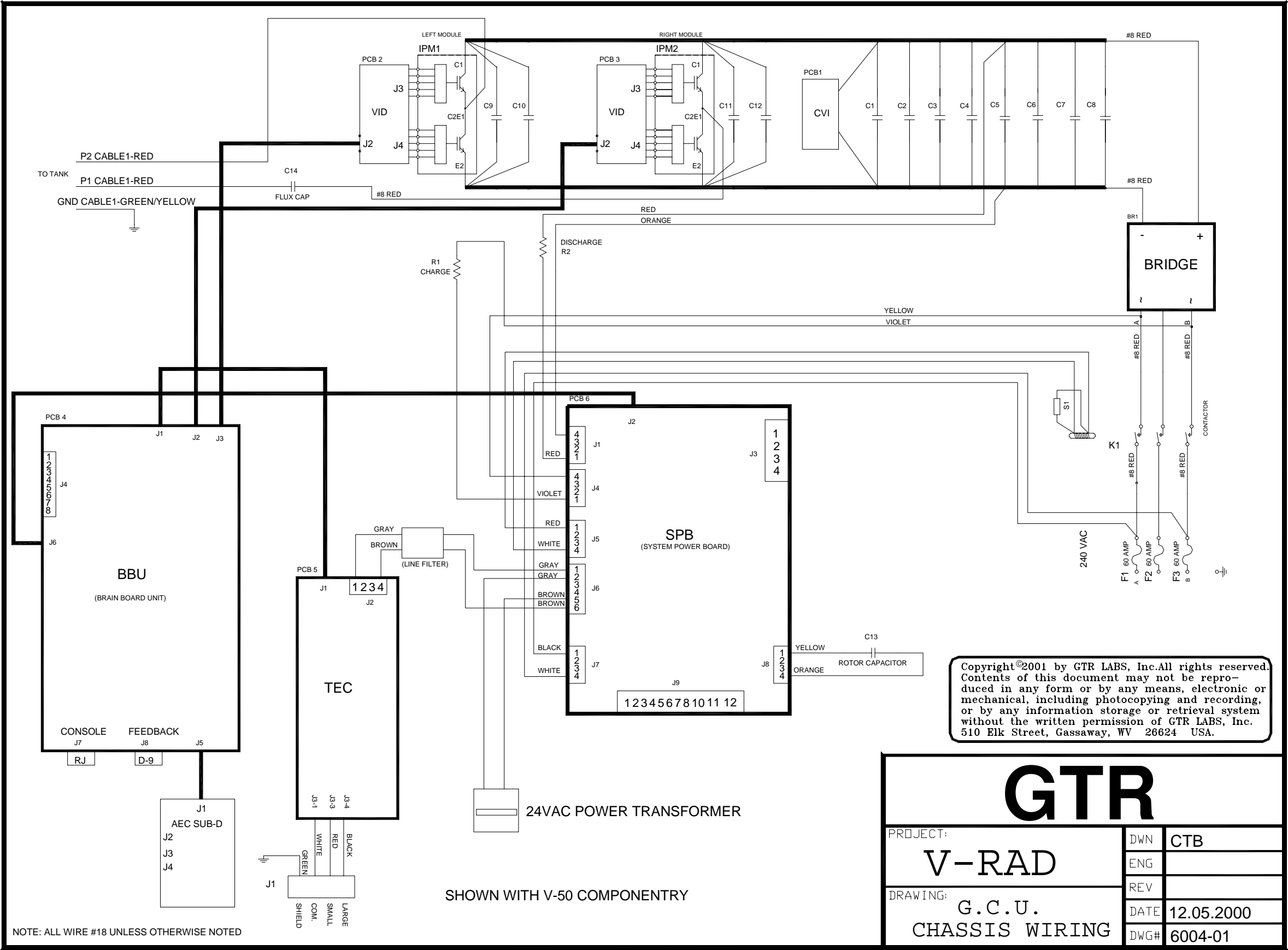

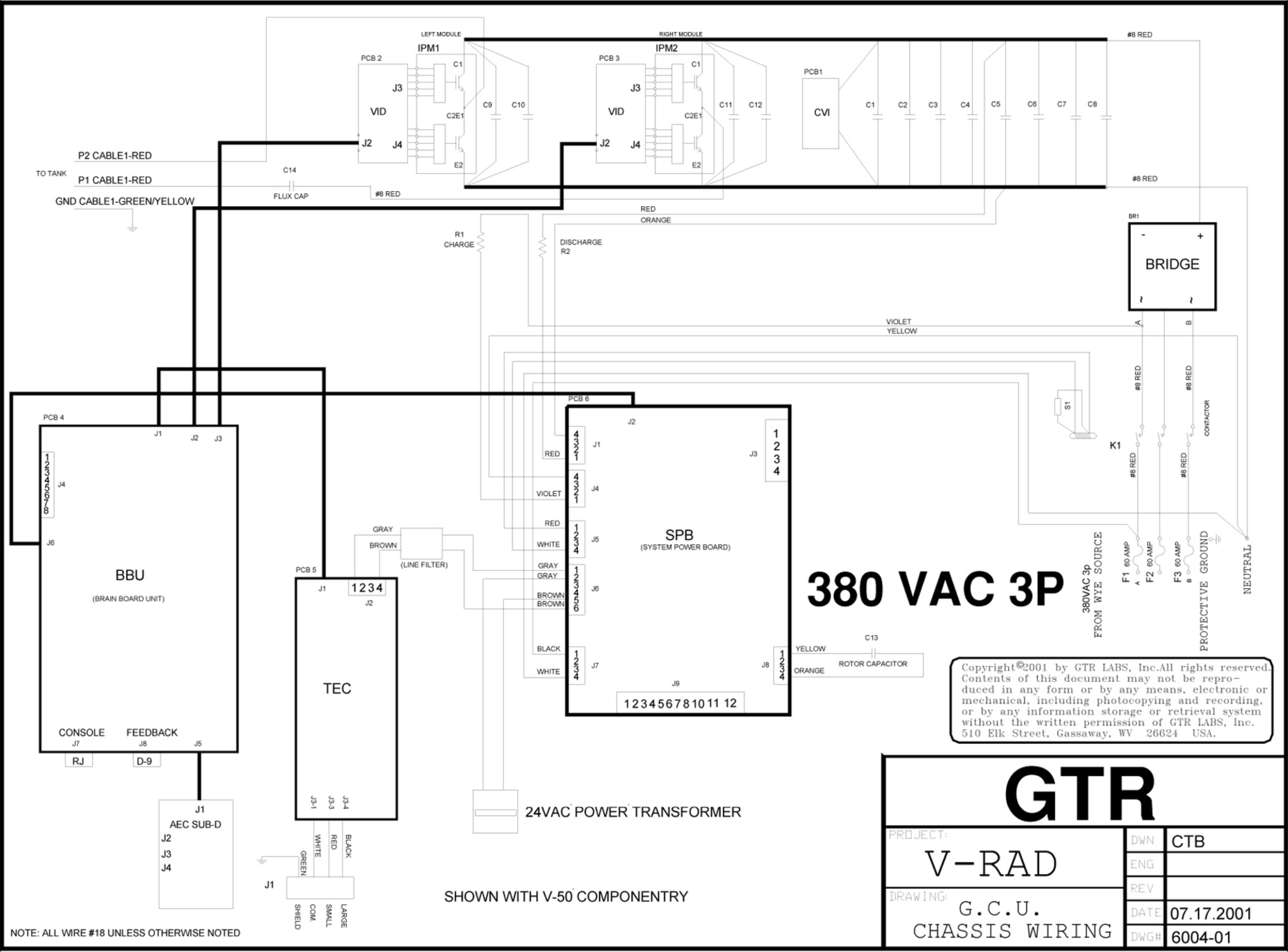

GTR		
PROJECT:	DWN	CTB
V-RAD	ENG	
DRAWING:	REV	
G.C.U.	DATE	07.17.2001
CHASSIS WIRING	DWG#	6004-01



GTR® LABS, llc

510 Elk Street

Gassaway, WV 26624

Voice: (304) 364-2211 FAX: (304) 364-2212

e-mail: gsa00214@mail.wvnet.edu

<http://gtrllc.com>

Directions to Manufacturing Facility

From the North (Pittsburg)

I-79 to exit 62

Travel Rt 4 S (toward Gassaway) for 3 miles

Cross bridge over Elk River, enter Gassaway

Travel through first stop light

Facility is located before the second stop light, a red brick building on the right, across from the GO-MART

Street parking is available on Rt 4 (Elk Street)

From the West (Charleston)

I-79 to exit 51

Travel Rt. 4 N (toward Gassaway) for 8 miles

Cross bridge over Elk River, enter Gassaway

Follow Rt. 4 into town to first traffic light

Facility is located after the first traffic light, a red brick building on the left, across from the GO-MART

Street parking is available on Rt 4 (Elk Street)

From the South (Beckley)

Rt. 19 N to I-79 N

Travel I-79 N for 5 miles to exit 62

Travel Rt 4 S (toward Gassaway) for 3 miles

Cross bridge over Elk River, enter Gassaway

Travel through first stop light

Facility is located before the second stop light, a red brick building on the right, across from the GO-MART

Street Parking is available on Rt 4 (Elk Street)

Hotel Accommodations

Take I-79 to exit 67 (Flatwoods)



GTR® LABS, llc


510 Elk Street


Gassaway, WV 26624

Voice: (304) 364-2211 FAX: (304) 364-2212

e-mail: gsa00214@mail.wvnet.edu

<http://gtrllc.com>

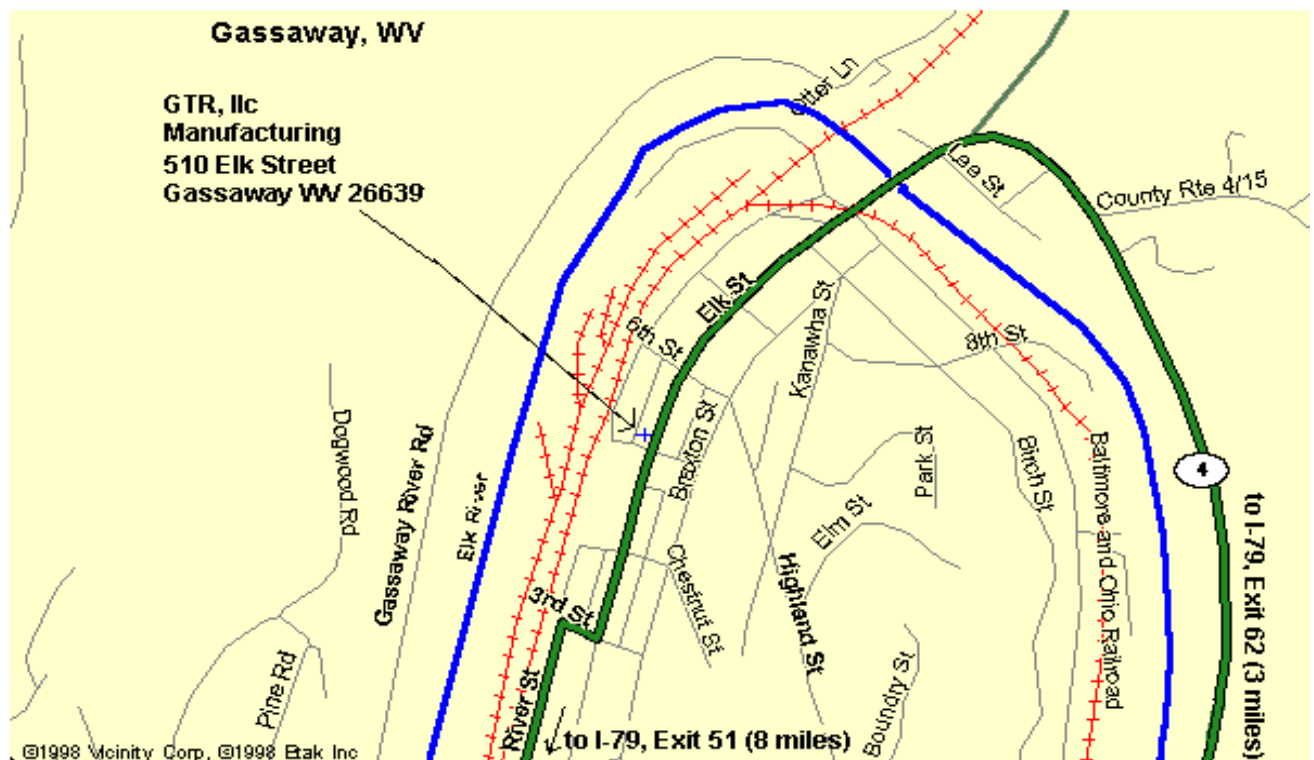

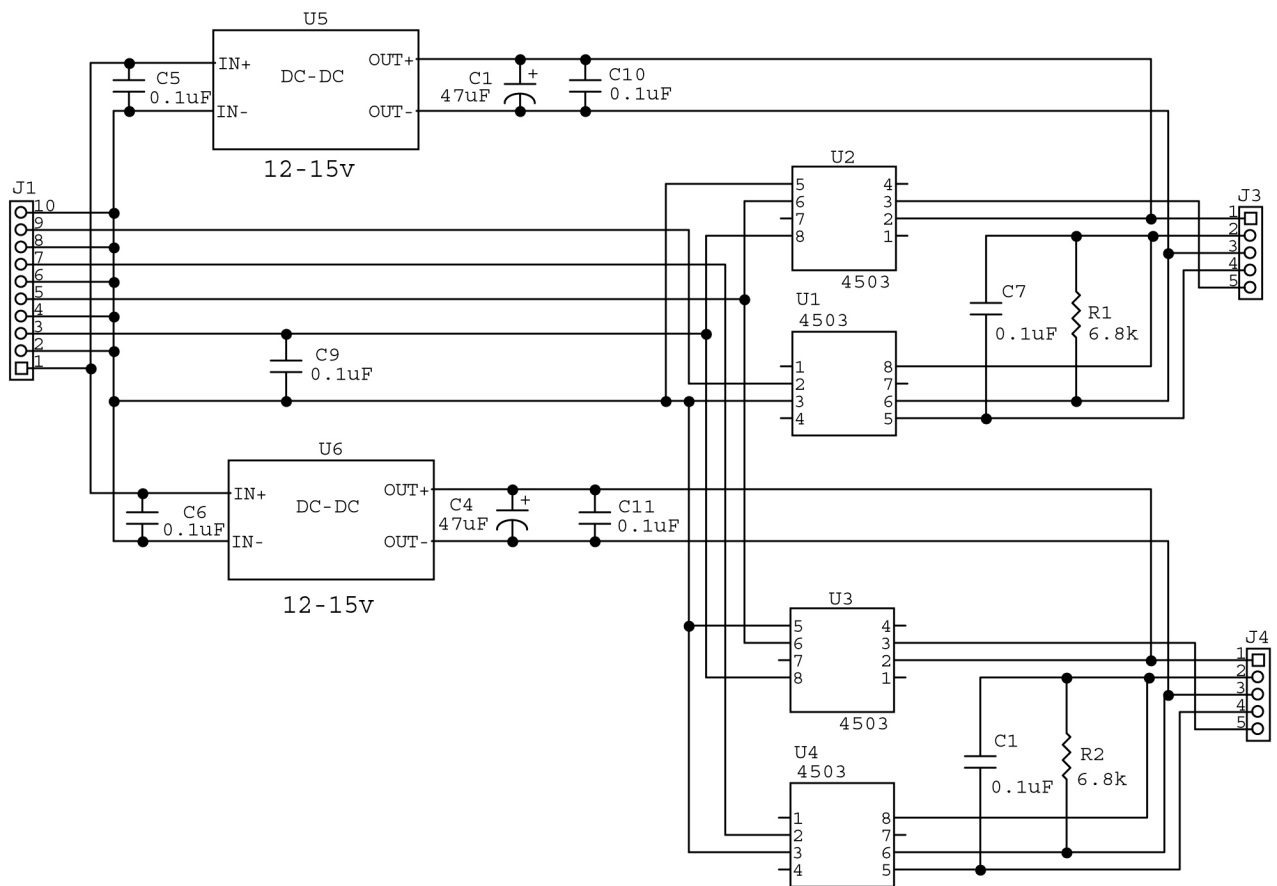

COPYRIGHT GTR LABS 2001 ALL
RIGHTS RESERVED.

DVA IPM DRIVER	
4034	
Rev 1	ID DVA.CKT
Date: 4/26/01	Page: 1 of 1

Warranty

GTR® LABS, Inc. warrants that this product will be free from defects in materials and workmanship for a period of five (5) years from the date of installation at the first end user's site; If any such product proves defective during this warranty period, **GTR®**, at it's option, either will repair the defective product without charge for parts or shop labor, or will provide a replacement in exchange for the defective product.

This warranty shall apply to all equipment and systems sold by **GTR®** with the exception of the following:

X-Ray Tubes

Image Intensifiers

TV Camera Tubes

The above items are subject to the manufacturer's warranty in effect as of the date sold. These warranties vary and are pro-rated. The manufacturer will be the final authority for any warranty claim.

In order to obtain service under this warranty, Customer must notify **GTR®** of the defect before the expiration of the warranty period and make suitable arrangements for the performance of service. Customer shall be responsible for packaging and shipping the defective product to the service center designated by **GTR®** with shipping charges prepaid. **GTR®** shall pay for the return of the product to Customer if the shipment is to a location within the country in which the **GTR®** designated service center is located. Customer shall be responsible for paying all shipping charges, duties, taxes, and any other charges for products returned to any other locations.

This warranty shall not apply to any defect, failure, or damage caused by improper or inadequate maintenance and care. **GTR®** shall not be obligated to furnish service under this warranty 1) to repair damage resulting from attempts by personnel other than **GTR®** representatives to install, repair, or service this product; 2) to repair damage resulting from improper use or connection to incompatible equipment or power source; or 3) to service a product that has been modified or integrated with other products when the effect of such modification or integration increases the time or difficulty of servicing the product.

Further, **GTR® LABS**, Inc. warrants that the product is in compliance with U.S.A. DHHS regulations which may be in force and effect at the time of shipment of the product.


12/01/2002

The following is not covered by the warranty:

General maintenance and simple adjustments mentioned in the manuals delivered with the material.

THIS WARRANTY IS GIVEN BY GTR® WITH RESPECT TO THIS PRODUCT IN LIEU OF ANY OTHER WARRANTIES, EXPRESS OR IMPLIED. GTR® AND IT'S VENDORS DISCLAIM ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. GTR®'S RESPONSIBILITY TO REPAIR OR REPLACE DEFECTIVE PRODUCTS IS THE SOLE REMEDY PROVIDED TO THE CUSTOMER FOR BREACH OF THIS WARRANTY. GTR® AND IT'S VENDORS WILL NOT BE LIABLE FOR ANY INDIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES IRRESPECTIVE OF WHETHER GTR® OR THE VENDOR HAS ADVANCE NOTICE OF THE POSSIBILITY OF SUCH DAMAGES.

There are no warranties which extend beyond the description mentioned in this document.

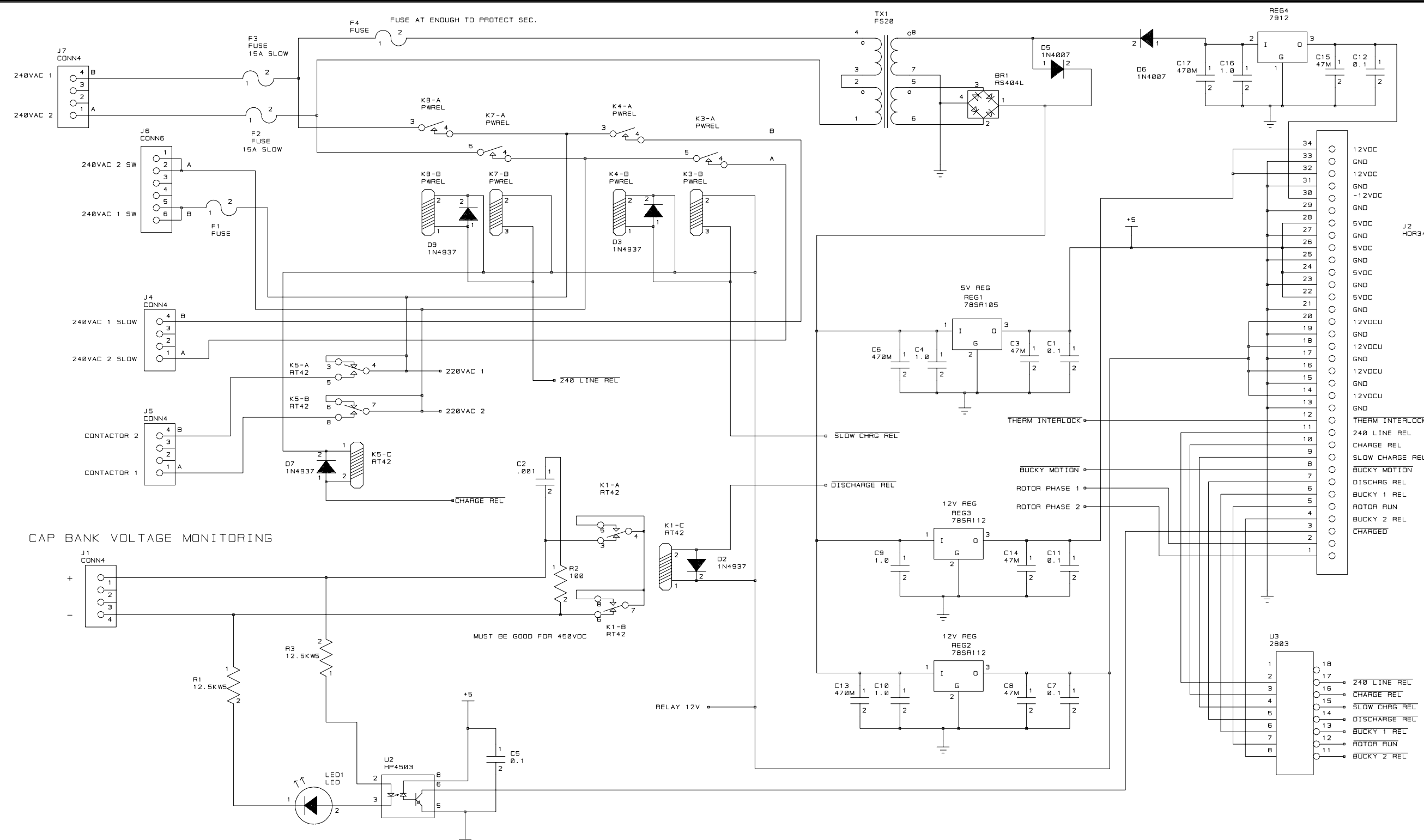

AN:

SYSTEM POWER BOARD (SPB)	DISSEMINATION: INTERNAL ONLY
	PAGE: 1 OF 1
	FILE: 1075
	REV: 1 10/5/00, 2 1/5/01
	ENG:
GTR LABS, LLC	DATE: 4/19/99

COPYRIGHT 1997, 1998, 1999, 2000, 2001 BY GTR LABS, LLC.
ALL RIGHTS RESERVED.
CONTENTS OF THIS DOCUMENT MAY NOT BE REPRODUCED
IN ANY FORM OR BY ANY MEANS, ELECTRONIC OR
MECHANICAL, INCLUDING PHOTOCOPYING AND RECORDING,
OR BY ANY INFORMATION STORAGE OR RETRIEVAL SYSTEM
WITHOUT THE WRITTEN PERMISSION OF GTR LABS, LLC, PO BOX
779, BROOKFIELD CT, 06804.

PROPRIETARY INFORMATION
DO NOT DISSEMINATE

PROPRIETARY INFORMATION
DO NOT DISSEMINATE

AN:

SYSTEM POWER
BOARD (SPB)

GTR LABS, LLC

DISSEMINATION: INTERNAL ONLY
PAGE: 1 OF 1
FILE: 1075
REV: 1 10/5/00, 2 1/5/01
ENG:
DATE: 4/19/99

COPYRIGHT 1997, 1998, 1999, 2000, 2001 BY GTR LABS, LLC.
ALL RIGHTS RESERVED.
CONTENTS OF THIS DOCUMENT MAY NOT BE REPRODUCED
IN ANY FORM OR BY ANY MEANS, ELECTRONIC OR
MECHANICAL, INCLUDING PHOTOCOPYING AND RECORDING,
OR BY ANY INFORMATION STORAGE OR RETRIEVAL SYSTEM
WITHOUT THE WRITTEN PERMISSION OF GTR LABS, LLC, PO BOX
779, BROOKFIELD CT, 06804.

USER RELAY

