

SERIES 9600™

**Imaging Workstation
Illustrated Parts Manual**



OEC
OEC MEDICAL
SYSTEMS, INC

(PN 00-876789-02)

This manual contains information on the SERIES**9600** Imaging Workstation. It should serve as an aid to field service engineers and technicians in locating parts and identifying part numbers. It is not a complete parts list and should not be used as a substitute for a bill of materials. Furthermore, product changes and improvements may occur at any time and these changes may not be reflected in this manual.

Removing or installing parts should not be attempted by anyone who is not specifically trained by OEC Medical Systems, Inc. to work on the SERIES**9600** Imaging Workstation System.

This manual may not be reproduced in whole or in part without written permission of OEC Medical Systems Incorporated.

This manual remains the property of OEC Medical Systems, Inc.

© 1995

OEC Medical Systems, Inc.
384 Wright Brothers Drive
Salt Lake City, Utah 84116
U.S.A.

The SERIES**9600** Mobile Digital Imaging System is manufactured under the following US patents: 4.768.216; 4.797.907; and 4.209.706. Other patents pending.

Lenzar is a trademark of Lenzar Corporation.

Sony is a registered trademark of Sony Corporation of America.

Trademarks of other products mentioned in this manual are held by the companies producing them.

Revision History

<u>Rev</u>	<u>Dash</u>	<u>Date</u>	<u>Change Description</u>
A	-01	March, 1994	Release
B	-02	January, 1995	Document batch release changes to SERIES 9600 Mobile Imaging System.

Table of Contents

Section 1: Older SERIES9600 Imaging Workstation (S/N 69-001 thru 69-999)

Fig. 1-1	ESP (Expanded Surgical Package) Exploded Front View	11
Fig. 1-2	SP (Surgical Platform) Model Exploded Front View	13
Fig. 1-3	Power Control Panel, Transformer, PCB Assy Surge Supr Exploded View	15
Fig. 1-4	Rear Panel, Fan, Power Cord Assy, Interconnect Cable Assy Exploded View	17
Fig. 1-5	Monitor Rear View Exploded View.....	19
Fig. 1-6	Brake, Caster Exploded View	21
Fig. 1-7	Electronic Rack, Generic Exploded View.....	23
Fig. 1-8	Electronic Rack, EPS with 4 FPS Storage Exploded View.....	25
Fig. 1-9	Electronic Rack, Vascular with 30 FPS Storage Exploded View.....	27
Fig. 1-10	Keyboard, SP (Surgical Platform) Model Exploded View	29
Fig. 1-11	Keyboard, ESP/Cholangiography Model Exploded View.....	31
Fig. 1-12	Keyboard, ESP/Acquire Model Exploded View.....	33
Fig. 1-13	Keyboard, ESP Vascular Model Exploded View	35
Fig. 1-14A	Interface Cable Diagram, Imaging Workstation (Sheet 1 of 5)	37
Fig. 1-14B	Interface Cable Diagram, Imaging Workstation (Sheet 2 of 5)	39
Fig. 1-14C	Interface Cable Diagram, Imaging Workstation (Sheet 3 of 5)	41

Fig. 1-14D	Interface Cable Diagram, Imaging Workstation (Sheet 4 of 5)	43
Fig. 1-14E	Interface Cable Diagram, Imaging Workstation (Sheet 5 of 5)	46
	Attaching Hardware	47

Section 2: Newer SERIES9600 Imaging Workstation (S/N 69-1000 & up)

Fig. 2-1	ESP (Expanded Surgical Package) Exploded Front View	53
Fig. 2-2	SP (Surgical Platform) Model Exploded Front View	55
Fig. 2-3	Power Control Panel, Transformer, PCB Assy Surge Supr Exploded View	57
Fig. 2-4	Rear Panel, Fan, Power Cord Assy, Interconnect Cable Assy Exploded View	59
Fig. 2-5	Monitor Rear View Exploded View.....	61
Fig. 2-6	Brake, Caster Exploded View	63
Fig. 2-7	Electronic Rack, Generic Exploded View	65
Fig. 2-8	Electronic Rack, EPS with 4 FPS Storage Exploded View.....	67
Fig. 2-9	Electronic Rack, Vascular with 30 FPS Storage Exploded View	69
Fig. 2-10	Keyboard, SP (Surgical Platform) Model Exploded View	71
Fig. 2-11	Keyboard, ESP/Cholangiography Model Exploded View	73
Fig. 2-12	Keyboard, ESP/Acquire Model Exploded View.....	75
Fig. 2-13	Keyboard, ESP Vascular Model Exploded View	77
Fig. 2-14A	Interface Cable Diagram, Imaging Workstation (Sheet 1 of 5)	79
Fig. 2-14B	Interface Cable Diagram, Imaging Workstation (Sheet 2 of 5)	81

Fig. 2-14C	Interface Cable Diagram, Imaging Workstation (Sheet 3 of 5)	83
Fig. 2-14D	Interface Cable Diagram, Imaging Workstation (Sheet 4 of 5)	85
Fig. 2-14E	Interface Cable Diagram, Imaging Workstation (Sheet 5 of 5)	88
	Attaching Hardware	89

Section 3: SERIES9600** Imaging Workstation Illustrated Parts Manual Updates**

Update July 1995

Fig. 1 July 1995		95
------------------	-------	----

Introduction

Who Should Use This Manual

This manual is intended for the properly trained engineer or technician who designs, installs, maintains or calibrates the OEC Medical Systems, Inc. SERIES9600 Imaging Workstation System.

How This Manual is Organized

This manual is organized into sections. Section 1 contains information on replaceable parts for systems with serial numbers 69-001 thru 69-999. Section 2 contains information on replaceable parts for systems with serial numbers 69-1000 and up.

Both sections of this manual provide exploded view illustrations, lists, and descriptions of replaceable parts necessary for repair of the Imaging Workstation System.


A list of attaching hardware parts have been included for part number reference in the event reordering is necessary located at the back of the manual.

Contacting Technical Support

For assistance with part number changes, contact Technical Support at US 1-800-874-7378, Europe 41 1 811 07 70, others 1-801-536-4688.

Note: *The item numbers on the illustrations do not match the item numbers on the assembly drawings or the item numbers on the Bills of Materials.*

Section 1

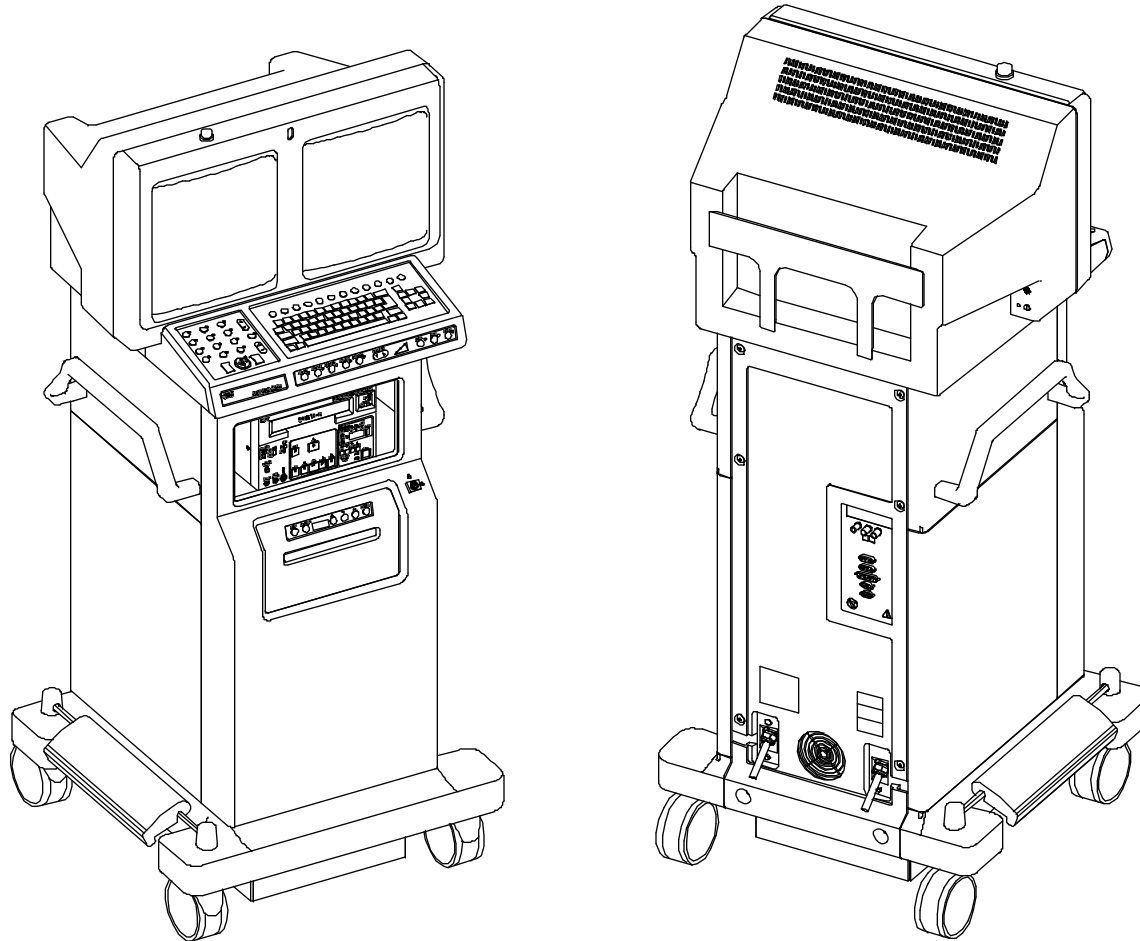

SERIES9600 Imaging Workstation

FIG. 1-1 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-875986-01	COVER, SIDE, LOWER, LEFT	1
2	00-876466-01	HANDLE, MONITOR CART (NOT EXPLODED)	2
3	00-875981-01	COVER, SIDE, UPPER LEFT (NOT EXPLODED)	1
4	00-876746-01	KIT, VTR, SONY 9500 MD4, 60HZ W/CBL	1
5	00-875986-02	COVER, SIDE, LOWER, RIGHT	1
6	00-876285-01	BRACKET, MOUNTING, LENZAR CAMERA	1
7	00-877592-01	BRACKET, MOUNTING, LENZAR CAMERA, LOW (NOT SHOWN)	1
8	00-876544-01	CABLE ASSEMBLY, KEY SWITCH TO P15	1
9	00-875987-02	COVER, FRONT PANEL, LENZAR	1
10	00-900640-01	PANEL ASSEMBLY, CONTROL, LENZAR, ENGLISH, 9600	1
10	00-900640-02	PANEL ASSEMBLY, CONTROL, LENZAR, GERMAN, 9600	1
10	00-900640-03	PANEL ASSEMBLY, CONTROL, LENZAR, FRENCH, 9600	1
10	00-900640-04	PANEL ASSEMBLY, CONTROL, LENZAR, SPANISH, 9600	1
10	00-900640-05	PANEL ASSEMBLY, CONTROL, LENZAR, ITALIAN, 9600	1
11	00-874026-06	ASSEMBLY, IR REMOTE CONTROL VAS, 4 GEN	1
12	00-877611-01	ASSEMBLY, IR REMOTE CONTROL, ENGLISH	1
12	00-877611-03	ASSEMBLY, IR REMOTE CONTROL, GERMAN	1
12	00-877611-04	ASSEMBLY, IR REMOTE CONTROL, FRENCH	1
12	00-877611-05	ASSEMBLY, IR REMOTE CONTROL, SPANISH	1
12	00-877611-07	ASSEMBLY, IR REMOTE CONTROL, ITALIAN	1

FIG. 1-1 PARTS LIST (CONT.)

ITEM	PART NUMBER	DESCRIPTION	QTY
13	00-878031-01	ASSEMBLY, IR REMOTE CONTROL, 9600, ENGLISH	1
13	00-878031-02	ASSEMBLY, IR REMOTE CONTROL, 9600, GERMAN	1
13	00-878031-03	ASSEMBLY, IR REMOTE CONTROL, 9600, FRENCH	1
13	00-878031-04	ASSEMBLY, IR REMOTE CONTROL, 9600, SPANISH	1
13	00-878031-05	ASSEMBLY, IR REMOTE CONTROL, 9600, ITALIAN	1
14	00-876161-01	KIT, LENZAR, 60HZ, ENGLISH, 9600	1
15	00-900666-01	CAMERA, LENZAR, 60HZ (NOT EXPLODED)	1
16	00-875981-02	COVER, SIDE, UPPER, RIGHT (NOT EXPLODED/SHOWN)	1
17	00-876747-01	KIT, VTR, SONY 9510 MDP4, 50HZ W/CBL (NOT SHOWN)	1
18	00-900585-01	RECORDER, SONY VIDEO TAPE 1/2" 60HZ (NOT SHOWN)	1
19	00-900585-02	RECORDER, SONY VIDEO TAPE 1/2" 50HZ (NOT SHOWN)	1
20	00-900267-01	TAPE, VIDEO, 1/2" (NOT SHOWN)	AR
21	00-876748-01	ASSEMBLY, OPTION, THERMAL IMAGER (NOT SHOWN)	1
22	00-900394-01	PRINTER, THERMAL IMAGING (NOT SHOWN)	1
23	00-876178-01	ASSEMBLY, CRATE, WORKSTATION, 9600/2600 (NOT SHOWN)	1

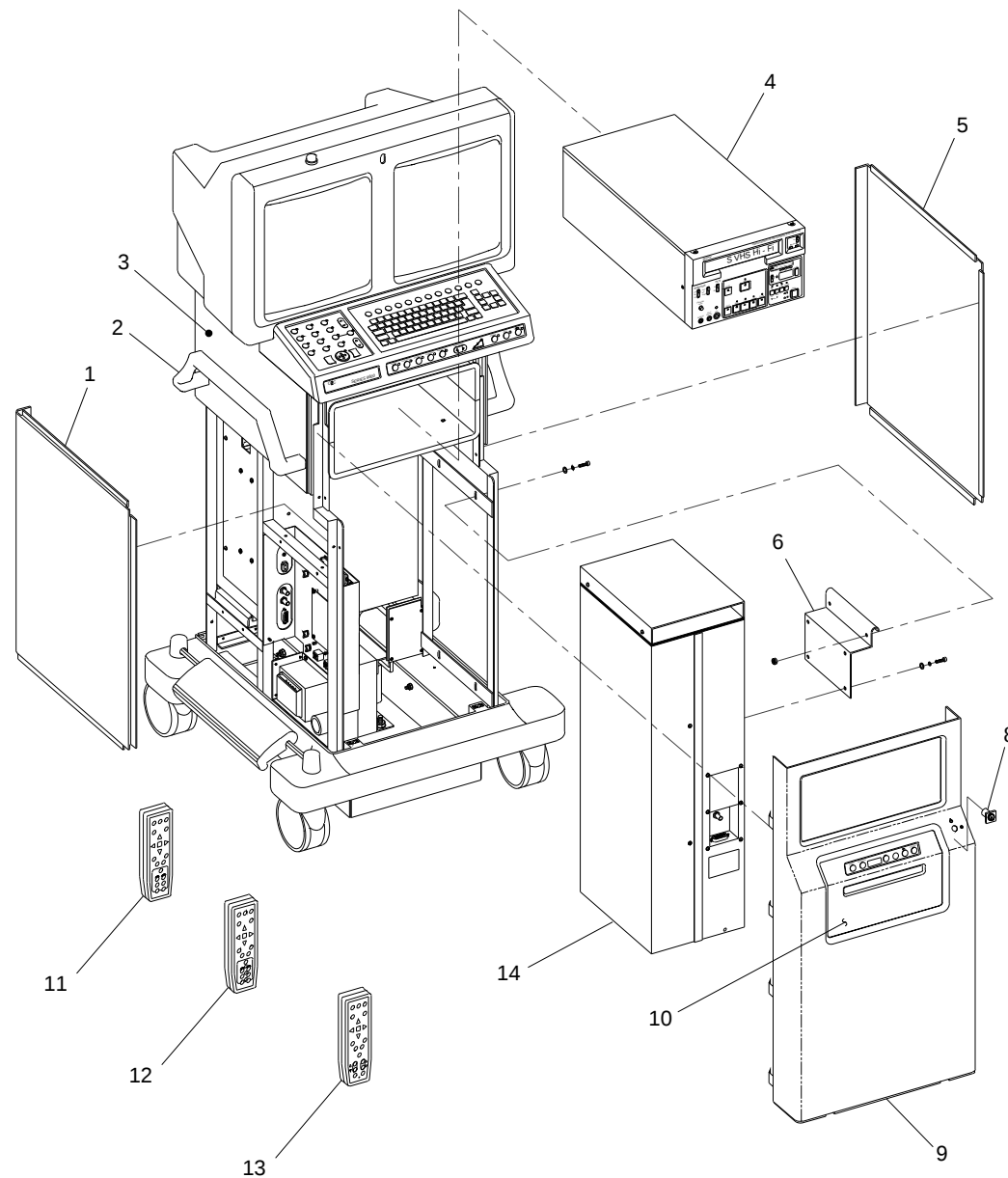


Fig. 1-1 ESP (Expanded Surgical Package) Exploded Front View



FIG. 1-2 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-875986-01	COVER, SIDE, LOWER, LEFT	1
2	00-876466-01	HANDLE, MONITOR CART (NOT EXPLODED)	2
3	00-875981-01	COVER, SIDE, UPPER, LEFT (NOT EXPLODED)	1
4	00-875986-02	COVER, SIDE, LOWER, RIGHT	1
5	00-876544-01	CABLE ASSEMBLY, KEY SWITCH TO P15	1
6	00-875987-01	COVER, FRONT PANEL	1
7	00-875992-01	COVER, SHELF, LOWER (NOT EXPLODED)	1
8	00-875991-01	COVER, SHELF, UPPER (NOT EXPLODED)	1
9	00-875981-02	COVER, SIDE, UPPER, RIGHT (NOT EXPLODED/SHOWN)	1

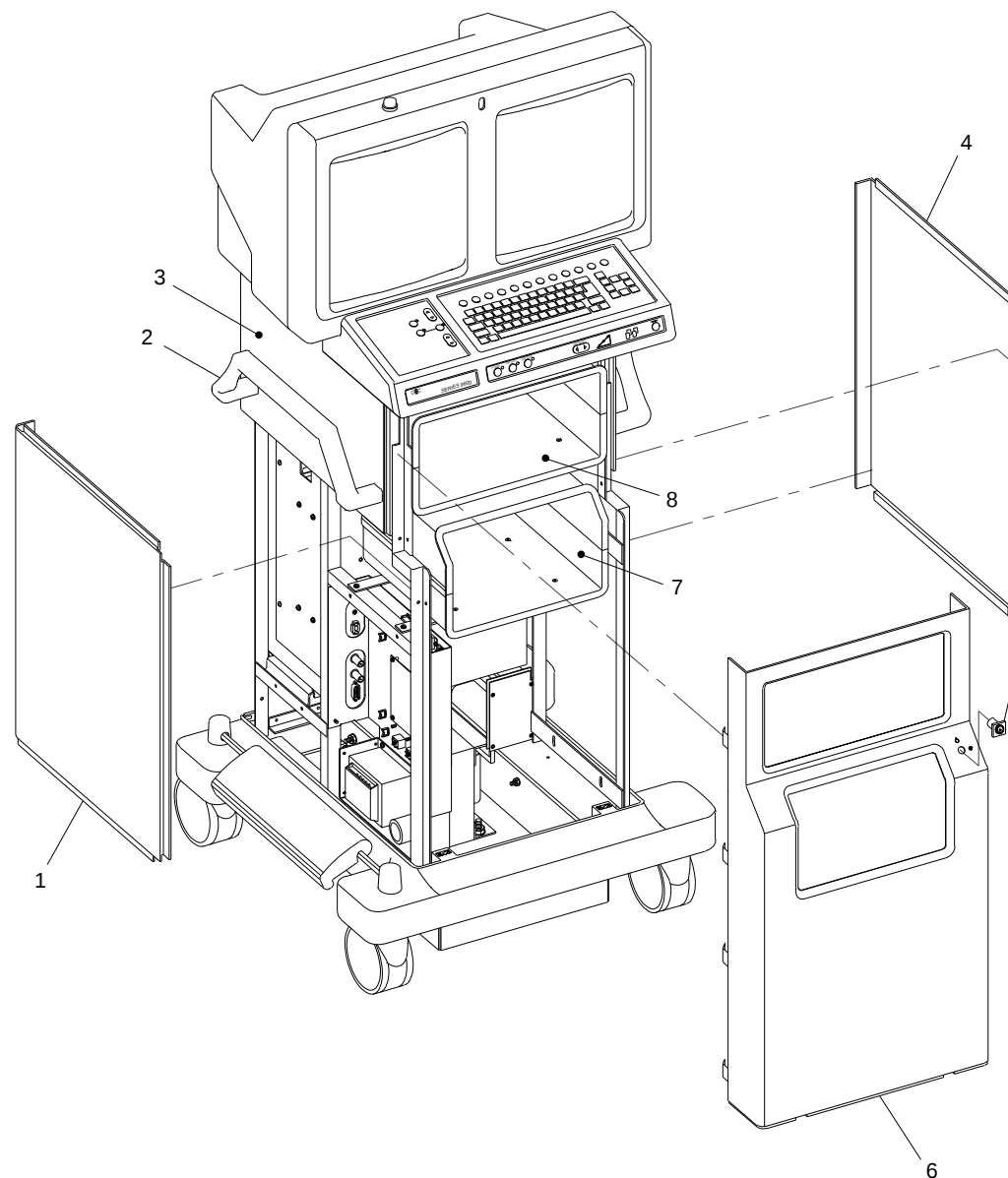


Fig. 1-2 SP (Surgical Platform) Model Exploded Front View



FIG. 1-3 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-876786-01	ASSEMBLY, PCB, SURGE SUPPRESSER	1
2	00-900712-01	TRANSFORMER, ISOLATION WORKSTATION	1
3	66-235026-00	RECTIFIER, SILICON BRIDGE, 100PRV, 30A	2
4	78-590390-00	POWER SUPPLY, MOD +24 VDC	1
5	55-700452-00	CIRCUIT, BREAKER, 10A DPST 125, 250 VAC, PANEL	1
6	00-877037-01	BRACKET, MOUNTING, CIRCUIT BREAKER	1
7	00-876549-01	ASSEMBLY, PCB, POWER CONTROL	1
8	56-996209-01	TERMINAL BLOCK, PANEL 4 POS, 26-12 AWG, 20A	1
9	56-996214-00	JUMPER, TERMINAL BLOCK	1
10	56-996215-00	TERMINAL BLOCK, END PLATE, 26-12 AWG	1

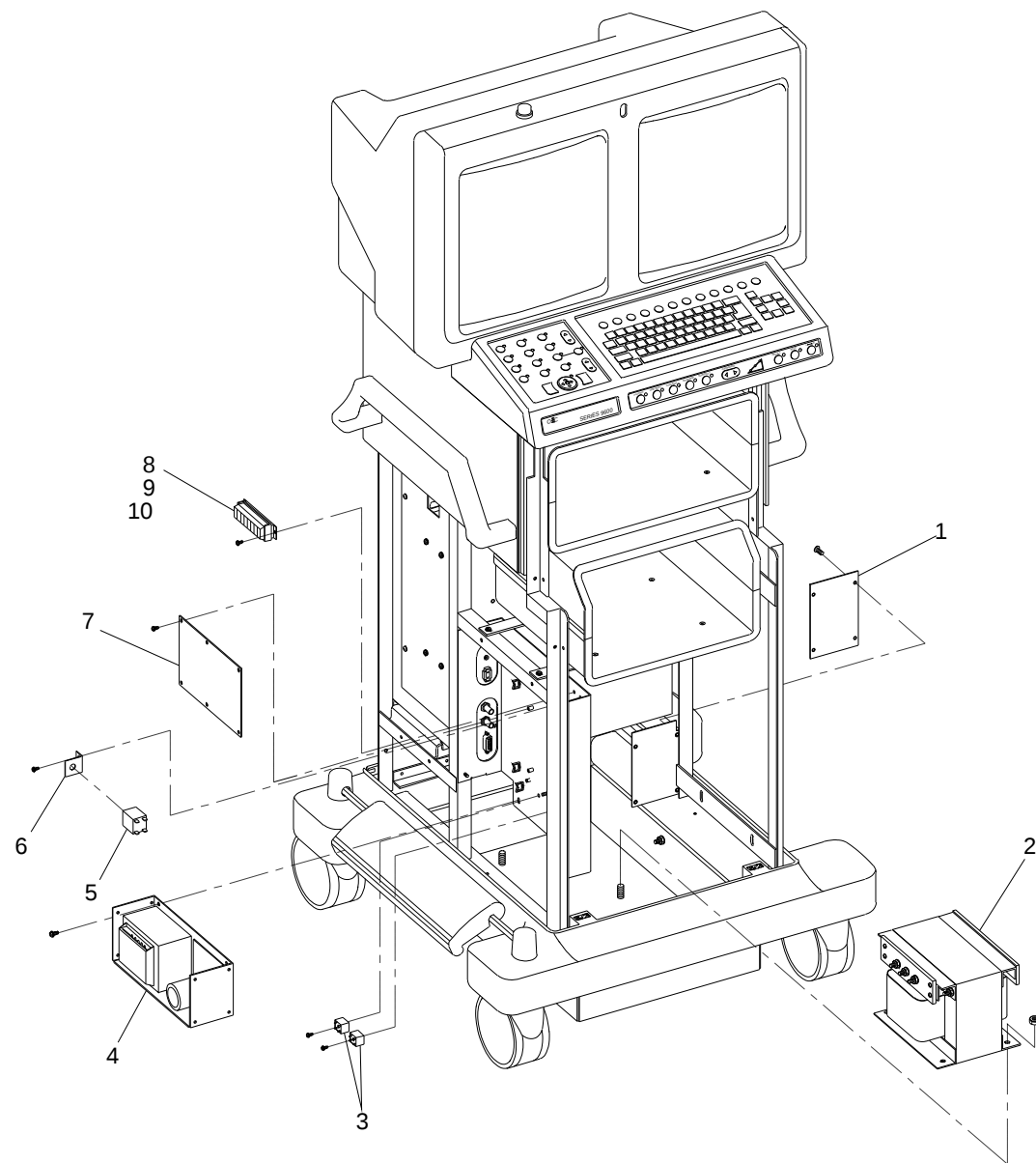


Fig. 1-3 Power Control Panel, Transformer, PCB Assy Surge Suppressor Exploded View

FIG. 1-4 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	55-700452-00	CIRCUIT BREAKER, 10A, DPST 125, 250 VAC, PANEL	2
2	00-876469-01	BRACKET, INTERCONNECT	1
3	00-876183-01	CABLE ASSEMBLY, INTERCONNECT, 20', 9600	1
4	00-876183-02	CABLE ASSEMBLY, INTERCONNECT, 30', 9600	1
5	20-839918-00	FAN, SXL, 115 VAC, 115 CFM, 3200 RPM	1
6	20-839926-00	GUARD, FINGER, FAN	1
7	00-875990-01	COVER, REAR PANEL	1
8	00-876468-01	BRACKET, POWER CORD	1
9	00-876180-01	ASSEMBLY, POWER CORD, 100-127V, 20 FT, 9600	2
10	00-876181-01	ASSEMBLY, POWER CORD, 220-250V, 20 FT, 9600	1
10	00-876181-02	ASSEMBLY, POWER CORD, 220-250V, 30 FT, 9600	1
11	00-876182-01	ASSEMBLY, POWER CORD, 220V, 60 HZ, 20', 9600	1
12	67-430001-00	IND, LAMP, NE, 110V, GREEN	1
13	56-299702-00	TERMINAL STRIP, PANEL, 3 POS, 380V	1
14	00-875981-02	COVER, SIDE, UPPER RIGHT (NOT EXPLODED)	1
15	27-590972-00	FILTER, AIR, FRAMELESS, 6 X 6 X .188	1

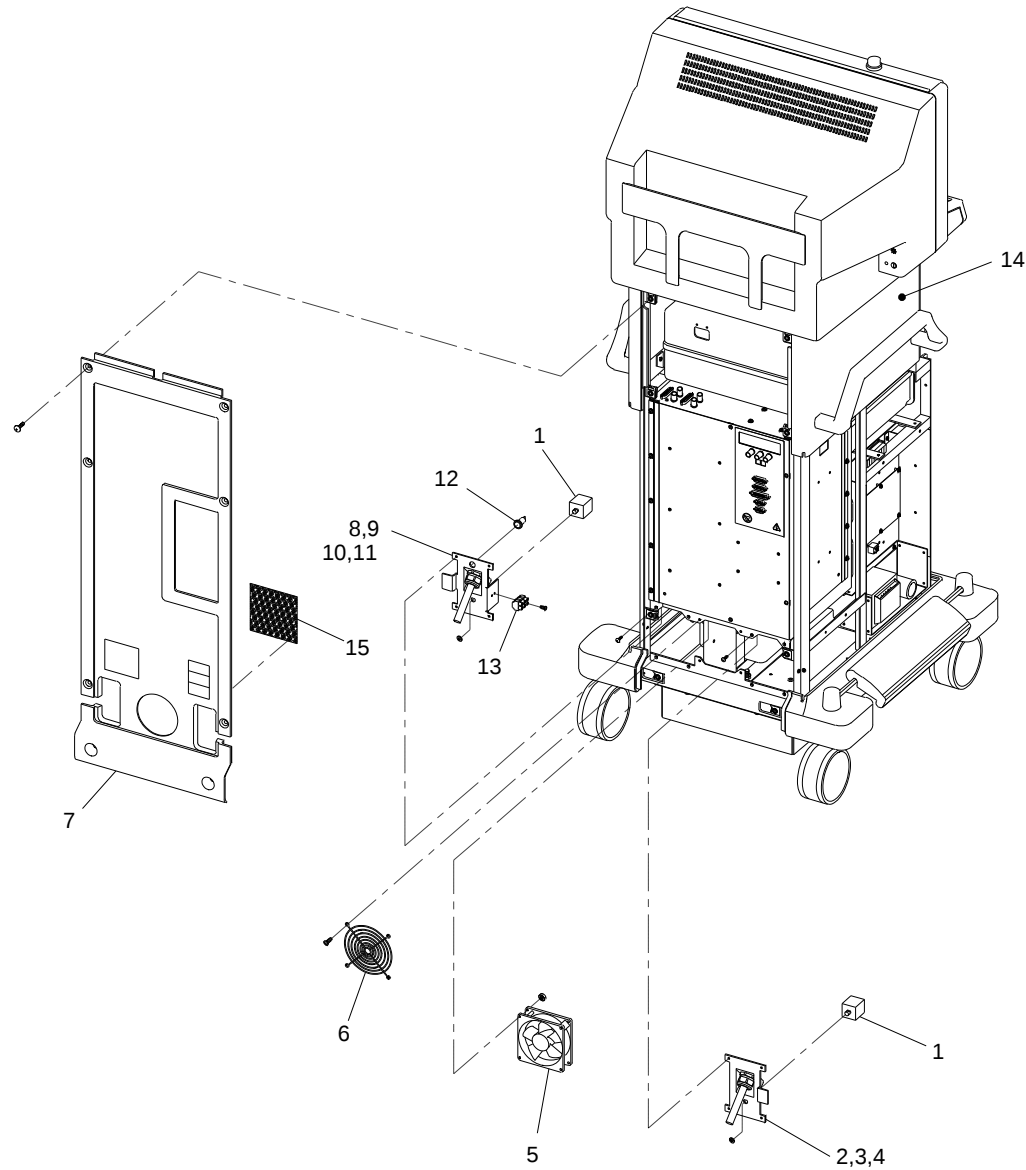


Fig. 1-4 Rear Panel, Fan, Power Cord Assy, Interconnect Cable Assy Exploded View



FIG. 1-5 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-900657-03	MONITOR, FAST SCAN, 16"	2
2	00-875983-01	HOUSING, MONITOR	1
3	00-877124-01	BRACE, BEZEL	1
4	00-876470-01	BRACKET, IR RECEIVER	1
5	00-874222-04	ASSEMBLY, PCB, IR RECEIVER 9600	1
6	00-900751-01	LAMP, X-RAY ON (SCD)	1
7	00-875982-01	BEZEL, MONITOR (NOT EXPLODED)	1
8	00-876541-01	CABLE ASSEMBLY, CONT/BRT POTS	1
9	00-876541-02	CABLE ASSEMBLY, CONT/BRT POTS	1
10	00-876471-02	BRACKET, B/C MOUNTING	1
11	00-877009-03	ASSEMBLY, OUTLET STRIP, INTL AC (120V AC 50/60 Hz, 240V AC 50/60Hz)	1
11	00-877009-04	ASSEMBLY, OUTLET STRIP, INTL AC (115V AC 50/60 Hz, 230V AC 50/60Hz)	1
12	00-900792-01	PCA, DEFLECTION, MONITOR (SCD)	2
13	00-876353-01	ASSEMBLY, PCB, HI SCAN BRGTS/CTRS CONT	1
14	00-874661-01	LENS, FRONT, COLORED (NOT EXPLODED/SHOWN)	1
15	00-874661-02	LENS, FRONT, CLEAR (NOT EXPLODED/SHOWN)	1

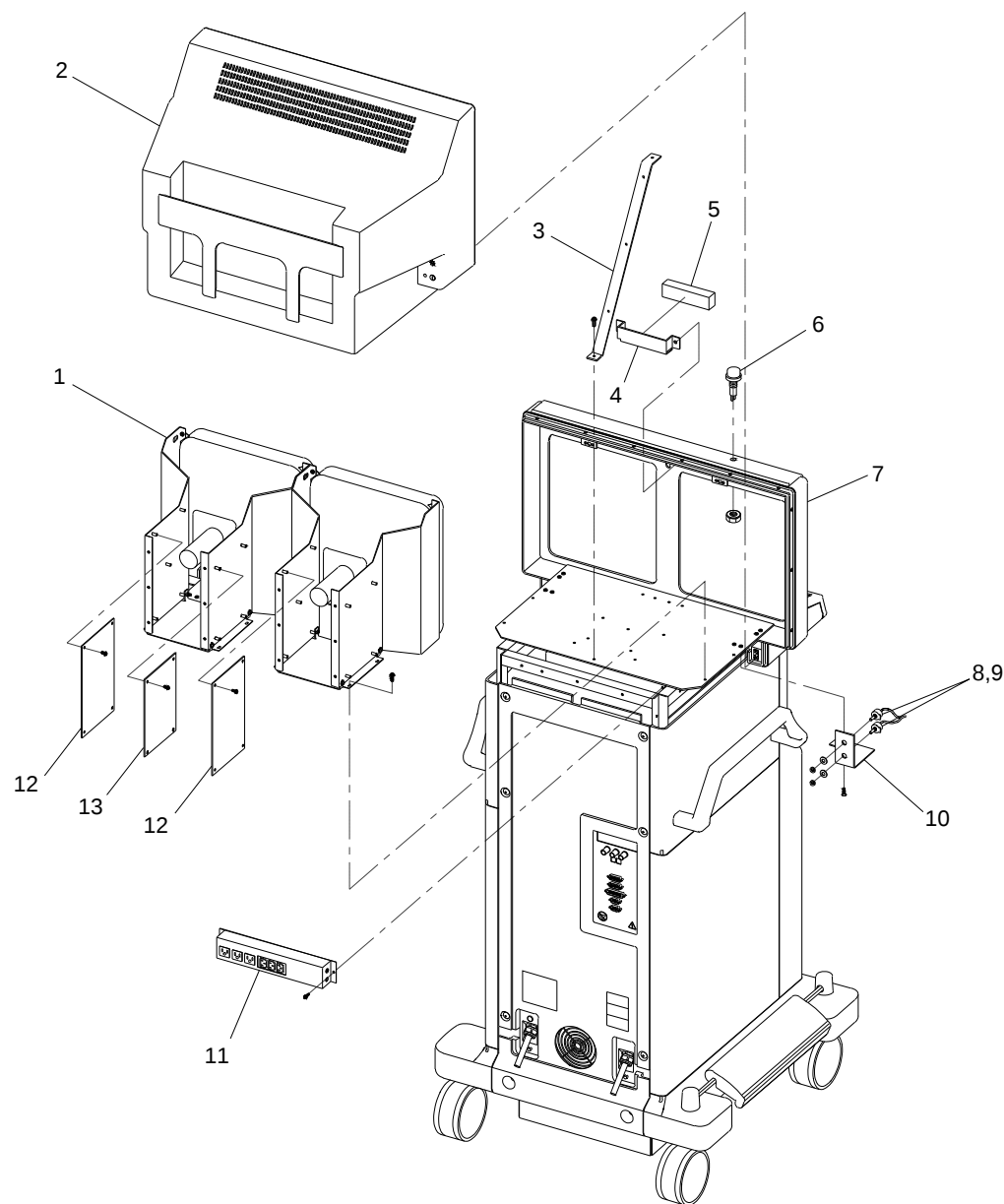


Fig. 1-5 Monitor Rear View Exploded View



FIG. 1-6 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-875984-01	COVER, BASE	1
2	00-900696-01	CASTER, LOCK, 5" (SCD)	2
3	00-876702-01	PEDAL, BRAKE	1
4	00-876624-01	BAR, BRAKE	2
5	00-900695-01	CASTER, SWIVEL, 5" (SCD)	2
6	21-709008-00	SPRING, COMPRESSION, .420 OD, .750 LG, .042 WD, SST	2

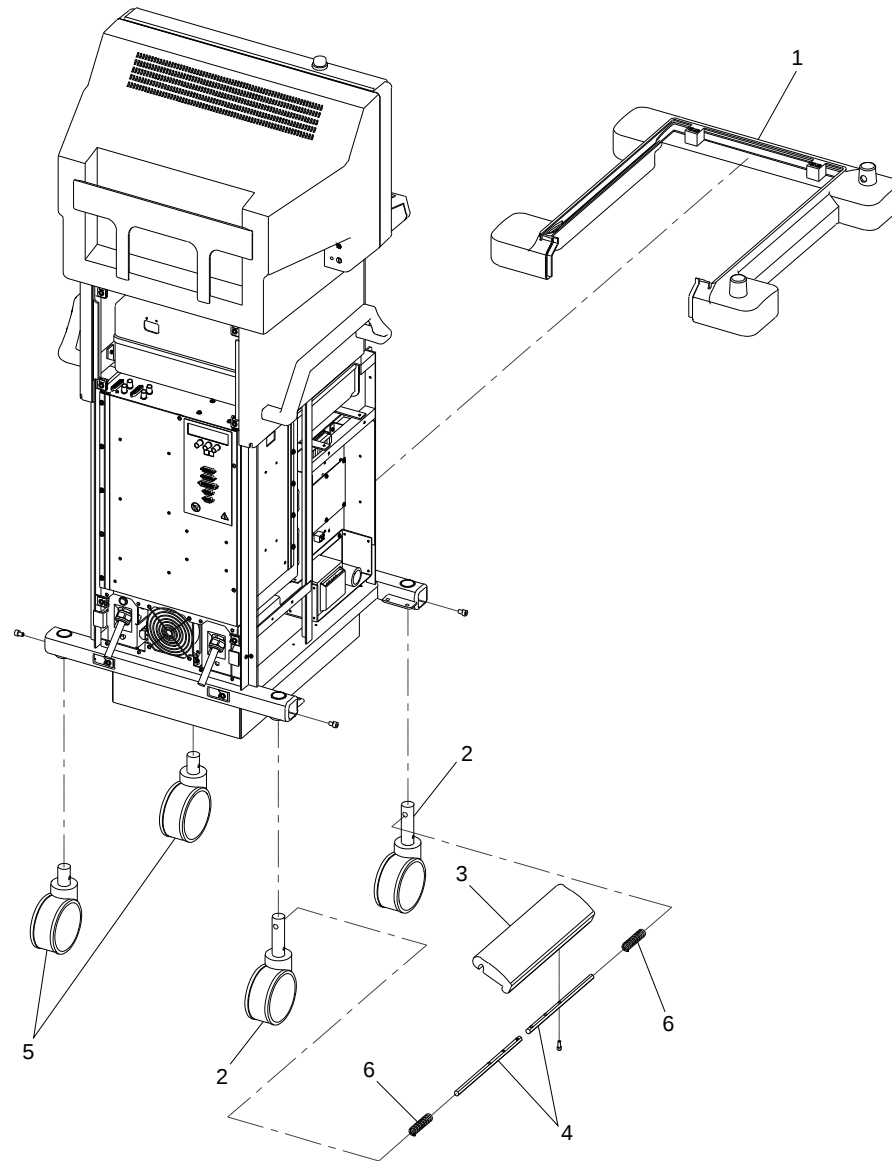


Fig. 1-6 Brake, Caster Exploded View



FIG. 1-7 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-876504-01	ASSEMBLY, PCB, AUX INTERFACE, 9600	1
2	00-872239-01	ASSEMBLY, PCB, VIDEO SWITCHING 4 GEN	1
3	67-700000-00	METER, ELEC, DC, HR/TNTS, 6 DIGIT	1
4	00-900689-01	POWER, SUPPLY, QUAD OUTPUT (SCD)	1
5	00-876841-01	ASSEMBLY, PCB, DC POWER DIST, 9600	1
6	20-839956-00	FAN, AXL, 115 VAC, 75 CFM, 2200 RPM	2
7	20-839926-00	GUARD, FINGER, FAN	2
8	00-876995-01	MOUNT, HARD DRIVE MON CART	1
9	00-875954-01	ASSEMBLY, PCB, IMAGE PROCESSOR, SMT	1
10	00-876399-02	ASSEMBLY, PCB, FAST SCAN CONVERTER	1
11	00-872127-04	ASSEMBLY, PCB, COMMUNICATIONS 4 GEN	1
12	00-900658-01	PCA, MOTHERBOARD, 386 (SCD)	1
13	00-900669-01	DRIVE FLOPPY DISK, 3.5", 1.44 MB	1
14	00-877024-01	DISK, FORMATTED, DISK DRIVE, 42 MB	1
15	00-876712-01	MOUNT, MOTHERBOARD, 9600 (NOT SHOWN)	1
16	00-876284-01	BRACKET, MOUNTING, FLOPPY DRIVE (NOT SHOWN)	1

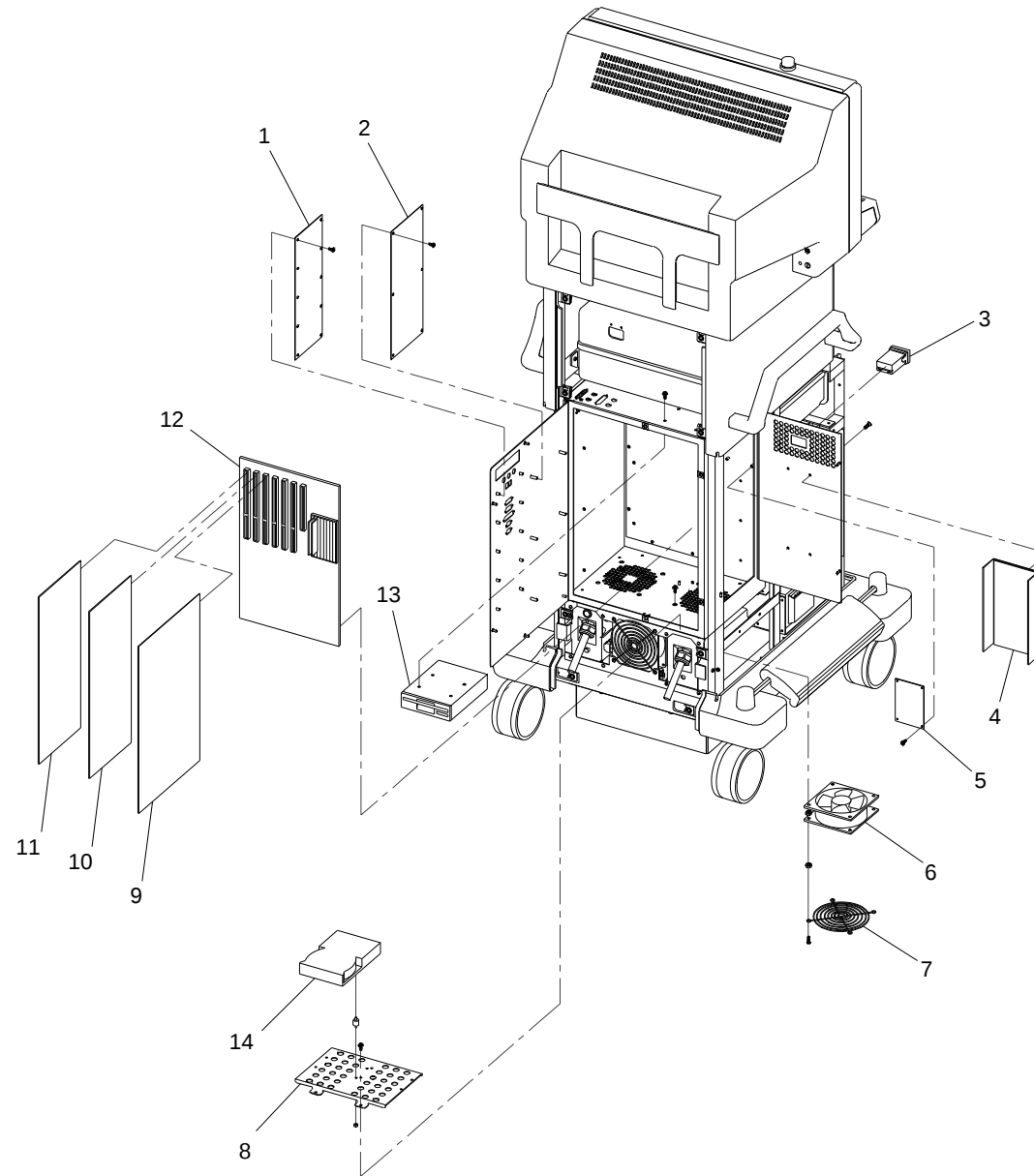


Fig. 1-7 Electronic Rack, Generic Exploded View



FIG. 1-8 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-876504-01	ASSEMBLY, PCB, AUX INTERFACE, 9600	1
2	00-872239-01	ASSEMBLY, PCB, VIDEO SWITCHING 4 GEN	1
3	00-876536-01	CABLE ASSEMBLY, POWER ETI	1
4	00-900689-01	POWER, SUPPLY, QUAD OUTPUT (SCD)	1
5	00-876841-01	ASSEMBLY, PCB, DC POWER DIST, 9600	1
6	20-839956-00	FAN, AXL, 115 VAC, 75 CFM, 2200 RPM	2
7	20-839926-00	GUARD, FINGER, FAN	2
8	00-876995-01	MOUNT, HARD DRIVE MON CART	1
9	00-875954-01	ASSEMBLY, PCB, IMAGE PROCESSOR, SMT	1
10	00-876399-02	ASSEMBLY, PCB, FAST SCAN CONVERTER	1
11	00-872127-04	ASSEMBLY, PCB, COMMUNICATIONS 4 GEN	1
12	00-900658-01	PCA, MOTHERBOARD, 386 (SCD)	1
13	00-900669-01	DRIVE FLOPPY DISK, 3.5", 1.44 MB	1
14	00-877025-01	DISK, FORMATTED, DISK DRIVE, SCSI	1
15	0-878229-01	DISK CONT, TESTED ALTA	1
16	00-900659-01	DISK CONTROLLER, SCSI (SCD)	1
17	00-876712-01	MOUNT, MOTHERBOARD, 9600 (NOT SHOWN)	1
18	00-876284-01	BRACKET, MOUNTING, FLOPPY DRIVE (NOT SHOWN)	1

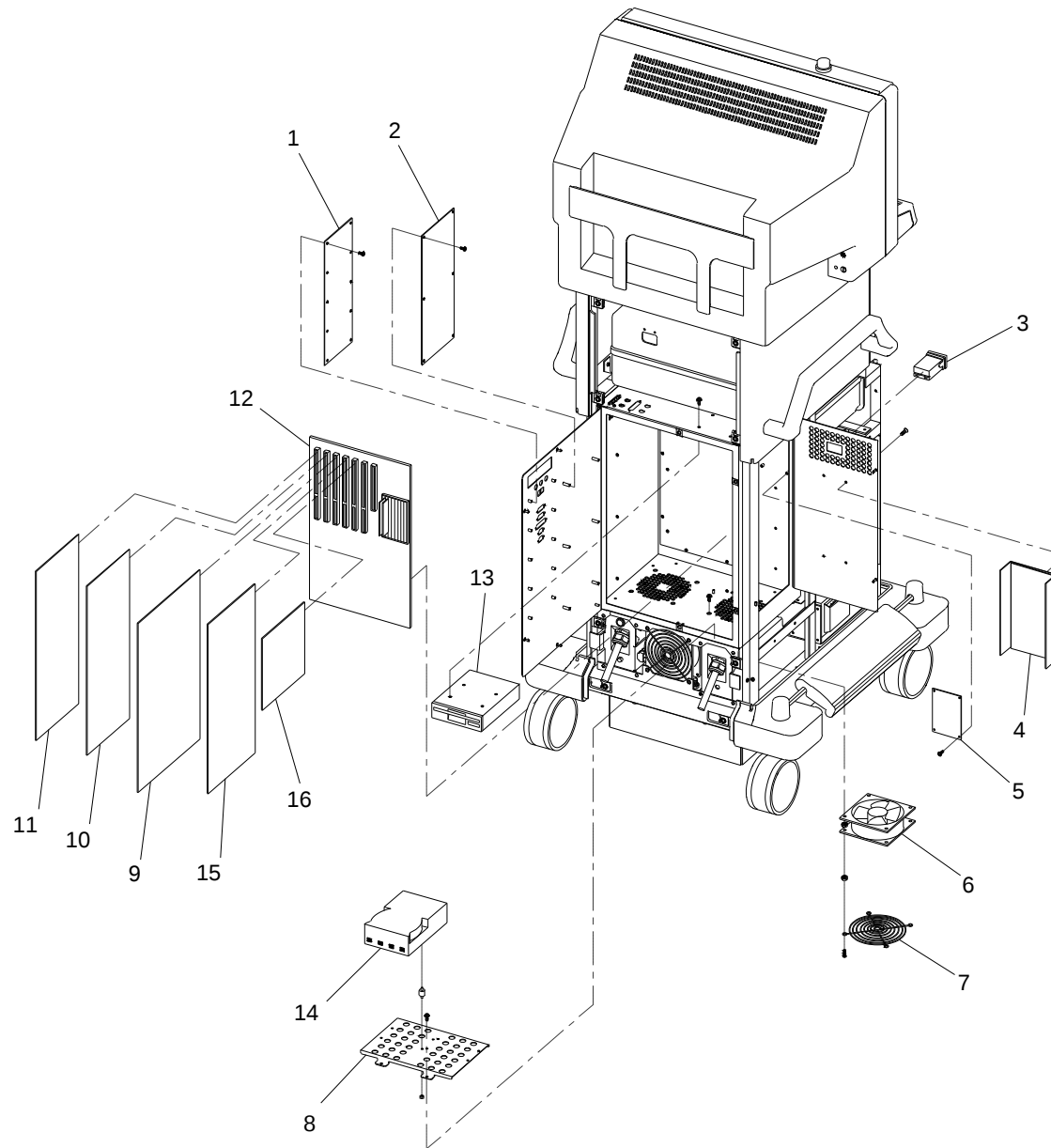


Fig. 1-8 Electronic Rack, ESP with 4 FPS Storage Exploded View



FIG. 1-9 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-876504-01	ASSEMBLY, PCB, AUX INTERFACE, 9600	1
2	00-872239-01	ASSEMBLY, PCB, VIDEO SWITCHING 4 GEN	1
3	00-876536-01	CABLE ASSEMBLY, POWER ETI	1
4	00-900689-01	POWER, SUPPLY, QUAD OUTPUT (SCD)	1
5	00-876841-01	ASSEMBLY, PCB, BC POWER DIST, 9600	1
6	20-839956-00	FAN, AXL, 115 VAC, 75 CFM, 2200 RPM	2
7	20-839926-00	GUARD, FINGER, FAN	2
8	00-876995-01	MOUNT, HARD DRIVE MON CART	1
9	00-875954-01	ASSEMBLY, PCB, IMAGE PROCESSOR, SMT	1
10	00-876399-02	ASSEMBLY, PCB, FAST SCAN CONVERTER	1
11	00-872127-04	ASSEMBLY, PCB, COMMUNICATIONS 4 GEN	1
12	00-900658-01	PCA, MOTHERBOARD, 386 (SCD)	1
13	00-900669-01	DRIVE FLOPPY DISK, 3.5", 1.44 MB	1
14	00-878230-01	ASSEMBLY, DISK DRIVE, 30 FPS, FORMATTED	2
15	0-878229-01	DISK CONT, TESTED ALTA	2
16	00-900659-01	DISK CONTROLLER, SCSI (SCD)	1
17	00-876712-01	MOUNT, MOTHERBOARD, 9600 (NOT SHOWN)	1
18	00-876284-01	BRACKET, MOUNTING, FLOPPY DRIVE (NOT SHOWN)	1

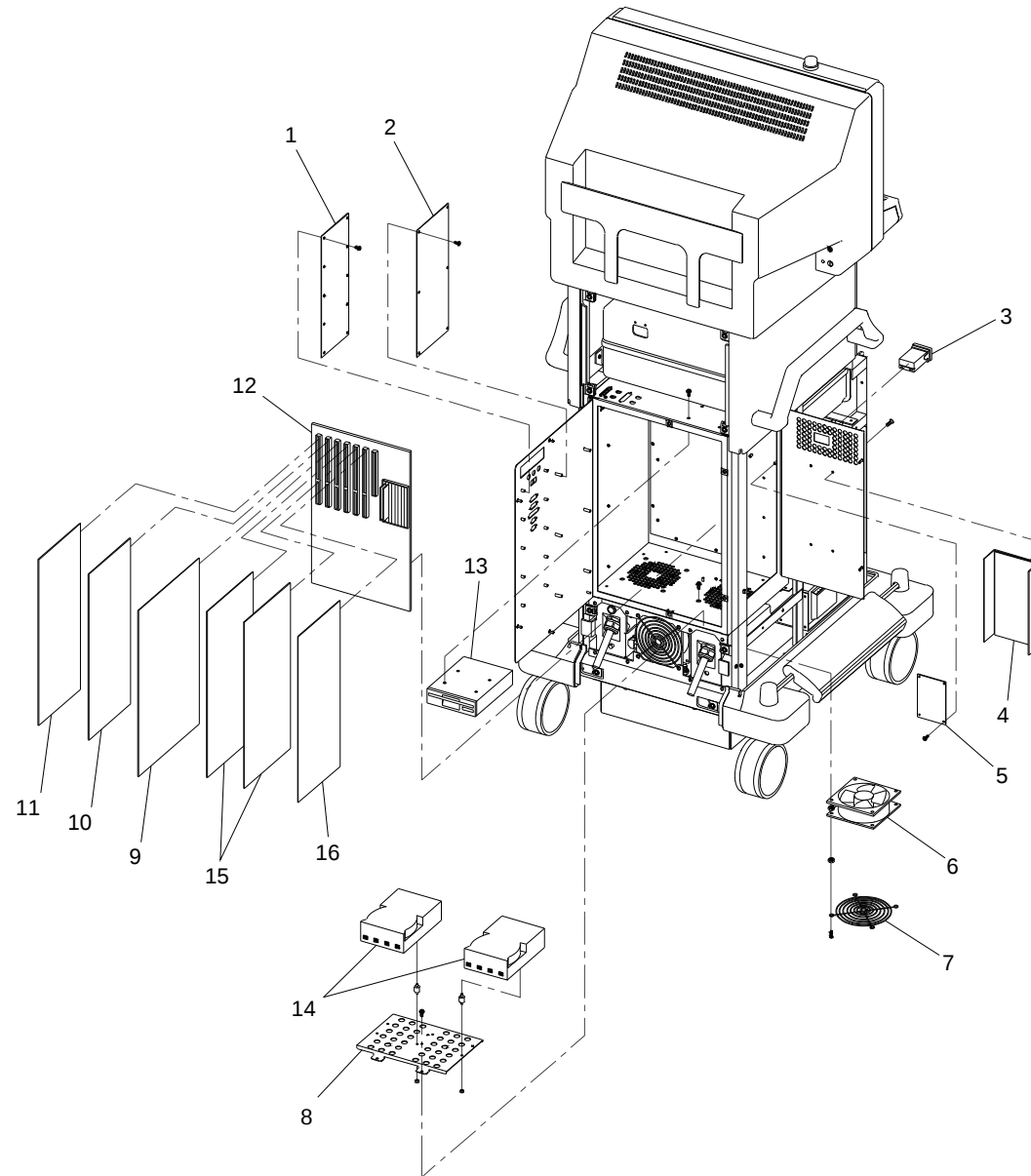


Fig. 1-9 Electronic Rack, Vascular with 30 FPS Storage Exploded View

FIG. 1-10 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-875988-01	COVER, KEYBOARD, TOP	1
2	00-900769-01	PANEL ASSEMBLY, CONTROL, CART SP, 9600 (ENGLISH)	1
2	00-900769-02	PANEL ASSEMBLY, CONTROL, CART SP, 9600 (GERMAN)	1
2	00-900769-03	PANEL ASSEMBLY, CONTROL, CART SP, 9600 (FRENCH)	1
2	00-900769-04	PANEL ASSEMBLY, CONTROL, CART SP, 9600 (SPANISH)	1
2	00-900769-05	PANEL ASSEMBLY, CONTROL, CART SP, 9600 (ITALIAN)	1
3	00-900641-01	PANEL ASSEMBLY, QWERTY KEYBOARD ENGLISH 96	1
4	00-900772-01	PANEL ASSEMBLY, CONTROL, FRONT SP, 9600 (ENGLISH)	1
4	00-900772-02	PANEL ASSEMBLY, CONTROL, FRONT SP, 9600 (GERMAN)	1
4	00-900772-03	PANEL ASSEMBLY, CONTROL, FRONT SP, 9600 (FRENCH)	1
4	00-900772-04	PANEL ASSEMBLY, CONTROL, FRONT SP, 9600 (SPANISH)	1
4	00-900772-05	PANEL ASSEMBLY, CONTROL, FRONT SP, 9600 (ITALIAN)	1
5	00-876767-01	ASSEMBLY, ALARM, AUDIO	1
6	00-877670-01	COVER, BOX, RFI, 9600 WS	
7	00-876613-01	ASSEMBLY, PCB, CONT PANEL PROC, 9600 MC	1
8	00-877671-01	BASE, BOX, RFI., 9600 WS	1
9	00-876224-01	NAMEPLATE, PRODUCT, 9600 (NOT EXPLODED)	1
10	00-876533-01	CABLE ASSEMBLY, QW-CPP (NOT SHOWN)	1
11	00-876693-01	CABLE ASSEMBLY, FRONT PANEL-CPP (NOT SHOWN)	1
12	00-876538-01	CABLE ASSEMBLY, CONTROL PANEL TO CPP (NOT SHOWN)	1

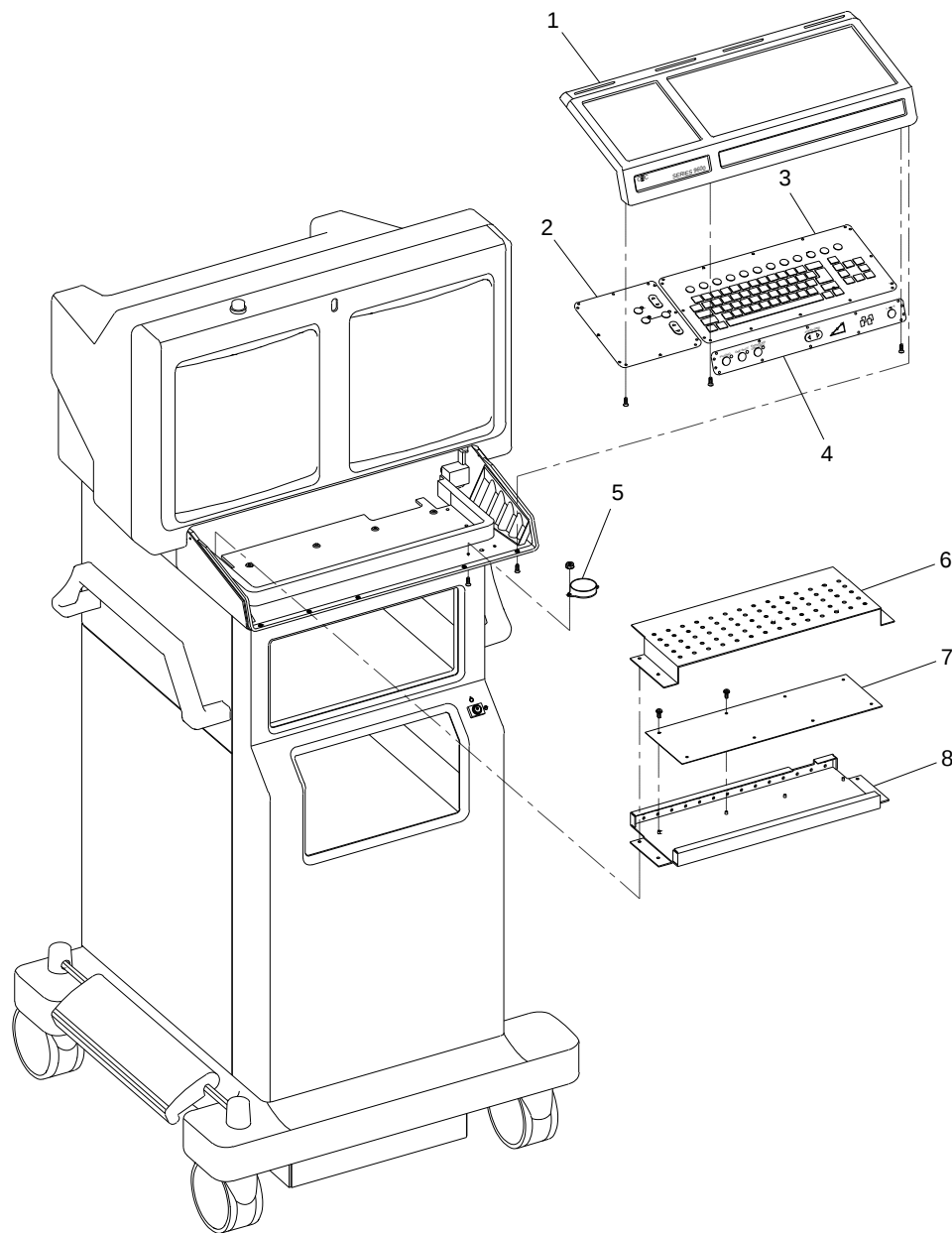


Fig. 1-10 Keyboard, SP (Surgical Platform) Model Exploded View

FIG. 1-11 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-875988-01	COVER, KEYBOARD, TOP	1
2	00-900768-01	PANEL ASSEMBLY, CONTROL, CART ESP, 9600 (ENGLISH)	1
2	00-900768-02	PANEL ASSEMBLY, CONTROL, CART ESP, 9600 (GERMAN)	1
2	00-900768-03	PANEL ASSEMBLY, CONTROL, CART ESP, 9600 (FRENCH)	1
2	00-900768-04	PANEL ASSEMBLY, CONTROL, CART ESP, 9600 (SPANISH)	1
2	00-900768-05	PANEL ASSEMBLY, CONTROL, CART ESP, 9600 (ITALIAN)	1
3	00-900641-01	PANEL ASSEMBLY, QWERTY KEYBOARD ENGLISH 96	1
4	00-900771-01	PANEL ASSEMBLY, FRONT, ESP, ESP/4F, 9600 (ENGLISH)	1
4	00-900771-02	PANEL ASSEMBLY, FRONT, ESP, ESP/ACQ, 9600 (GERMAN)	1
4	00-900771-03	PANEL ASSEMBLY, FRONT, ESP, ESP/ACQ, 9600 (FRENCH)	1
4	00-900771-04	PANEL ASSEMBLY, FRONT, ESP, ESP/ACQ, 9600 (SPANISH)	1
4	00-900771-05	PANEL ASSEMBLY, FRONT, ESP, ESP/ACQ, 9600 (ITALIAN)	1
5	00-876767-01	ASSEMBLY, ALARM, AUDIO	1
6	00-877670-01	COVER, BOX, RFI, 9600 WS	
7	00-876613-01	ASSEMBLY, PCB, CONT PANEL PROC, 9600 MC	1
8	00-877671-01	BASE, BOX, RFI., 9600 WS	1
9	00-876224-01	NAMEPLATE, PRODUCT, 9600 (NOT EXPLODED)	1
10	00-876533-01	CABLE ASSEMBLY, QW-CPP (NOT SHOWN)	1
11	00-876693-01	CABLE ASSEMBLY, FRONT PANEL-CPP (NOT SHOWN)	1
12	00-876538-01	CABLE ASSEMBLY, CONTROL PANEL TO CPP (NOT SHOWN)	1

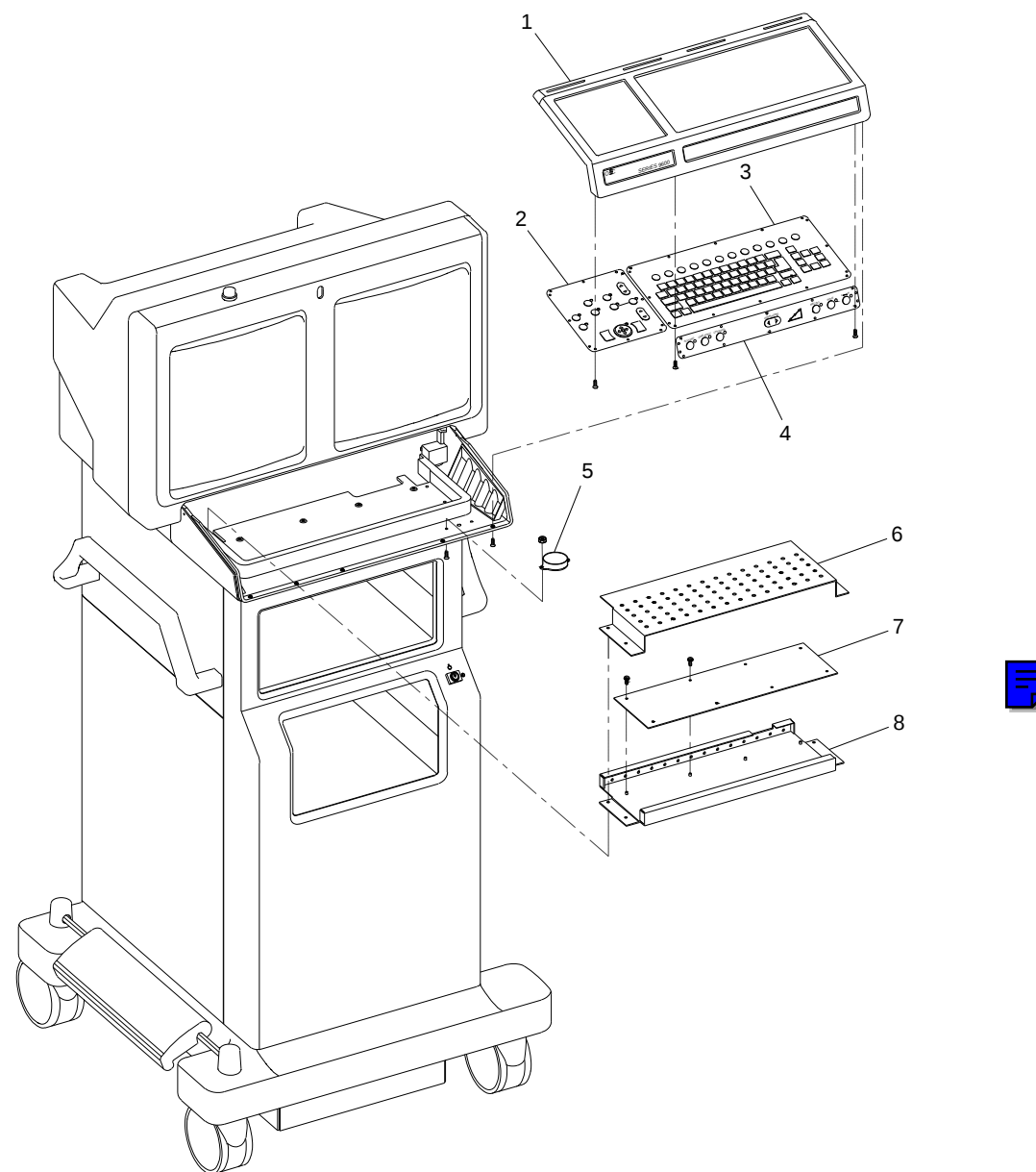


Fig. 1-11 Keyboard, ESP/Cholangiography Model Exploded View

FIG. 1-12 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-875988-01	COVER, KEYBOARD, TOP	1
2	00-900767-01	PANEL ASSEMBLY, CONTROL, CART ESP/4F, 9600 (ENGLISH)	1
2	00-900767-02	PANEL ASSEMBLY, CONTROL, CART ESP/ACQ, 9600 (GERMAN)	1
2	00-900767-03	PANEL ASSEMBLY, CONTROL, CART ESP/ACQ, 9600 (FRENCH)	1
2	00-900767-04	PANEL ASSEMBLY, CONTROL, CART ESP/ACQ, 9600 (SPANISH)	1
2	00-900767-05	PANEL ASSEMBLY, CONTROL, CART ESP/ACQ, 9600 (ITALIAN)	1
3	00-900641-01	PANEL ASSEMBLY, QWERTY KEYBOARD ENGLISH 9600	1
4	00-900771-01	PANEL ASSEMBLY, FRONT, ESP & ESP/4F, 9600 (ENGLISH)	1
4	00-900771-02	PANEL ASSEMBLY, FRONT, ESP & ESP/ACQ, 9600 (GERMAN)	1
4	00-900771-03	PANEL ASSEMBLY, FRONT, ESP & ESP/ACQ, 9600 (FRENCH)	1
4	00-900771-04	PANEL ASSEMBLY, FRONT, ESP & ESP/ACQ, 9600 (SPANISH)	1
4	00-900771-05	PANEL ASSEMBLY, FRONT, ESP & ESP/ACQ, 9600 (ITALIAN)	1
5	00-876767-01	ASSEMBLY, ALARM, AUDIO	1
6	00-877670-01	COVER, BOX, RFI, 9600 WS	
7	00-876613-01	ASSEMBLY, PCB, CONT PANEL PROC, 9600 MC	1
8	00-877671-01	BASE, BOX, RFI., 9600 WS	1
9	00-876224-01	NAMEPLATE, PRODUCT, 9600 (NOT EXPLODED)	1
10	00-876533-01	CABLE ASSEMBLY, QW-CPP (NOT SHOWN)	1
11	00-876693-01	CABLE ASSEMBLY, FRONT PANEL-CPP (NOT SHOWN)	1
12	00-876538-01	CABLE ASSEMBLY, CONTROL PANEL TO CPP (NOT SHOWN)	1

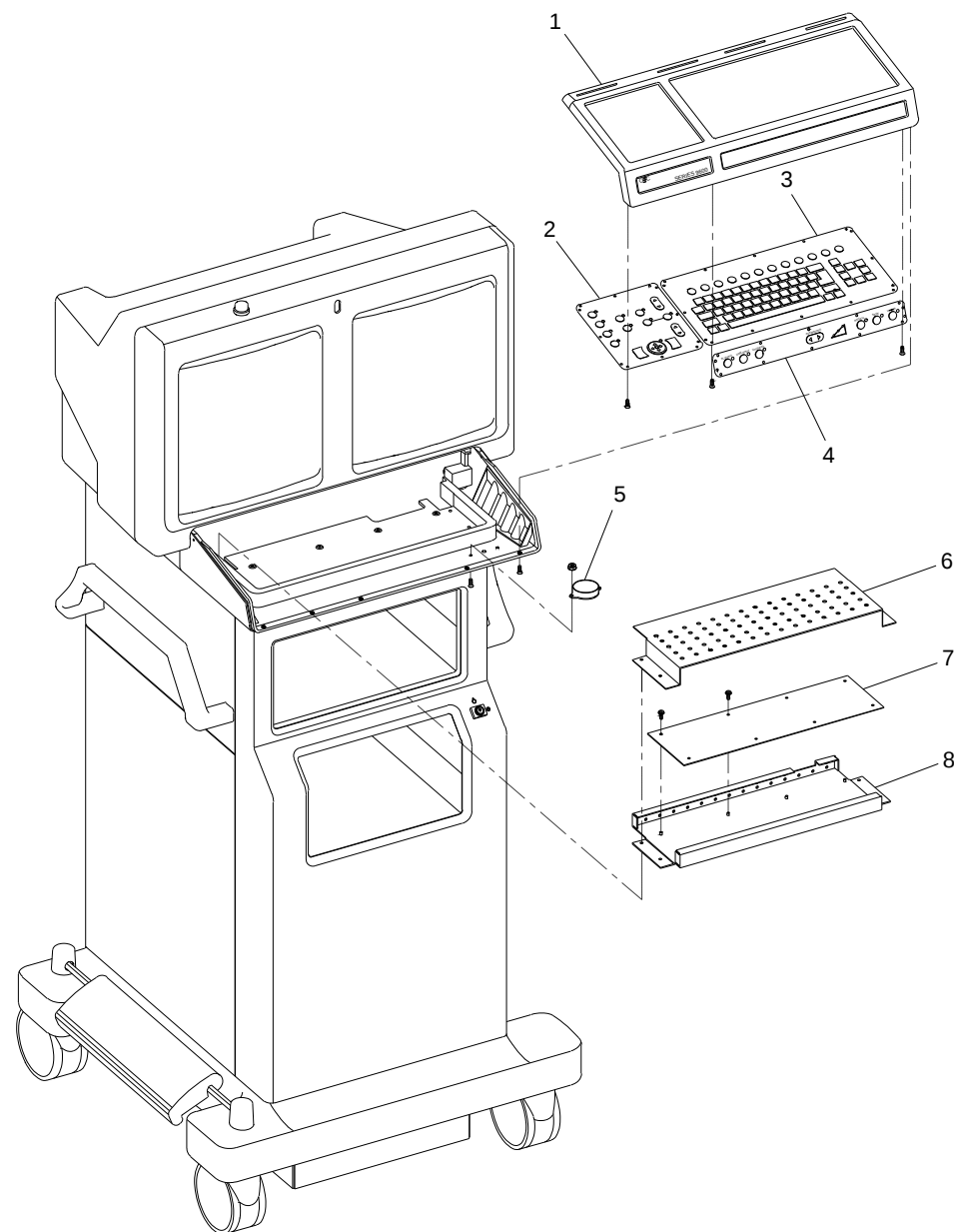


Fig. 1-12 Keyboard, ESP/Acquire Model Exploded View

FIG. 1-13 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-875988-01	COVER, KEYBOARD, TOP	1
2	00-900766-01	PANEL ASSEMBLY, CONTROL, CART VAS, 9600 (ENGLISH)	1
2	00-900766-02	PANEL ASSEMBLY, CONTROL, CART VAS, 9600 (GERMAN)	1
2	00-900766-03	PANEL ASSEMBLY, CONTROL, CART VAS, 9600 (FRENCH)	1
2	00-900766-04	PANEL ASSEMBLY, CONTROL, CART VAS, 9600 (SPANISH)	1
2	00-900766-05	PANEL ASSEMBLY, CONTROL, CART VAS, 9600 (ITALIAN)	1
3	00-900641-01	PANEL ASSEMBLY, QWERTY KEYBOARD ENGLISH 96	1
4	00-900770-01	PANEL ASSEMBLY, CONTROL, FRONT, VAS, (ENGLISH)	1
4	00-900770-02	PANEL ASSEMBLY, CONTROL, FRONT, VAS, (GERMAN)	1
4	00-900770-03	PANEL ASSEMBLY, CONTROL, FRONT, VAS, (FRENCH)	1
4	00-900770-04	PANEL ASSEMBLY, CONTROL, FRONT, VAS, (SPANISH)	1
4	00-900770-05	PANEL ASSEMBLY, CONTROL, FRONT, VAS, (ITALIAN)	1
5	00-876767-01	ASSEMBLY, ALARM, AUDIO	1
6	00-877670-01	COVER, BOX, RFI, 9600 WS	
7	00-876613-01	ASSEMBLY, PCB, CONT PANEL PROC, 9600 MC	1
8	00-877671-01	BASE, BOX, RFI., 9600 WS	1
9	00-876224-01	NAMEPLATE, PRODUCT, 9600 (NOT EXPLODED)	1
10	00-876533-01	CABLE ASSEMBLY, QW-CPP (NOT SHOWN)	1
11	00-876693-01	CABLE ASSEMBLY, FRONT PANEL-CPP (NOT SHOWN)	1
12	00-876538-01	CABLE ASSEMBLY, CONTROL PANEL TO CPP (NOT SHOWN)	1

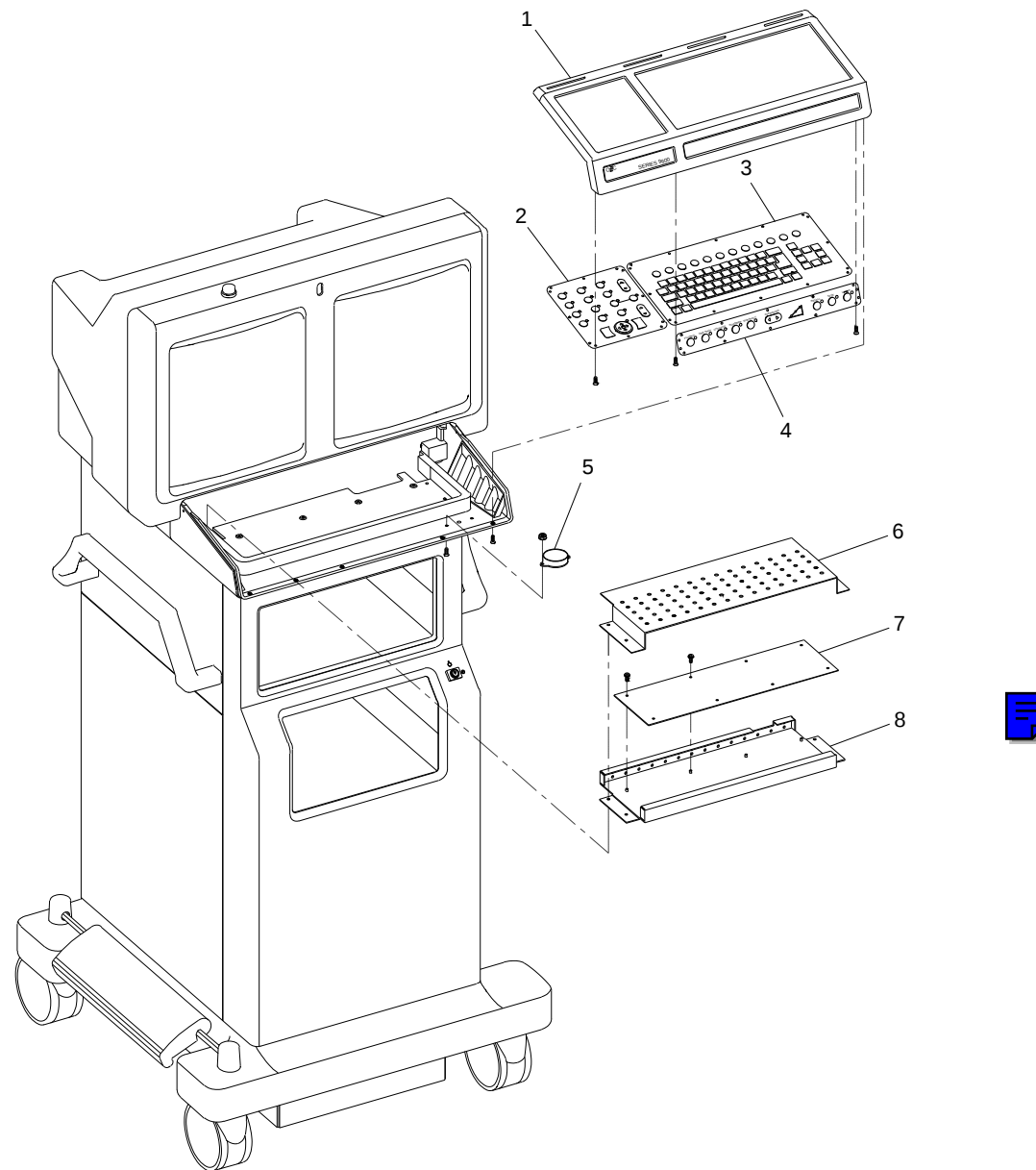


Fig. 1-13 Keyboard, ESP Vascular, Neuro-Vascular Model Exploded View

FIG. 1-14A PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-876538-01	CABLE ASSEMBLY, CONTROL PANEL TO CPP	1
2	00-876533-01	CABLE ASSEMBLY, QW-CPP	1
3	00-876693-01	CABLE ASSEMBLY, FRONT PANEL-CPP	1
4	00-876846-04	CABLE ASSEMBLY, GROUND, CONT PANELS, 9600	1
5	00-876846-05	CABLE ASSEMBLY, GROUND, CONT PANELS, 9600	1
6	00-876846-06	CABLE ASSEMBLY, GROUND, CONT PANELS, 9600	1
7	00-876530-01	CABLE ASSEMBLY, CPP/REM CONT/X-RAY ON	3
8	00-876770-01	CABLE ASSEMBLY, X-RAY ON LAMP	1
9	00-876767-01	ASSEMBLY, ALARM, AUDIO	1
10	00-876541-01	CABLE ASSEMBLY, CONT/BRT POTS	2
11	00-876541-02	CABLE ASSEMBLY, CONT/BRT POTS	2
12	00-876540-01	CABLE ASSEMBLY, CONT/BRT RIGHT	1
13	00-876540-02	CABLE ASSEMBLY, CONT/BRT LEFT	1
14	00-876694-01	CABLE ASSEMBLY, BNC MONITOR RIGHT	1
15	00-876694-02	CABLE ASSEMBLY, BNC MONITOR LEFT	1
16	00-876438-01	HARNESS ASSEMBLY, POWER CONTROL-PS4	2

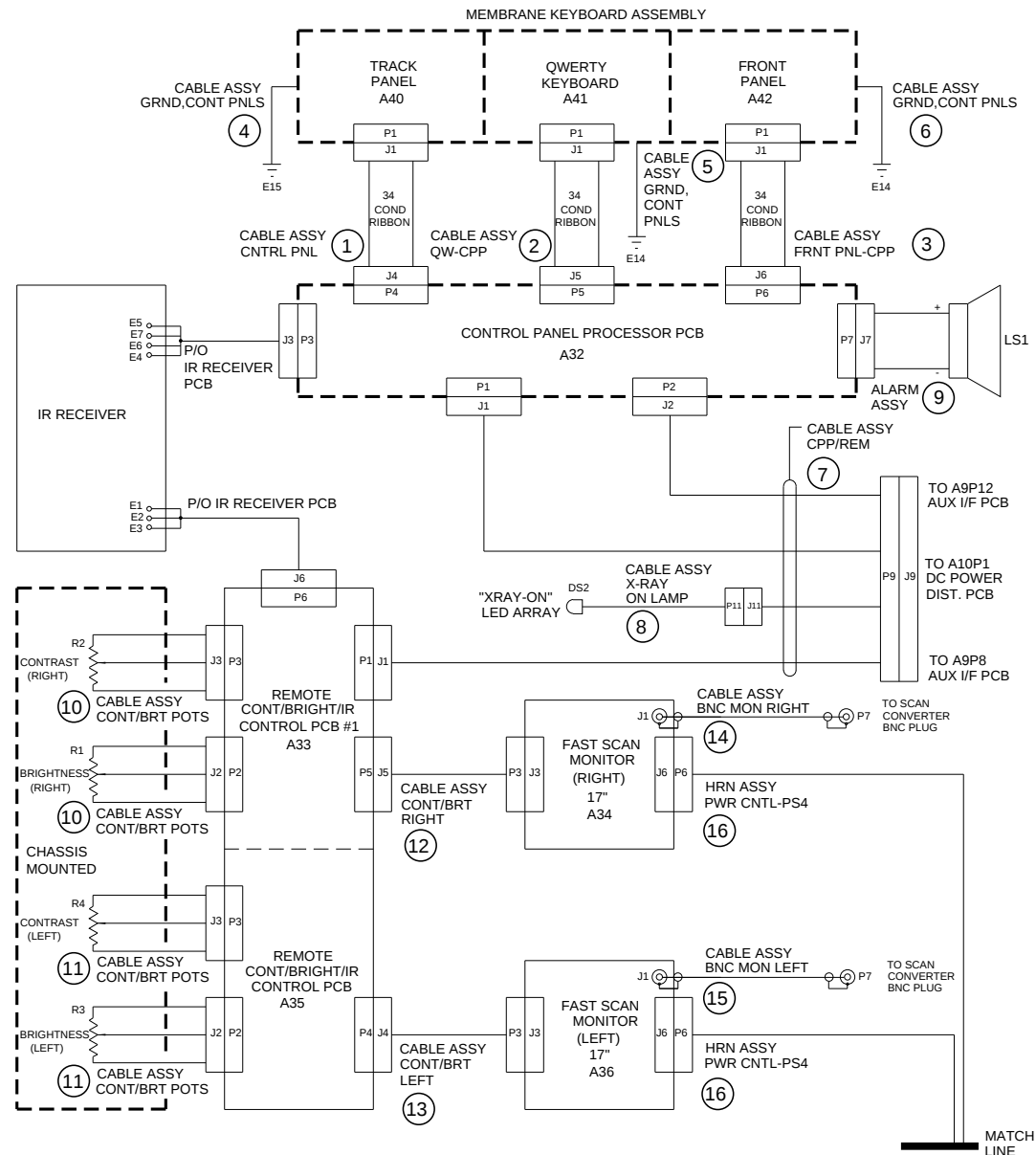


Fig. 1-14A Interface Cable Diagram, Imaging Workstation (Sheet 1 of 5)

FIG. 1-14B PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
16	00-876438-01	HARNESS ASSEMBLY, POWER CONTROL-PS4	1
17	00-876181-01	ASSEMBLY, POWER CORD, 220-250V, 20 FT, 9600	1
18	00-876181-02	ASSEMBLY, POWER CORD, 220-250V, 30 FT, 9600	1
19	00-876182-01	ASSEMBLY, POWER CORD, 220V, 60 HZ, 20', 9600	1
20	00-876180-01	ASSEMBLY, POWER CORD, 100-270V, 20 FT, 9600	2
21	00-876435-02	CABLE ASSEMBLY, SAFETY GROUND STRAP	1
22	00-876435-01	CABLE ASSEMBLY, SAFETY GROUND STRAP	1
23	00-876437-01	CABLE ASSEMBLY, SURGE SUP T1	1
24	00-876542-01	CABLE ASSEMBLY, POWER, POWER CONTROL PCB	1
25	00-876812-01	CABLE ASSEMBLY, GROUND CURRENT LIMITER	1
26	00-876183-01	ASSEMBLY, CABLE, INTERCONNECT, 20', 9600	2
27	00-876184-01	ASSEMBLY, CABLE, INTERCONNECT, 30', 9600	2
28	00-876543-01	CABLE ASSEMBLY, POWER CONTROL TO TB2	1
29	00-876998-01	CABLE ASSEMBLY, POWER CONTROL TO J15	1



Fig. 1-14B Interface Cable Diagram, Imaging Workstation (Sheet 2 of 5)

FIG. 1-14C PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
26	00-876183-01	CABLE ASSEMBLY, INTERCONNECT, 20', 9600	2
27	00-876183-02	CABLE ASSEMBLY, INTERCONNECT, 30', 9600	2
28	00-876543-01	CABLE ASSEMBLY, POWER CONT TO TB2	1
29	00-876998-01	CABLE ASSEMBLY, POWER CONT TO J15	1
30	00-876544-01	CABLE ASSEMBLY, KEY SWITCH TO P15	1
31	00-876161-01	KIT, LENZAR, 60 HZ, 9600, ENGLISH	3
32	00-876846-07	CABLE ASSEMBLY, GROUND, CONT PANELS, 9600	1
33	00-877009-01	ASSEMBLY, OUTLET STRIP, INTERNATIONAL AC	1
34	00-876803-01	CABLE ASSEMBLY, VIDEO, INT, VCR, 9600	1
35	00-876802-01	CABLE ASSEMBLY, CONT, INT, VCR, 9600	1
36	00-876803-02	CABLE ASSEMBLY, VIDEO, INT, VCR, 9600	1
37	00-876444-01	CABLE ASSEMBLY, TB2-J16	1
38	00-900740-01	CABLE ASSEMBLY, A9P10 TO J12, 13, 14	1

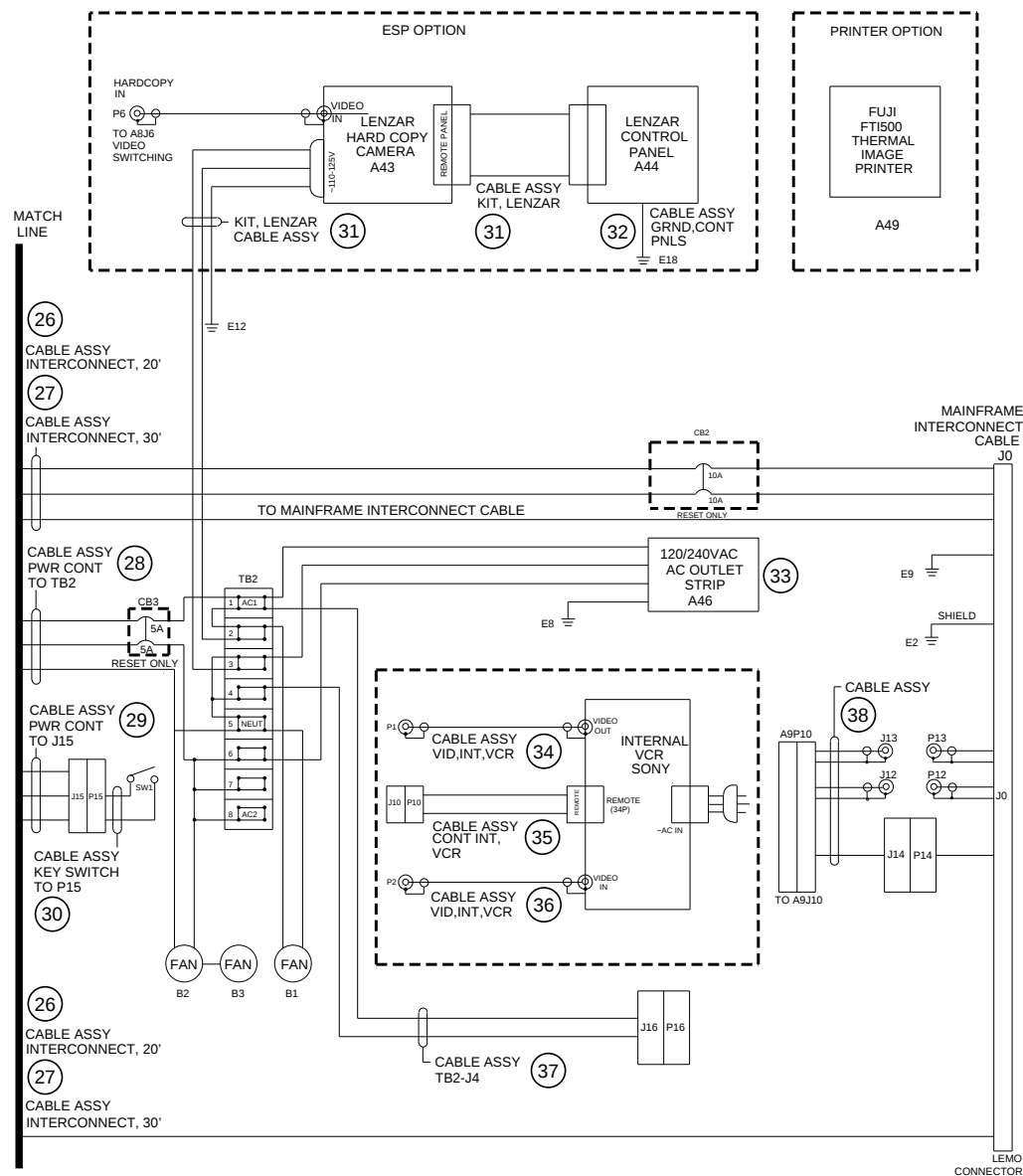


Fig. 1-14C Interface Cable Diagram, Imaging Workstation (Sheet 3 of 5)

FIG. 1-14D PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
39	00-900720-01	CABLE ASSEMBLY, VCR1/VCR2, 9600	2
40	00-900462-01	CABLE, SIGNAL, VIDEO, SW TO AUX	1
41	00-876448-01	CABLE ASSEMBLY, SIGNAL, VIDEO, SW/AUX IF	1
42	00-876442-01	CABLE ASSEMBLY, SIGNAL, VIDEO, SW/AUX I/F	1
43	00-876449-01	CABLE ASSEMBLY, SIGNAL, AUX INTF/VIDEO SW	1
44	00-900719-01	CABLE ASSEMBLY, VIDEO, 9600	1
45	00-876453-01	CABLE ASSEMBLY, SIGNAL, IP/VIDEO, SW	1
46	00-900721-01	CABLE ASSEMBLY, COAX, VIDEO SW-IP, 9600	1
47	00-876634-01	CABLE ASSEMBLY, POWER, IP, VIDEO SW & AUX I/F	2
48	00-876534-01	CABLE ASSEMBLY, INTERNAL VCR CONTROL	1
49	00-876450-01	CABLE ASSEMBLY, SIGNAL AT COM 26 PIN	1
50	00-876451-01	CABLE ASSEMBLY, SIGNAL AT COM 34 PIN	1
51	00-876452-01	CABLE ASSEMBLY, SIGNAL AT COM 50 PIN	1
52	00-876550-01	CABLE ASSEMBLY, COAX, NETWORK	1
53	00-876447-01	CABLE ASSEMBLY, POWER, DISK DRIVES	1
54	00-876446-01	CABLE ASSEMBLY, POWER, MOTHERBOARD	1
55	00-876394-01	CABLE ASSEMBLY, DISK DRIVE & ALTA	1
56	00-877291-01	CABLE ASSEMBLY, 30 FPS, HARD DRIVE POWER, 9600	1
57	00-876536-01	CABLE ASSEMBLY, POWER, ETI	1
58	00-876445-01	CABLE ASSEMBLY, POWER SUPPLY	1
59	00-876546-01	CABLE ASSEMBLY, PS1 TO A10	1
60	00-876439-01	CABLE ASSEMBLY, DC POWER & AUX I/F TO J9	1

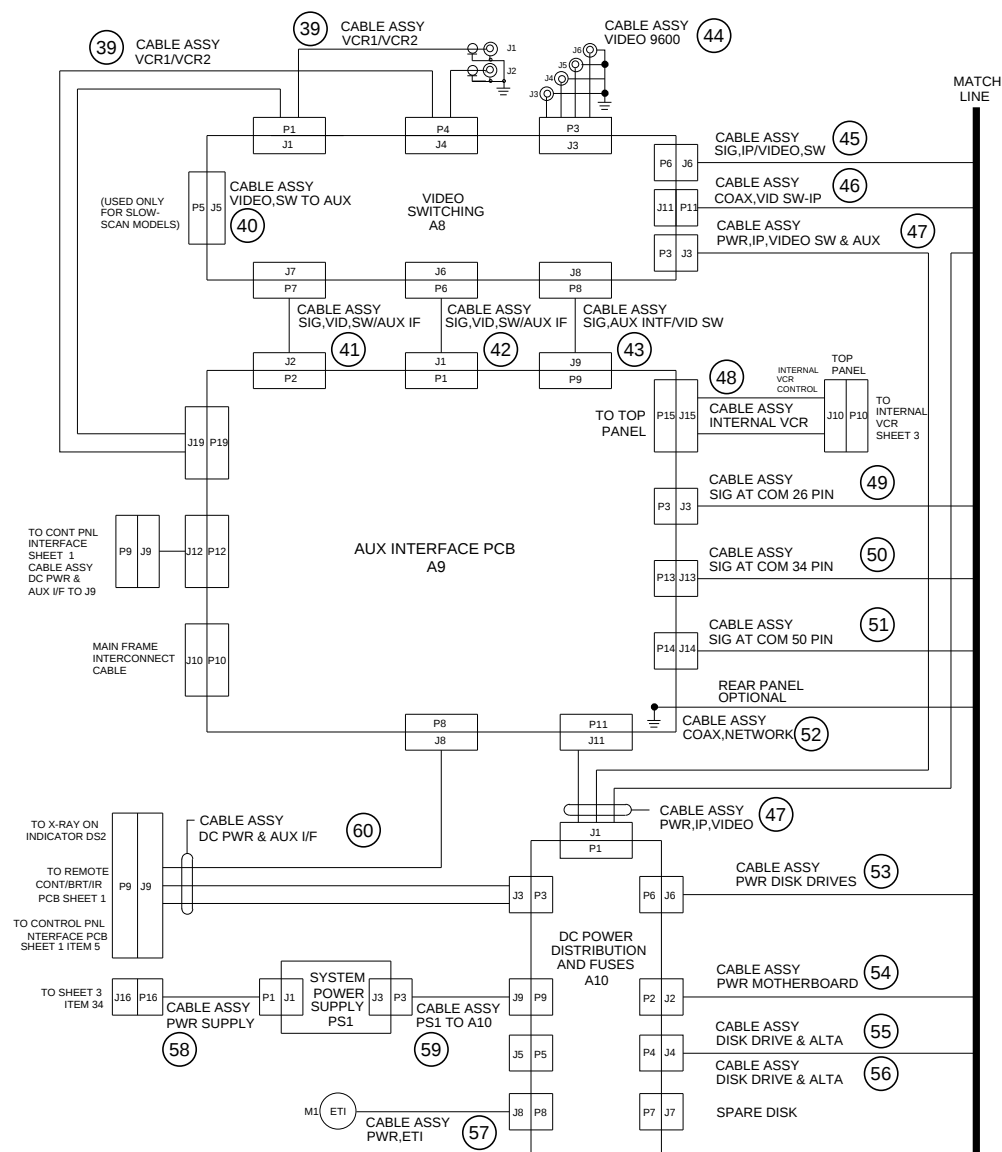


Fig. 1-14D Interface Cable Diagram, Imaging Workstation (Sheet 4 of 5)

FIG. 1-14E PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
45	00-876453-01	CABLE ASSEMBLY, SIGNAL, IP/VIDEO, SW	1
46	00-900721-01	CABLE ASSEMBLY, COAX, VIDEO SW-IP, 9600	1
47	00-876634-01	CABLE ASSEMBLY, POWER, IP, VIDEO SW & AUX I/F	2
48	00-876534-01	CABLE ASSEMBLY, INTERNAL VCR CONT (FIG. 1-14D)	1
49	00-876450-01	CABLE ASSEMBLY, SIGNAL AT COM 26 PIN	1
50	00-876451-01	CABLE ASSEMBLY, SIGNAL AT COM 34 PIN	1
51	00-876452-01	CABLE ASSEMBLY, SIGNAL AT COM 50 PIN	1
52	00-876550-01	CABLE ASSEMBLY, COAX, NETWORK	1
53	00-876447-01	CABLE ASSEMBLY, POWER, DISK DRIVES	1
54	00-876446-01	CABLE ASSEMBLY, POWER, MOTHERBOARD	1
55	00-876394-01	CABLE ASSEMBLY, DISK DRIVE & ALTA	1
56	00-877291-01	CABLE ASSEMBLY, 30 FPS HARD DRIVE POWER, 9600	1
57	00-876536-01	CABLE ASSEMBLY, POWER, ETI (FIG. 1-14D)	1
58	00-876445-01	CABLE ASSEMBLY, POWER SUPPLY (FIG. 1-14D)	1
59	00-876546-01	CABLE ASSEMBLY, PS1 TO A10 (FIG. 1-14D)	1
60	00-876439-01	CABLE ASSEMBLY, DC POWER & AUX I/F TO J9 (FIG. 1-14D)	1
61	00-876455-01	CABLE ASSEMBLY, SIGNAL, FLOPPY DISK	1
62	00-876367-01	CABLE ASSEMBLY, DATA FAST SCAN-IP	1
63	00-877365-01	CABLE ASSEMBLY, SIGNAL, ALTA 2 HD, 2, 30 FPS	1
64	00-877364-01	CABLE ASSEMBLY, SIGNAL, ALTA 1-ALTA, 30 FPS	1
65	00-876395-01	CABLE ASSEMBLY, SIGNAL, ALTA IPI TO IP	1
66	00-877412-01	CABLE ASSEMBLY, SIGNAL, IP-ALTA	1

FIG. 1-14E PARTS LIST (CONT.)

ITEM	PART NUMBER	DESCRIPTION	QTY
67	00-877363-01	CABLE ASSEMBLY, SIGNAL, IP-ALTA, 30 FPS	1
68	00-876393-01	CABLE ASSEMBLY, SIGNAL, ALTA TO SCSI HD	1
69	00-877366-01	CABLE ASSEMBLY, SIGNAL, ALTA-SCSI/HD	1
70	00-876454-01	CABLE ASSEMBLY, SIGNAL, HD, 40 PIN	1
71	00-876366-01	CABLE ASSEMBLY, FAST SCAN-BNC (RIGHT)	1
72	00-876366-02	CABLE ASSEMBLY, FAST SCAN-BNC (LEFT)	1

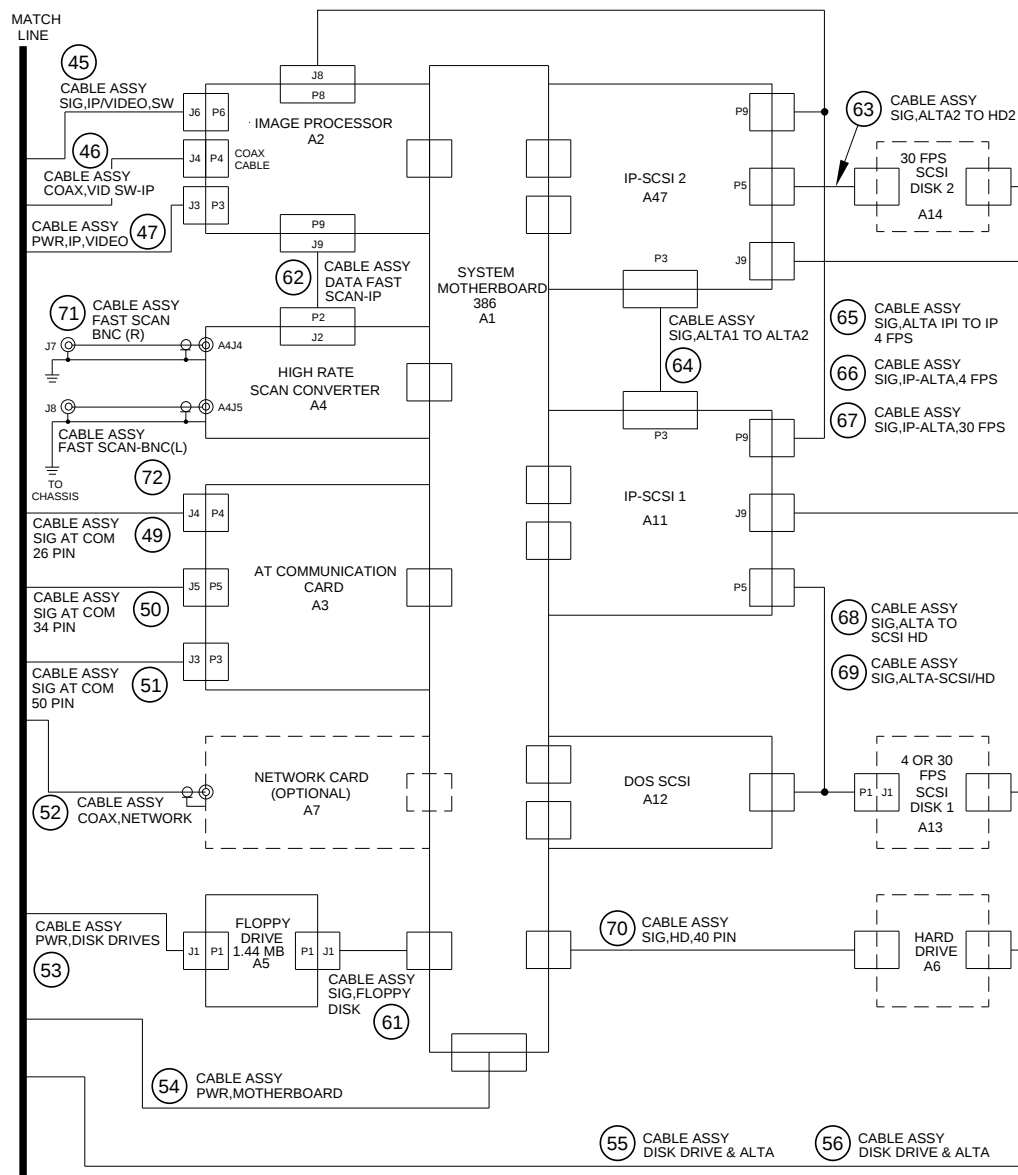


Fig. 1-14E Interface Cable Diagram, Imaging Workstation (Sheet 5 of 5)

ATTACHING HARDWARE

PART NUMBER	DESCRIPTION	QTY
00-876119-01	SPACER, HINGE, SIDE, ELECT BOX	1
00-876990-01	CLIP, SPRING, KEYBOARD	1
00-900407-02	WASHER, COLOR CODED	1
00-900407-03	WASHER, LOCK, SERRATED	1
00-900407-04	NUT, MU 0.8	1
00-900407-05	WASHER, U/M6	1
10-601106-04	SCREW, SLFTAP, PAN, SLT, 6-32, 1/4, SZC	2
12-215004-04	SCREW, MACH, FL, TRX, 4-40, 1/4, SZC	4
12-215006-06	SCREW, MACH, FL, TRX, 6-32, 3/8, SZC	18
12-215006-08	SCREW, MACH, FL, TRX, 6-32, 1/2, SZC	10
12-215006-12	SCREW, MACH, FL, TRX, 6-32, 3/4, SZC	8
12-215008-06	SCREW, MACH, FL, TRX, 8-32, 3/8, SZC	1
12-221104-04	SCREW, MACH, PAN, PHL, 4-40, 1/4, SST	7
12-221104-12	SCREW, MACH, PAN, PHL, 4-40, 3/4, SST	2
12-225104-04	SCREW, SEMS SPRING, PAN, PHL, 4-40, 1/4, SZC	2
12-225104-05	SCREW, SEMS SPRING, PAN, TRX, 4-10, 5/16, SZC	42
12-225104-06	SCREW, SEMS SPRING, PAN, TRX, 4-40, 3/8, SZC	2
12-225106-04	SCREW, SEMS APR, PAN, TRX, 6-32, 1/4, SZC	6
12-225106-06	SCREW, SEMS SPRING, PAN, TRX, 6-32, 3/8, SZC	31
12-225106-08	SCREW, SEMS SPRING, PAN, TRX, 6-32, 1/2, SZC	23
12-225106-12	SCREW, SEMS SPRING, PAN, TRX, 6-32, 3/4, SZC	10


ATTACHING HARDWARE (CONT.)

PART NUMBER	DESCRIPTION	QTY
12-225108-06	SCREW, SEMS SPRING, PAN, TRX, 8-32, 3/8, SZC	41
12-225108-08	SCREW, SEMS SPRING, PAN, TRX, 8/32, 1/2, SZC	14
12-225108-12	SCREW, SEMS SPRING, PAN, TRX, 8/32, 3/4, SZC	2
12-225303-08	SCREW, METRIC SEMS, PAN, TRX, M3X0.5, 6MM, SZ	14
12-312014-10	SCREW, NA, SKT, HEX, 1/4-20, 5/8, SST	14
12-600000-08	SCREW, NA, HEX, CAP, 1/4-20, 1/2, SZY, 8	8
12-900711-04	SCREW, MACH, PAN, SLT, 4-40, 1/4, NYLON	2
12-912008-14	SCREW, METRIC, HEX, CAP, M8X1.25, 12MM, SST	4
13-122104-00	NUT, HEX, 4-40, NYLON, NA, A = .250, C = .094	2
13-311104-00	NUT, HEX-LK-EXT-T, 4-40, SST, NA, A = .250	5
13-311106-00	NUT, HEX-LK-EXT-T, 6-32, SST, NA, A = .313	33
13-311108-00	NUT, HEX-LK-EXT-T, 8-32, SST, NA, A = .344	14
13-311114-00	NUT, HEX-LK-EXT-T, 1/4-20, SST, NA, A = .4	11
13-532011-00	NUT, HEX-LK-W/INSR, 5/16-18, SZC	4
14-101104-00	WASHER, PL, #4, SZC, NA, ID = .125, OD = .312	2
14-102014-00	WASHER, PL, 1/4, SST, NA, ID = .265, OD = .500	8
14-111104-00	WASHER, PL, #4, SST, MS15795-804, ID = .120	5
14-201104-00	WASHER, LK-HLCL-SPR, #4, SST, MS35338-135	8
14-211114-00	WASHER, LK-HLCL-SPR, 1/4, SST, MS35338-	8
14-311112-00	WASHER, LK-EXT-T, 5/16, SST, NA, ID = .320	4

ATTACHING HARDWARE (CONT.)

PART NUMBER	DESCRIPTION	QTY
14-311114-00	WASHER, LK-EXT-T, 1/4, SST, NA, ID = .256	12
14-900002-00	WASHER, PL, #4, SST, NA, ID = .125, OD = .281	2
14-999702-00	WASHER, PL, #4, NYLON, NA, ID = .125, OD = .250	2
21-709008-00	SPRING, CPRSN, .420OD, 3/4LG, .042WD, SST	2
22-012010-00	CLAMP, CABLE, 7/16D, 1/2W, STL	1
22-012101-00	CLAMP, CABLE/W, .437 X .39SQ, .219W, NYLON	18
22-012102-00	CLAMP, CABLE/W, .437 X .75SQ, .219W, NYLON	5
22-012103-00	CLAMP, CABLE/W, .437 X .34SQ, .219W, NYLON	23
22-012104-00	CLAMP, CABLE/W, .437 X 1.02SQ, .219W, NYLON	6
22-119605-00	CLAMP, CABLE TIE MT, NYLON	2
22-119650-00	CLAMP, CABLE TIE, 3/32 X 3-5/8, NYLON	AR
23-620843-00	RETAINER, 1/4TRN, .485OD X .013THK, SST	8
23-621841-00	FASTENER, 1/4TRN STD, PHL, SZC	8
23-621843-00	FASTENER, RCPT, 1/4TRN, CLP-ON, .062/.187	8
23-621844-00	FASTENER, 1/4TRN STD, PHL, MINTR, NYLON	6
23-621845-00	FASTENER, RCPT, 1/4TRN, CLP-ON, .0011/.080	6

Section 2

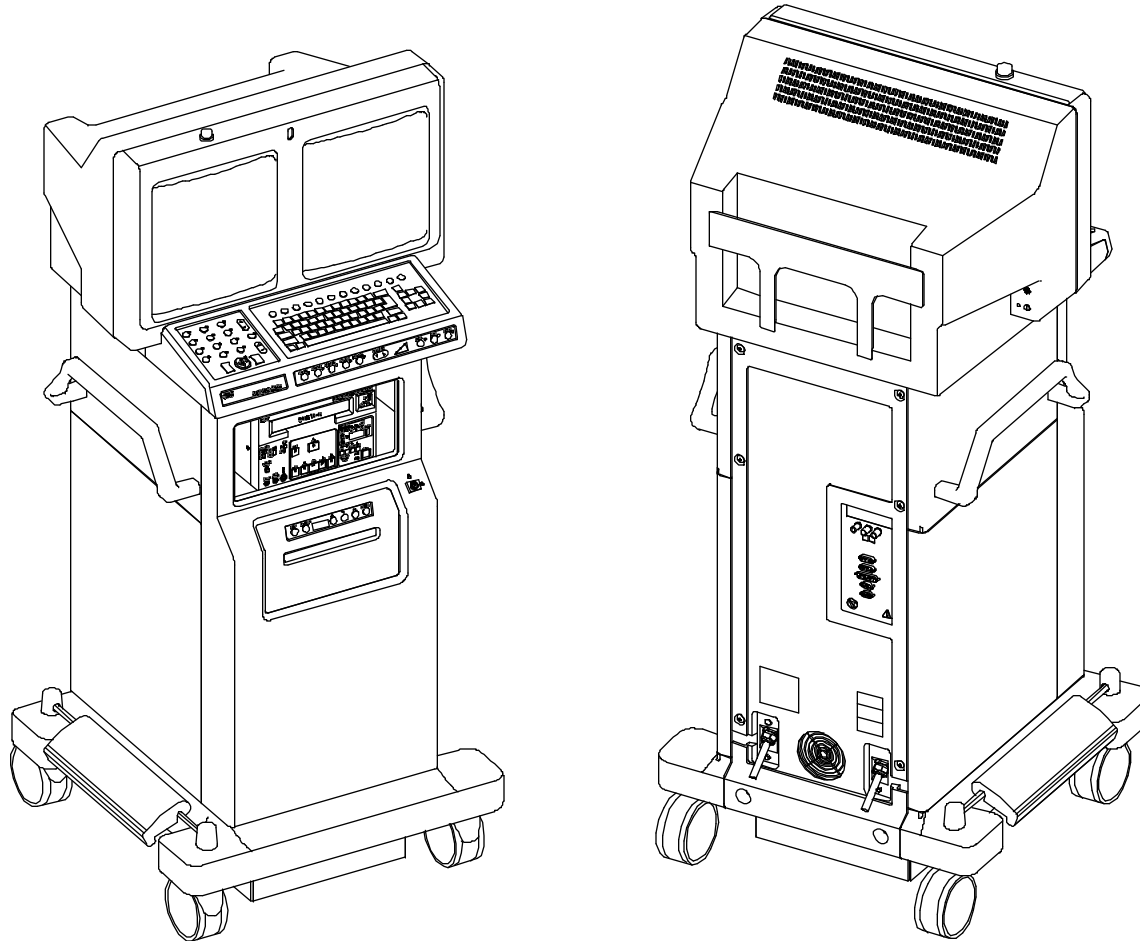

SERIES9600 Imaging Workstation

FIG. 2-1 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-875986-01	COVER, SIDE, LOWER, LEFT	1
2	00-876466-01	HANDLE, MONITOR CART (NOT EXPLODED)	2
3	00-875981-01	COVER, SIDE, UPPER LEFT (NOT EXPLODED)	1
4	00-876746-01	KIT, VTR, SONY 9500 MD4, 60HZ W/CBL	1
5	00-875986-02	COVER, SIDE, LOWER, RIGHT	1
6	00-876285-01	BRACKET, MOUNTING, LENZAR CAMERA	1
7	00-877592-01	BRACKET, MOUNTING, LENZAR CAMERA, LOW (NOT SHOWN)	1
8	00-876544-01	CABLE ASSEMBLY, KEY SWITCH TO P15	1
9	00-875987-02	COVER, FRONT PANEL, LENZAR	1
10	00-900640-01	PANEL ASSEMBLY, CONTROL, LENZAR, ENGLISH, 9600	1
10	00-900640-02	PANEL ASSEMBLY, CONTROL, LENZAR, GERMAN, 9600	1
10	00-900640-03	PANEL ASSEMBLY, CONTROL, LENZAR, FRENCH, 9600	1
10	00-900640-04	PANEL ASSEMBLY, CONTROL, LENZAR, SPANISH, 9600	1
10	00-900640-05	PANEL ASSEMBLY, CONTROL, LENZAR, ITALIAN, 9600	1
11	00-874026-06	ASSEMBLY, IR REMOTE CONTROL VAS, 4 GEN	1
12	00-877611-01	ASSEMBLY, IR REMOTE CONTROL, ENGLISH	1
12	00-877611-03	ASSEMBLY, IR REMOTE CONTROL, GERMAN	1
12	00-877611-04	ASSEMBLY, IR REMOTE CONTROL, FRENCH	1
12	00-877611-05	ASSEMBLY, IR REMOTE CONTROL, SPANISH	1
12	00-877611-07	ASSEMBLY, IR REMOTE CONTROL, ITALIAN	1

FIG. 2-1 PARTS LIST (CONT.)

ITEM	PART NUMBER	DESCRIPTION	QTY
13	00-878031-01	ASSEMBLY, IR REMOTE CONTROL, 9600, ENGLISH	1
13	00-878031-02	ASSEMBLY, IR REMOTE CONTROL, 9600, GERMAN	1
13	00-878031-03	ASSEMBLY, IR REMOTE CONTROL, 9600, FRENCH	1
13	00-878031-04	ASSEMBLY, IR REMOTE CONTROL, 9600, SPANISH	1
13	00-878031-05	ASSEMBLY, IR REMOTE CONTROL, 9600, ITALIAN	1
14	00-876161-01	KIT, LENZAR, 60HZ, ENGLISH, 9600	1
15	00-900666-01	CAMERA, LENZAR, 60HZ (NOT EXPLODED)	1
16	00-875981-02	COVER, SIDE, UPPER, RIGHT (NOT EXPLODED/SHOWN)	1
17	00-876747-01	KIT, VTR, SONY 9510 MDP4, 50HZ W/CBL (NOT SHOWN)	1
18	00-900585-01	RECORDER, SONY VIDEO TAPE 1/2" 60HZ (NOT SHOWN)	1
19	00-900585-02	RECORDER, SONY VIDEO TAPE 1/2" 50HZ (NOT SHOWN)	1
20	00-900267-01	TAPE, VIDEO, 1/2" (NOT SHOWN)	AR
21	00-876748-01	ASSEMBLY, OPTION, THERMAL IMAGER (NOT SHOWN)	1
22	00-900394-01	PRINTER, THERMAL IMAGING (NOT SHOWN)	1
23	00-876178-01	ASSEMBLY, CRATE, WORKSTATION, 9600/2600 (NOT SHOWN)	1

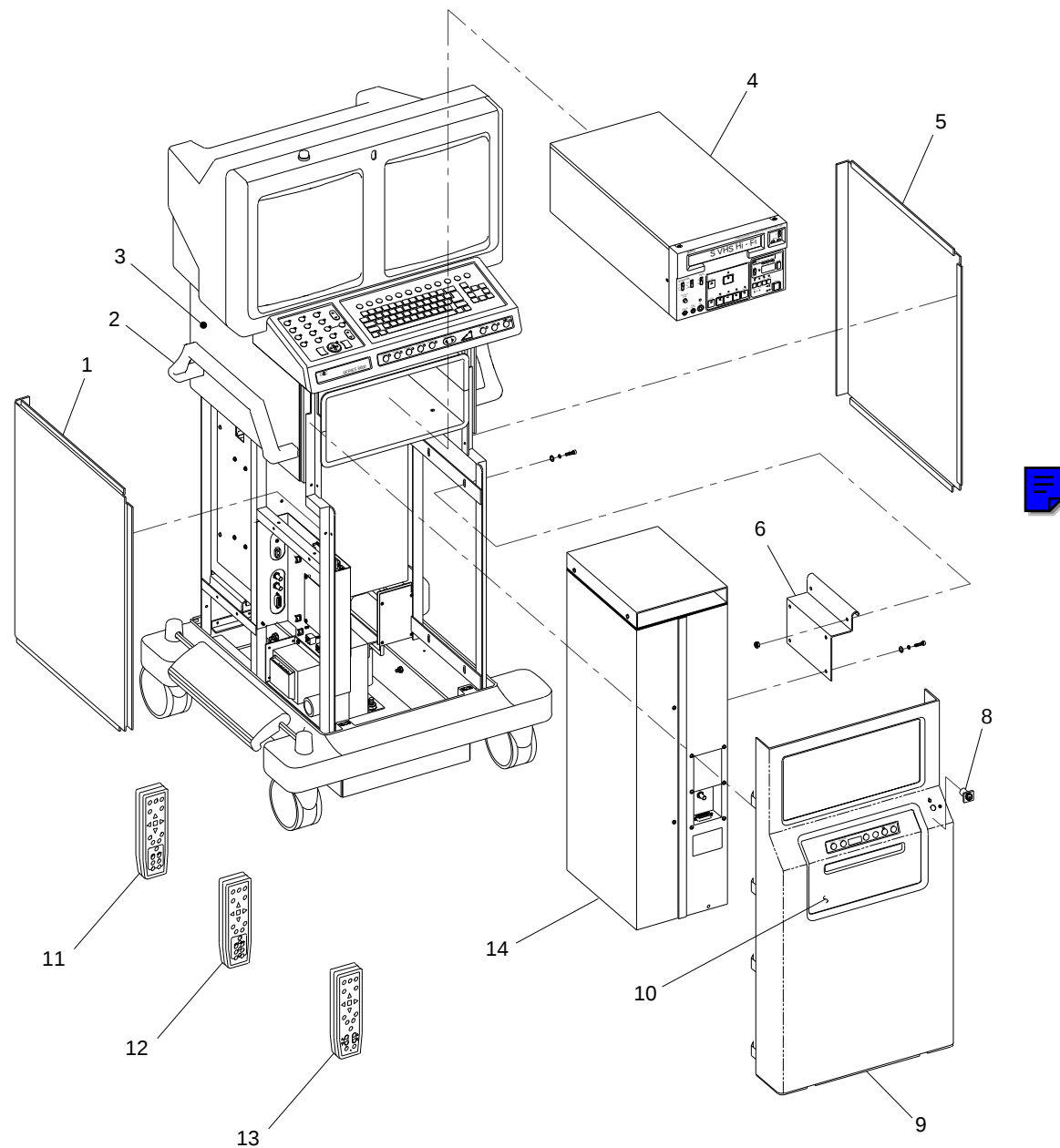


Fig. 2-1 ESP (Expanded Surgical Package) Exploded Front View

FIG. 2-2 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-875986-01	COVER, SIDE, LOWER, LEFT	1
2	00-876466-01	HANDLE, MONITOR CART (NOT EXPLODED)	2
3	00-875981-01	COVER, SIDE, UPPER, LEFT (NOT EXPLODED)	1
4	00-875986-02	COVER, SIDE, LOWER, RIGHT	1
5	00-876544-01	CABLE ASSEMBLY, KEY SWITCH TO P15	1
6	00-875987-01	COVER, FRONT PANEL	1
7	00-875992-01	COVER, SHELF, LOWER (NOT EXPLODED)	1
8	00-875991-01	COVER, SHELF, UPPER (NOT EXPLODED)	1
9	00-875981-02	COVER, SIDE, UPPER, RIGHT (NOT EXPLODED/SHOWN)	1

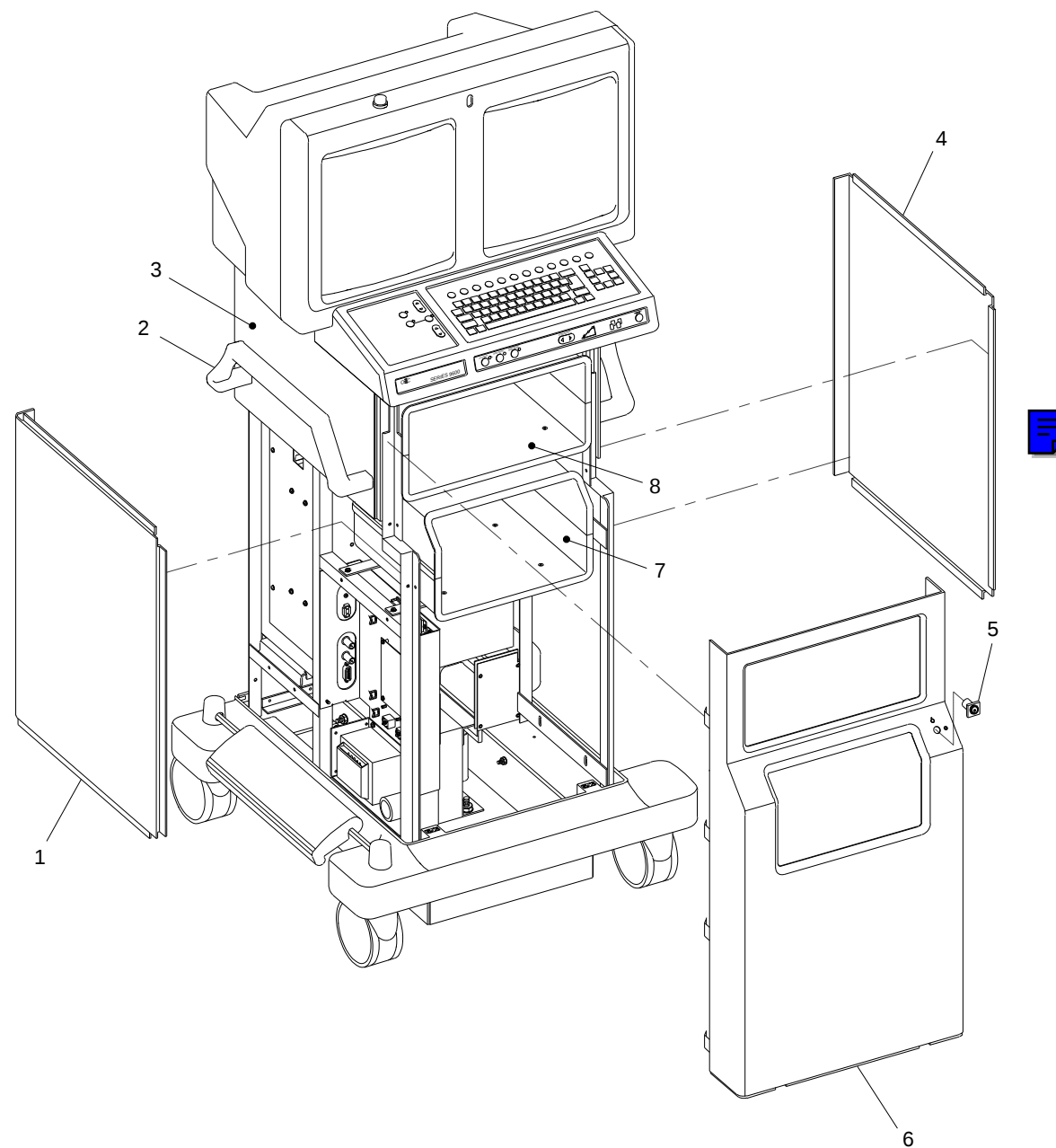


Fig. 2-2 SP (Surgical Platform) Model Exploded Front View

FIG. 2-3 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-876786-01	ASSEMBLY, PCB, SURGE SUPPRESSER	1
2	00-900712-01	TRANSFORMER, ISOLATION WORKSTATION	1
3	66-235026-00	RECTIFIER, SILICON BRIDGE, 100PRV, 30A	2
4	78-590390-00	POWER SUPPLY, MOD +24 VDC	1
5	55-700452-00	CIRCUIT, BREAKER, 10A DPST 125, 250 VAC, PANEL	1
6	00-877037-01	BRACKET, MOUNTING, CIRCUIT BREAKER	1
7	00-876549-01	ASSEMBLY, PCB, POWER CONTROL	1
8	56-996209-01	TERMINAL BLOCK, PANEL 4 POS, 26-12 AWG, 20A	1
9	56-996214-00	JUMPER, TERMINAL BLOCK	1
10	56-996215-00	TERMINAL BLOCK, END PLATE, 26-12 AWG	1

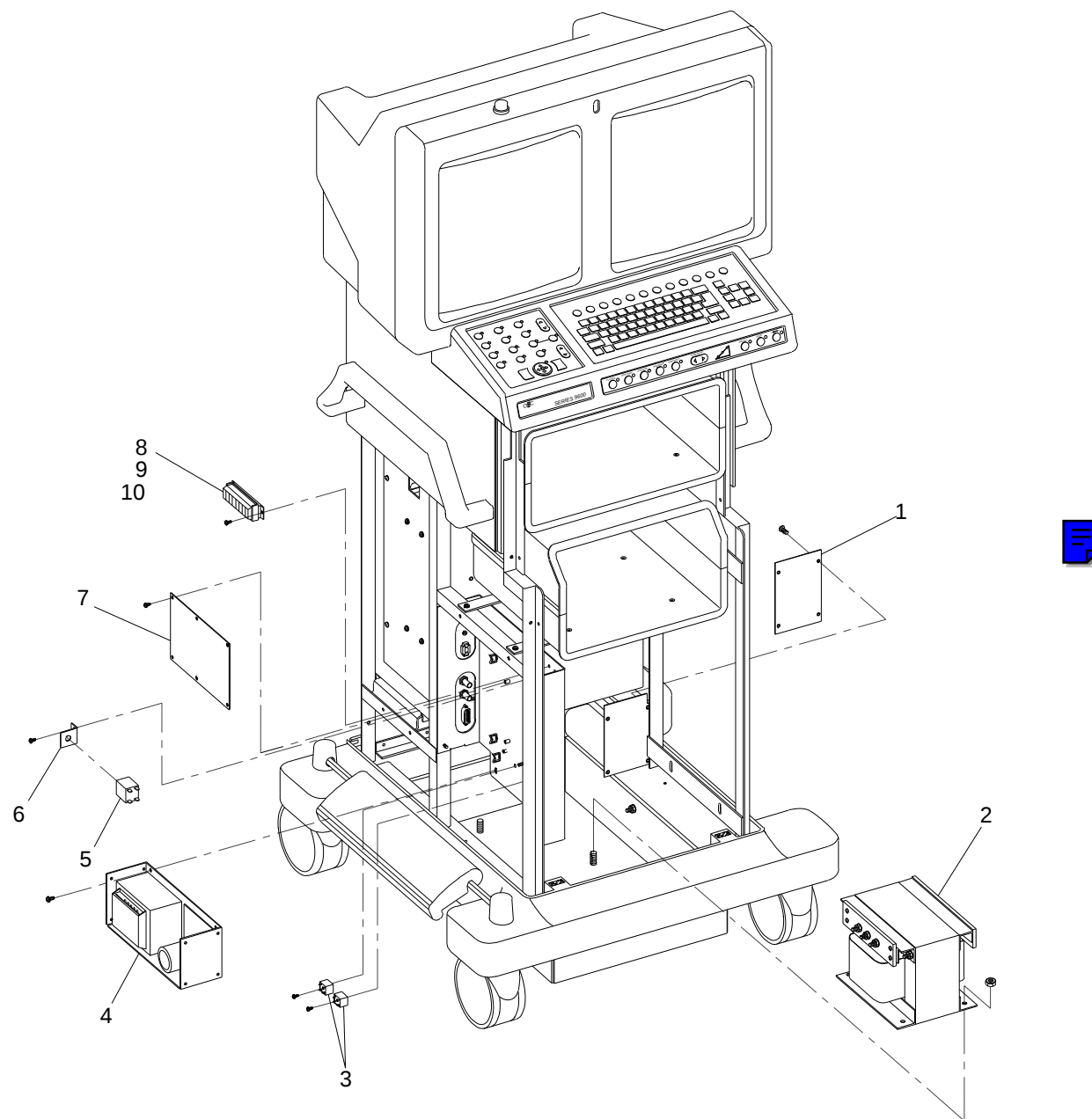


Fig. 2-3 Power Control Panel, Transformer, PCB Assy Surge Suppressor Exploded View

FIG. 2-4 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	55-700452-00	CIRCUIT BREAKER, 10A, DPST 125, 250 VAC, PANEL	2
2	00-876469-01	BRACKET, INTERCONNECT	1
3	00-876183-01	CABLE ASSEMBLY, INTERCONNECT, 20', 9600	1
4	00-876183-02	CABLE ASSEMBLY, INTERCONNECT, 30', 9600	1
5	20-839918-00	FAN, SXL, 115 VAC, 115 CFM, 3200 RPM	1
6	20-839926-00	GUARD, FINGER, FAN	1
7	00-875990-01	COVER, REAR PANEL	1
8	00-876468-01	BRACKET, POWER CORD	1
9	00-876180-01	ASSEMBLY, POWER CORD, 100-127V, 20 FT, 9600	2
10	00-876181-01	ASSEMBLY, POWER CORD, 220-250V, 20 FT, 9600	1
10	00-876181-02	ASSEMBLY, POWER CORD, 220-250V, 30 FT, 9600	1
11	00-876182-01	ASSEMBLY, POWER CORD, 220V, 60 HZ, 20', 9600	1
12	67-430001-00	IND, LAMP, NE, 110V, GREEN	1
13	56-299702-00	TERMINAL STRIP, PANEL, 3 POS, 380V	1
14	00-875981-02	COVER, SIDE, UPPER RIGHT (NOT EXPLODED)	1
15	27-590972-00	FILTER, AIR, FRAMELESS, 6 X 6 X .188	1

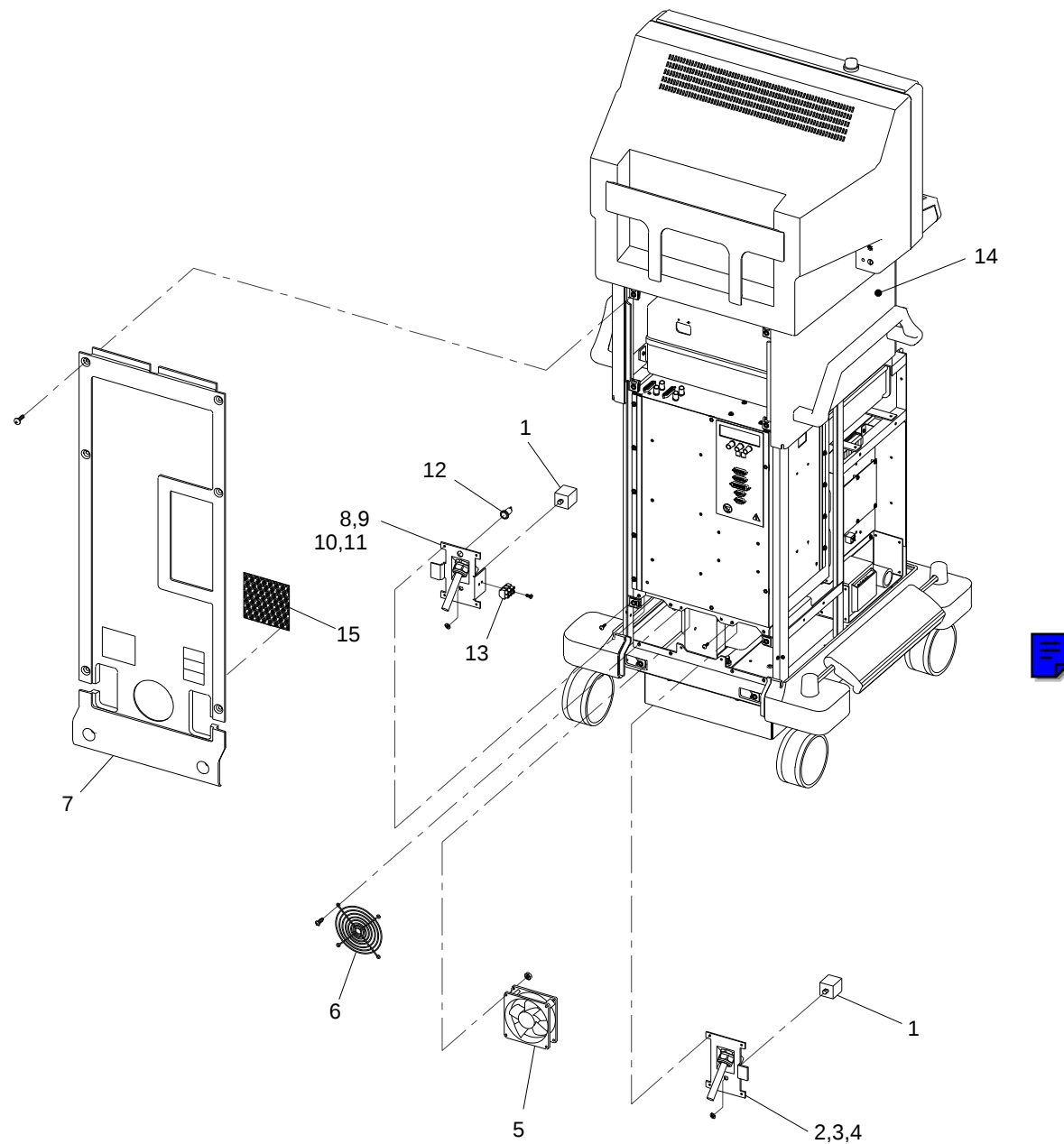


Fig. 2-4 Rear Panel, Fan, Power Cord Assy, Interconnect Cable Assy Exploded View

FIG. 2-5 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-900657-03	MONITOR, FAST SCAN, 16"	2
2	00-875983-01	HOUSING, MONITOR	1
3	00-877124-01	BRACE, BEZEL	1
4	00-876470-01	BRACKET, IR RECEIVER	1
5	00-874222-04	ASSEMBLY, PCB, IR RECEIVER 9600	1
6	00-900751-01	LAMP, X-RAY ON (SCD)	1
7	00-875982-01	BEZEL, MONITOR (NOT EXPLODED)	1
8	00-876541-01	CABLE ASSEMBLY, CONT/BRT POTS	1
9	00-876541-02	CABLE ASSEMBLY, CONT/BRT POTS	1
10	00-876471-02	BRACKET, B/C MOUNTING	1
11	00-877009-03	ASSEMBLY, OUTLET STRIP, INTL AC (120V AC 50/60 Hz, 240V AC 50/60Hz)	1
11	00-877009-04	ASSEMBLY, OUTLET STRIP, INTL AC (115V AC 50/60 Hz, 230V AC 50/60Hz)	1
12	00-900792-01	PCA, DEFLECTION, MONITOR (SCD)	2
13	00-876353-01	ASSEMBLY, PCB, HI SCAN BRGTS/CTRS CONT	1
14	00-874661-01	LENS, FRONT, COLORED (NOT EXPLODED/SHOWN)	1
15	00-874661-02	LENS, FRONT, CLEAR (NOT EXPLODED/SHOWN)	1

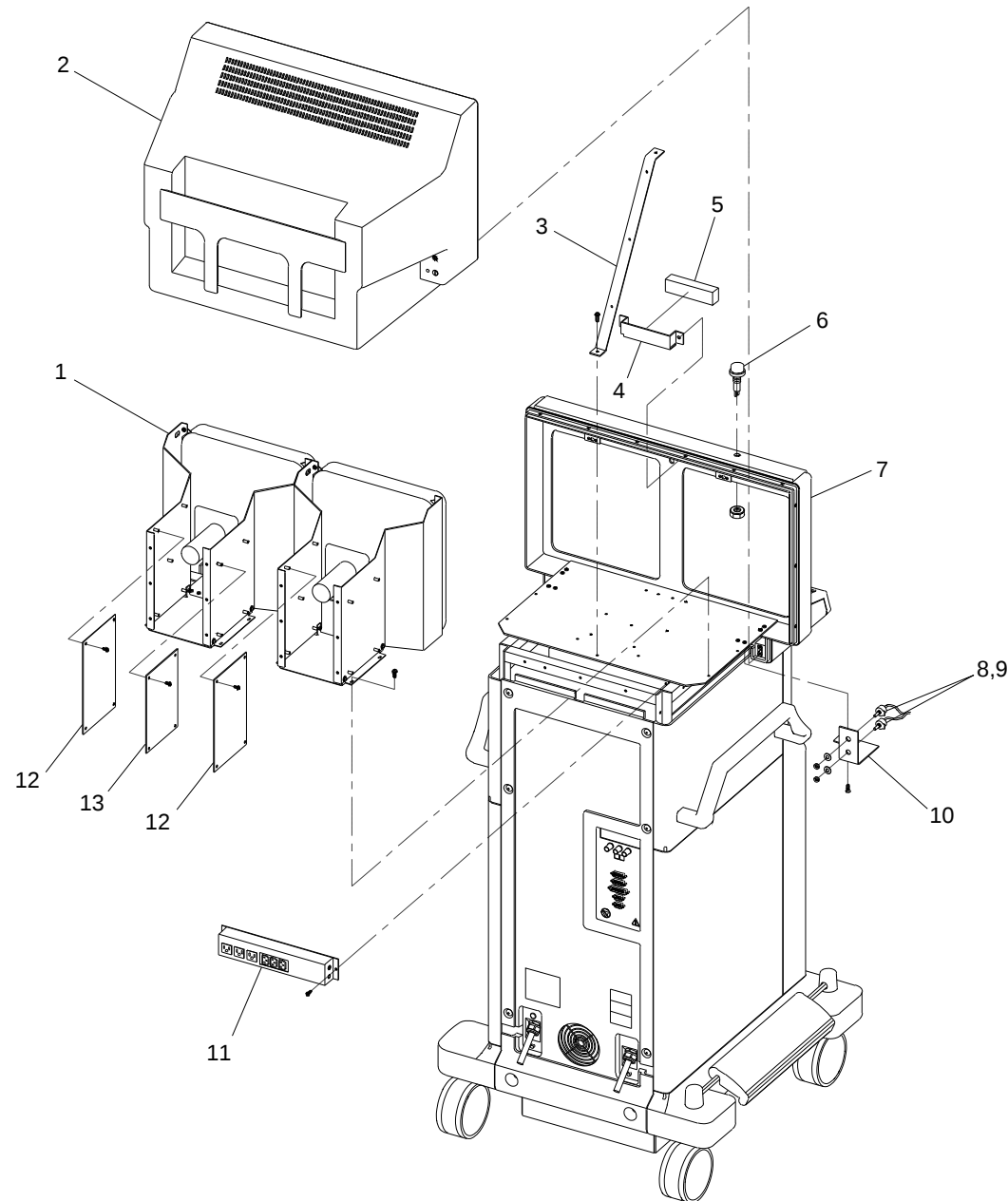


Fig. 2-5 Monitor Rear Exploded View

FIG. 2-6 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-875984-01	COVER, BASE	1
2	00-900696-01	CASTER, LOCK, 5" (SCD)	2
3	00-876702-01	PEDAL, BRAKE	1
4	00-876624-01	BAR, BRAKE	2
5	00-900695-01	CASTER, SWIVEL, 5" (SCD)	2
6	21-709008-00	SPRING, COMPRESSION, .420 OD, .750 LG, .042 WD, SST	2


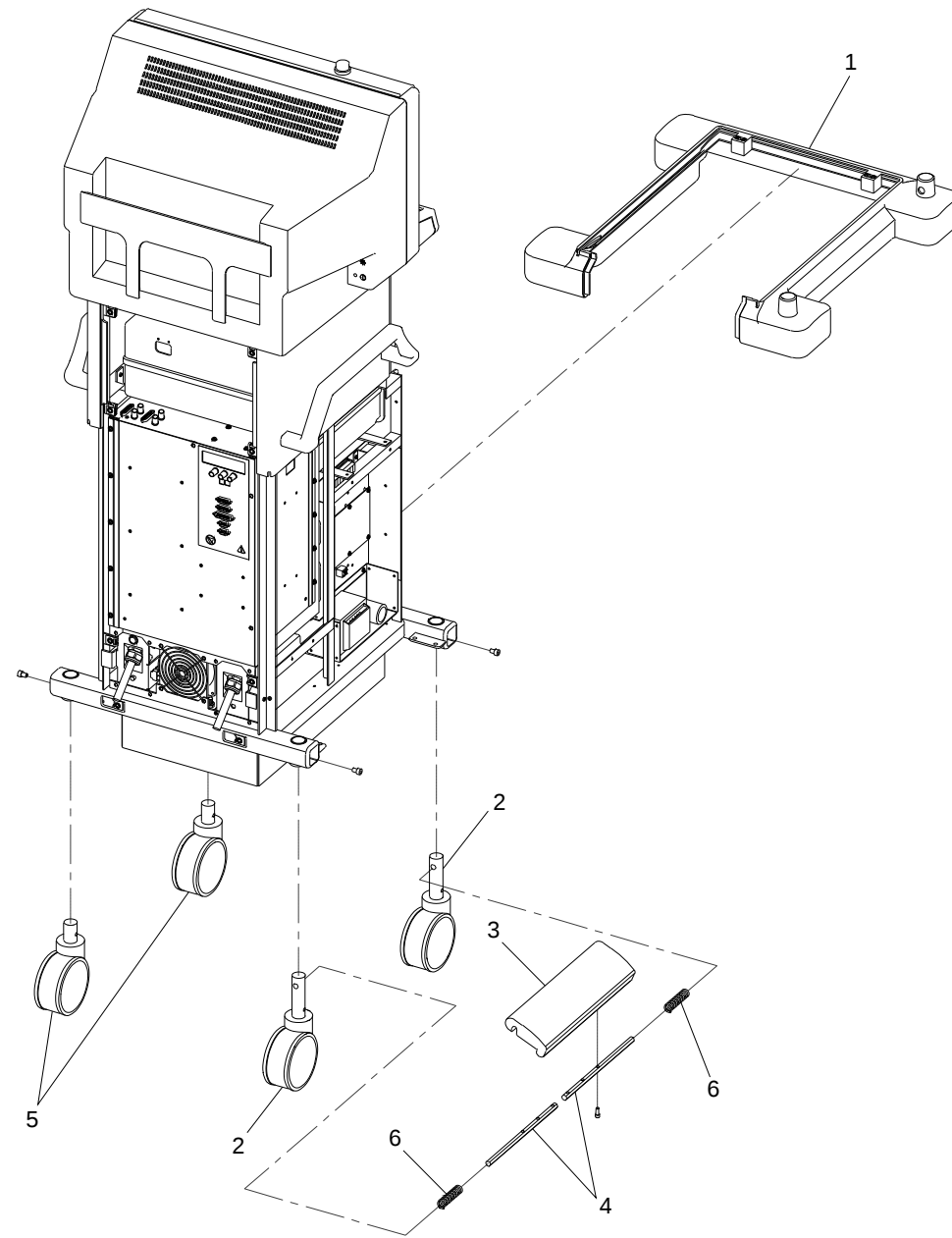


Fig. 2-6 Brake, Caster Exploded View

FIG. 2-7 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-876504-01	ASSEMBLY, PCB, AUX INTERFACE, 9600	1
2	00-872239-01	ASSEMBLY, PCB, VIDEO SWITCHING 4 GEN	1
3	67-700000-00	METER, ELEC, DC, HR/TNTS, 6 DIGIT	1
4	00-900689-01	POWER, SUPPLY, QUAD OUTPUT (SCD)	1
5	00-876841-01	ASSEMBLY, PCB, DC POWER DIST, 9600	1
6	20-839956-00	FAN, AXL, 115 VAC, 75 CFM, 2200 RPM	2
7	20-839926-00	GUARD, FINGER, FAN	2
8	00-876995-01	MOUNT, HARD DRIVE MON CART	1
9	00-875954-01	ASSEMBLY, PCB, IMAGE PROCESSOR, SMT	1
10	00-876399-02	ASSEMBLY, PCB, FAST SCAN CONVERTER	1
11	00-872127-04	ASSEMBLY, PCB, COMMUNICATIONS 4 GEN	1
12	00-900658-01	PCA, MOTHERBOARD, 386 (SCD)	1
13	00-900669-01	DRIVE FLOPPY DISK, 3.5", 1.44 MB	1
14	00-877024-01	DISK, FORMATTED, DISK DRIVE, 42 MB	1
15	00-876712-01	MOUNT, MOTHERBOARD, 9600 (NOT SHOWN)	1
16	00-876284-01	BRACKET, MOUNTING, FLOPPY DRIVE (NOT SHOWN)	1

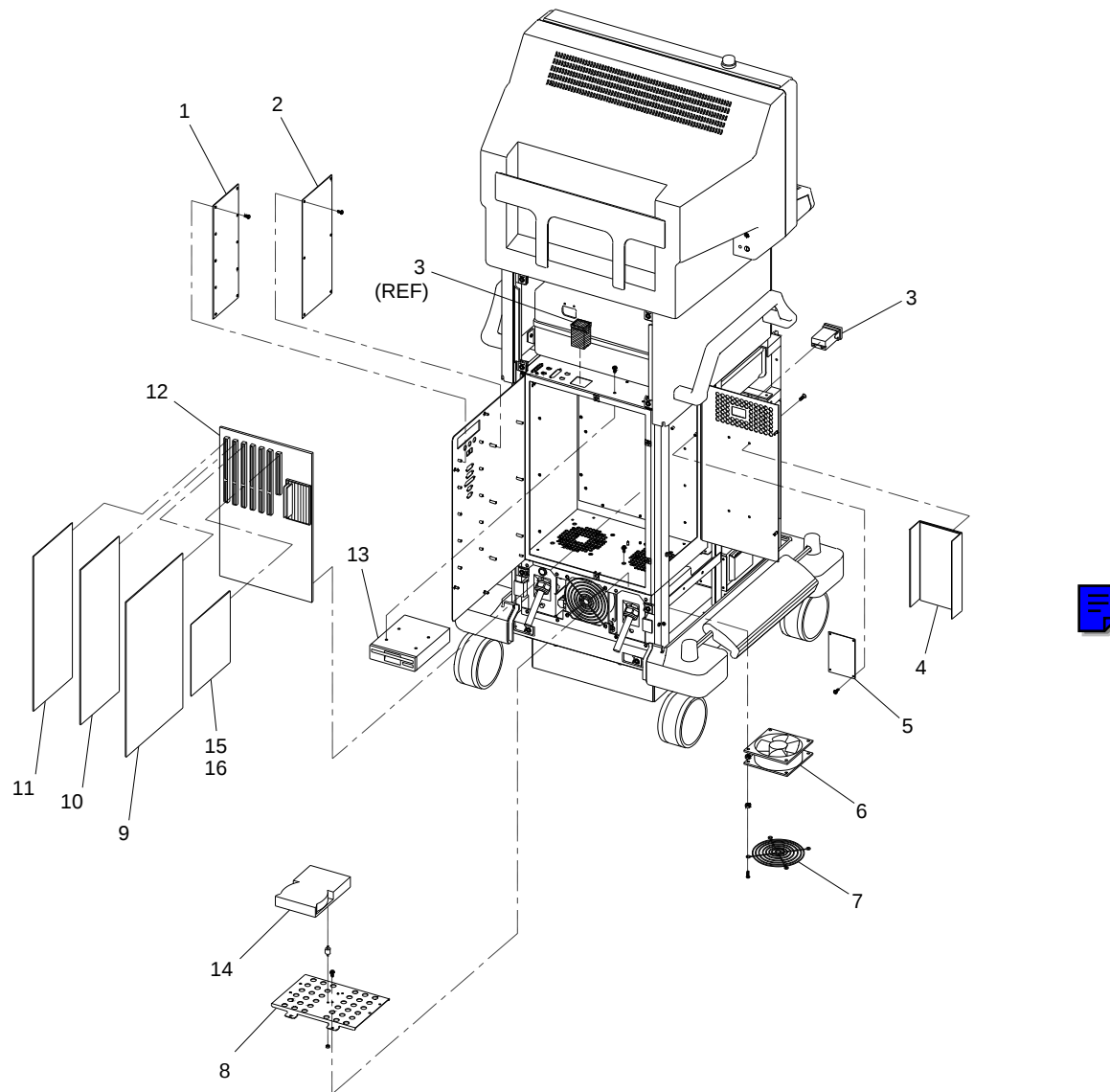


Note: Location of Item 3

Item 3 can be located in either position as shown. Refer to each workstation for exact location.

Fig. 2-7 Electronic Rack, Generic Exploded View

FIG. 2-8 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-876504-01	ASSEMBLY, PCB, AUX INTERFACE, 9600	1
2	00-872239-01	ASSEMBLY, PCB, VIDEO SWITCHING 4 GEN	1
3	00-876536-01	CABLE ASSEMBLY, POWER ETI	1
4	00-900689-01	POWER, SUPPLY, QUAD OUTPUT (SCD)	1
5	00-876841-01	ASSEMBLY, PCB, DC POWER DIST, 9600	1
6	20-839956-00	FAN, AXL, 115 VAC, 75 CFM, 2200 RPM	2
7	20-839926-00	GUARD, FINGER, FAN	2
8	00-876995-01	MOUNT, HARD DRIVE MON CART	1
9	00-875954-01	ASSEMBLY, PCB, IMAGE PROCESSOR, SMT	1
10	00-876399-02	ASSEMBLY, PCB, FAST SCAN CONVERTER	1
11	00-872127-04	ASSEMBLY, PCB, COMMUNICATIONS 4 GEN	1
12	00-900658-01	PCA, MOTHERBOARD, 386 (SCD)	1
13	00-900669-01	DRIVE FLOPPY DISK, 3.5", 1.44 MB	1
14	00-877025-01	DISK, FORMATTED, DISK DRIVE, SCSI	1
15	0-878229-01	DISK CONT, TESTED ALTA	1
16	00-900659-01	DISK CONTROLLER, SCSI (SCD)	1
17	00-876712-01	MOUNT, MOTHERBOARD, 9600 (NOT SHOWN)	1
18	00-876284-01	BRACKET, MOUNTING, FLOPPY DRIVE (NOT SHOWN)	1

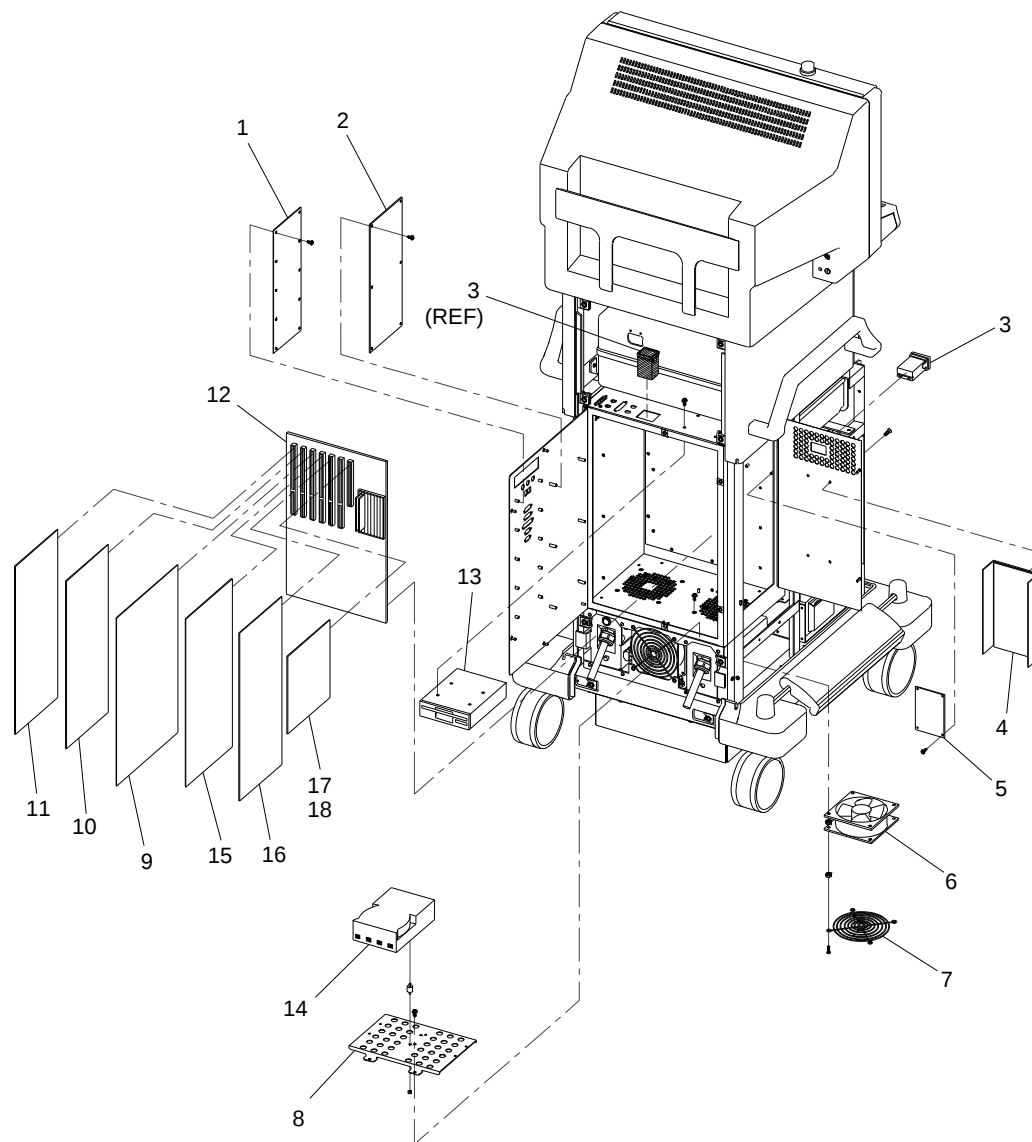


Note: Location of Item 3

Item 3 can be located in either position as shown. Refer to each workstation for exact location.

Fig. 2-8 Electronic Rack, ESP with 4 FPS Storage Exploded View

FIG. 2-9 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-876504-01	ASSEMBLY, PCB, AUX INTERFACE, 9600	1
2	00-872239-01	ASSEMBLY, PCB, VIDEO SWITCHING 4 GEN	1
3	00-876536-01	CABLE ASSEMBLY, POWER ETI	1
4	00-900689-01	POWER, SUPPLY, QUAD OUTPUT (SCD)	1
5	00-876841-01	ASSEMBLY, PCB, BC POWER DIST, 9600	1
6	20-839956-00	FAN, AXL, 115 VAC, 75 CFM, 2200 RPM	2
7	20-839926-00	GUARD, FINGER, FAN	2
8	00-876995-01	MOUNT, HARD DRIVE MON CART	1
9	00-875954-01	ASSEMBLY, PCB, IMAGE PROCESSOR, SMT	1
10	00-876399-02	ASSEMBLY, PCB, FAST SCAN CONVERTER	1
11	00-872127-04	ASSEMBLY, PCB, COMMUNICATIONS 4 GEN	1
12	00-900658-01	PCA, MOTHERBOARD, 386 (SCD)	1
13	00-900669-01	DRIVE FLOPPY DISK, 3.5", 1.44 MB	1
14	00-878230-01	ASSEMBLY, DISK DRIVE, 30 FPS, FORMATTED	2
15	0-878229-01	DISK CONT, TESTED ALTA	2
16	00-900659-01	DISK CONTROLLER, SCSI (SCD)	1
17	00-876712-01	MOUNT, MOTHERBOARD, 9600 (NOT SHOWN)	1
18	00-876284-01	BRACKET, MOUNTING, FLOPPY DRIVE (NOT SHOWN)	1

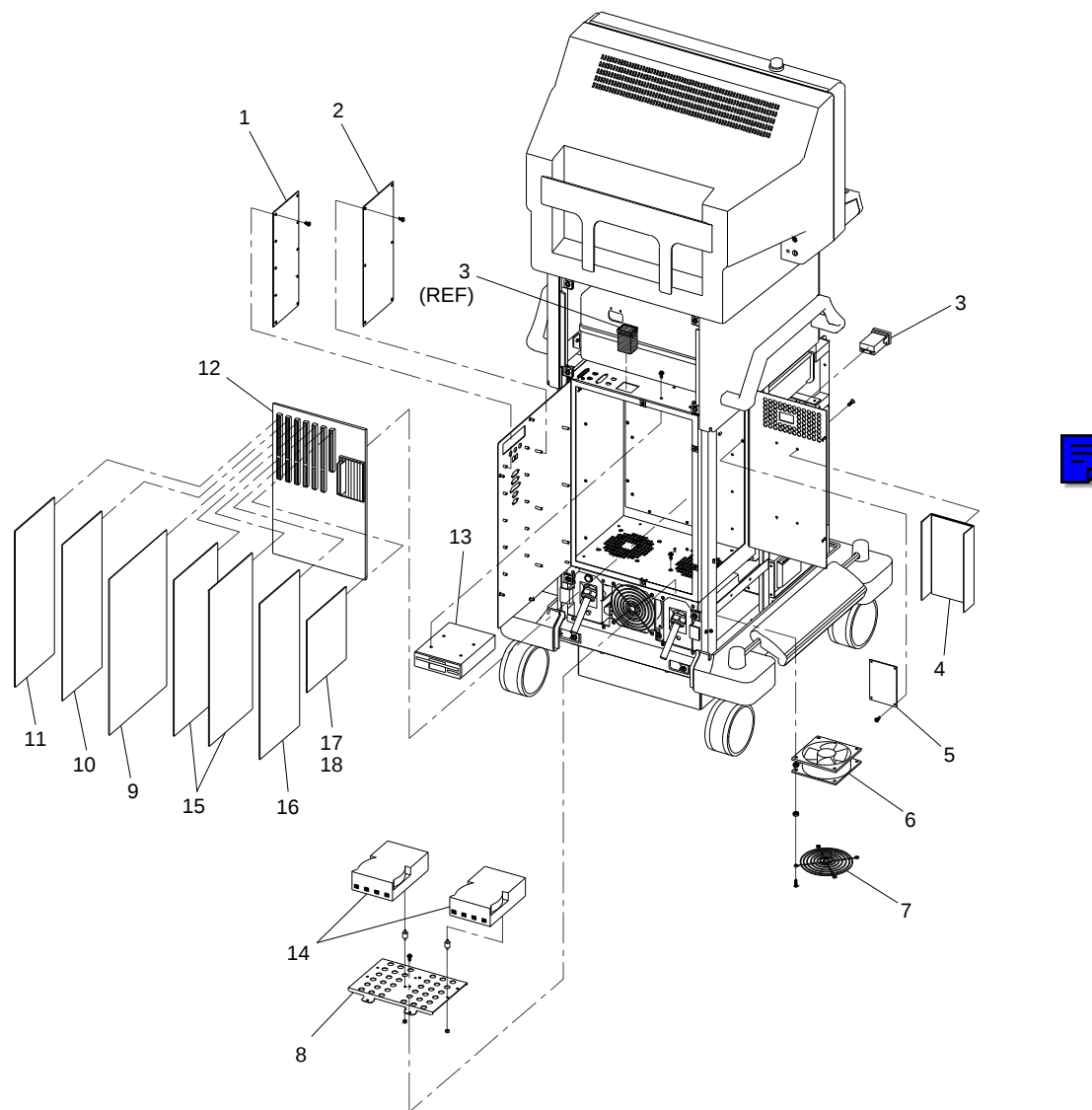

Note: Location of Item 3

Item 3 can be located in either position as shown. Refer to each workstation for exact location.

Fig. 2-9 Electronic Rack, Vascular with 30 FPS Storage Exploded View

FIG. 2-10 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-875988-01	COVER, KEYBOARD, TOP	1
2	00-900769-01	PANEL ASSEMBLY, CONTROL, CART SP, 9600 (ENGLISH)	1
2	00-900769-02	PANEL ASSEMBLY, CONTROL, CART SP, 9600 (GERMAN)	1
2	00-900769-03	PANEL ASSEMBLY, CONTROL, CART SP, 9600 (FRENCH)	1
2	00-900769-04	PANEL ASSEMBLY, CONTROL, CART SP, 9600 (SPANISH)	1
2	00-900769-05	PANEL ASSEMBLY, CONTROL, CART SP, 9600 (ITALIAN)	1
3	00-900641-01	PANEL ASSEMBLY, QWERTY KEYBOARD ENGLISH 96	1
4	00-900772-01	PANEL ASSEMBLY, CONTROL, FRONT SP, 9600 (ENGLISH)	1
4	00-900772-02	PANEL ASSEMBLY, CONTROL, FRONT SP, 9600 (GERMAN)	1
4	00-900772-03	PANEL ASSEMBLY, CONTROL, FRONT SP, 9600 (FRENCH)	1
4	00-900772-04	PANEL ASSEMBLY, CONTROL, FRONT SP, 9600 (SPANISH)	1
4	00-900772-05	PANEL ASSEMBLY, CONTROL, FRONT SP, 9600 (ITALIAN)	1
5	00-876767-01	ASSEMBLY, ALARM, AUDIO	1
6	00-877670-01	COVER, BOX, RFI, 9600 WS	
7	00-876613-01	ASSEMBLY, PCB, CONT PANEL PROC, 9600 MC	1
8	00-877671-01	BASE, BOX, RFI., 9600 WS	1
9	00-876224-01	NAMEPLATE, PRODUCT, 9600 (NOT EXPLODED)	1
10	00-876533-01	CABLE ASSEMBLY, QW-CPP (NOT SHOWN)	1
11	00-876693-01	CABLE ASSEMBLY, FRONT PANEL-CPP (NOT SHOWN)	1
12	00-876538-01	CABLE ASSEMBLY, CONTROL PANEL TO CPP (NOT SHOWN)	1

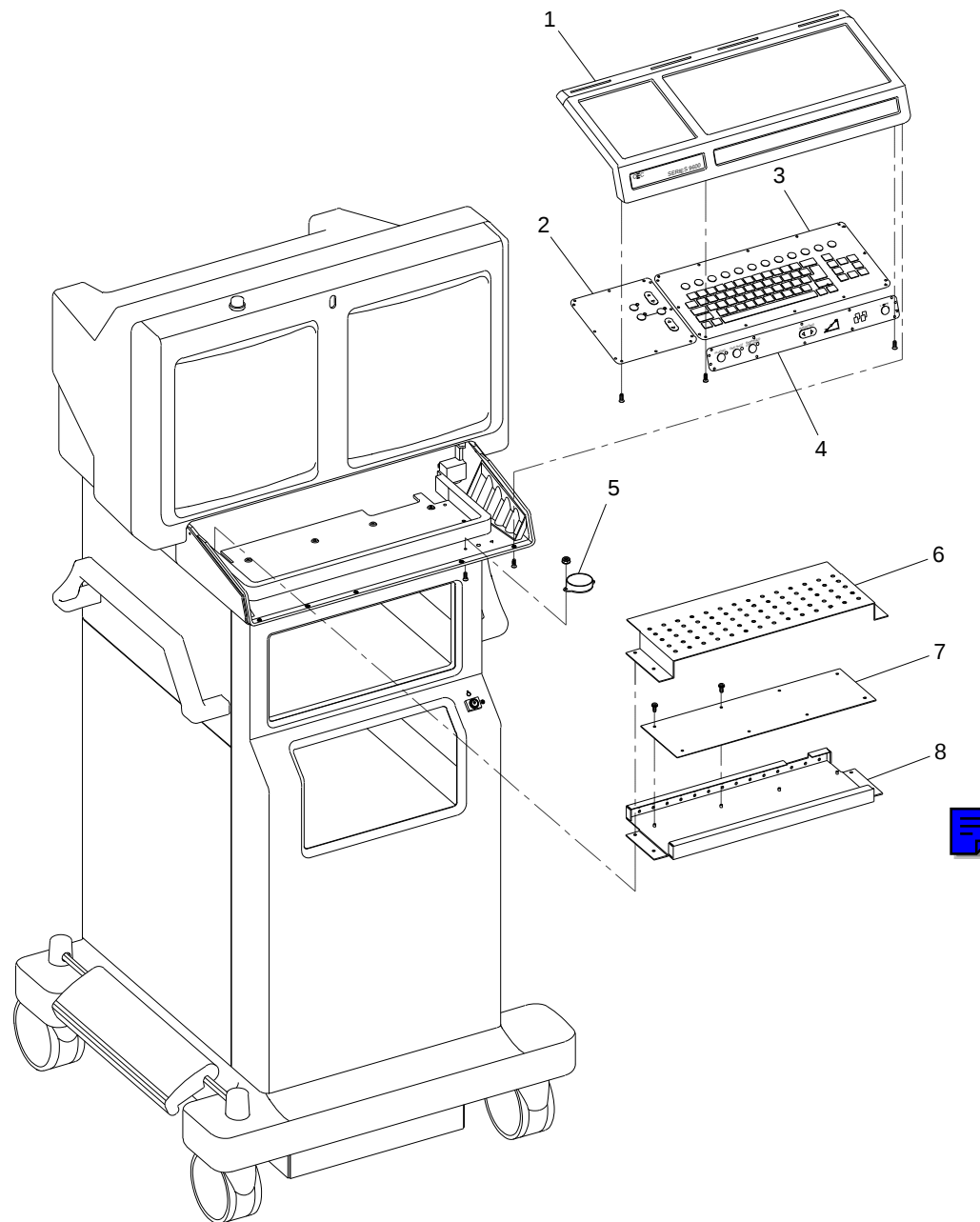


Fig. 2-10 Keyboard, SP (Surgical Platform) Model Exploded View

FIG. 2-11 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-875988-01	COVER, KEYBOARD, TOP	1
2	00-900768-01	PANEL ASSEMBLY, CONTROL, CART ESP, 9600 (ENGLISH)	1
2	00-900768-02	PANEL ASSEMBLY, CONTROL, CART ESP, 9600 (GERMAN)	1
2	00-900768-03	PANEL ASSEMBLY, CONTROL, CART ESP, 9600 (FRENCH)	1
2	00-900768-04	PANEL ASSEMBLY, CONTROL, CART ESP, 9600 (SPANISH)	1
2	00-900768-05	PANEL ASSEMBLY, CONTROL, CART ESP, 9600 (ITALIAN)	1
3	00-900641-01	PANEL ASSEMBLY, QWERTY KEYBOARD ENGLISH 96	1
4	00-900771-01	PANEL ASSEMBLY, FRONT, ESP, ESP/4F, 9600 (ENGLISH)	1
4	00-900771-02	PANEL ASSEMBLY, FRONT, ESP, ESP/ACQ, 9600 (GERMAN)	1
4	00-900771-03	PANEL ASSEMBLY, FRONT, ESP, ESP/ACQ, 9600 (FRENCH)	1
4	00-900771-04	PANEL ASSEMBLY, FRONT, ESP, ESP/ACQ, 9600 (SPANISH)	1
4	00-900771-05	PANEL ASSEMBLY, FRONT, ESP, ESP/ACQ, 9600 (ITALIAN)	1
5	00-876767-01	ASSEMBLY, ALARM, AUDIO	1
6	00-877670-01	COVER, BOX, RFI, 9600 WS	
7	00-876613-01	ASSEMBLY, PCB, CONT PANEL PROC, 9600 MC	1
8	00-877671-01	BASE, BOX, RFI., 9600 WS	1
9	00-876224-01	NAMEPLATE, PRODUCT, 9600 (NOT EXPLODED)	1
10	00-876533-01	CABLE ASSEMBLY, QW-CPP (NOT SHOWN)	1
11	00-876693-01	CABLE ASSEMBLY, FRONT PANEL-CPP (NOT SHOWN)	1
12	00-876538-01	CABLE ASSEMBLY, CONTROL PANEL TO CPP (NOT SHOWN)	1

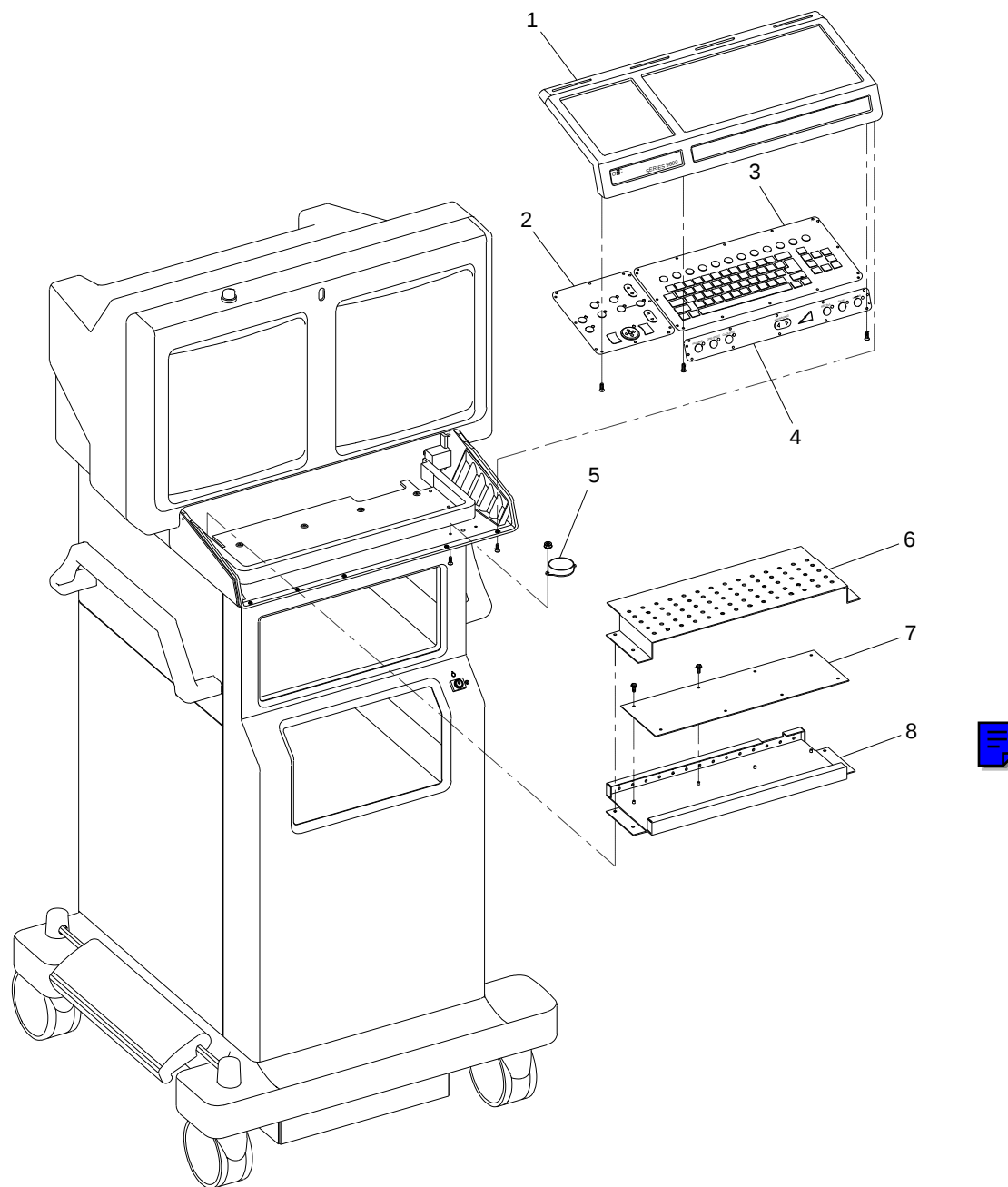


Fig. 2-11 Keyboard, ESP/Cholangiography Model Exploded View

FIG. 2-12 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-875988-01	COVER, KEYBOARD, TOP	1
2	00-900767-01	PANEL ASSEMBLY, CONTROL, CART ESP/4F, 9600 (ENGLISH)	1
2	00-900767-02	PANEL ASSEMBLY, CONTROL, CART ESP/ACQ, 9600 (GERMAN)	1
2	00-900767-03	PANEL ASSEMBLY, CONTROL, CART ESP/ACQ, 9600 (FRENCH)	1
2	00-900767-04	PANEL ASSEMBLY, CONTROL, CART ESP/ACQ, 9600 (SPANISH)	1
2	00-900767-05	PANEL ASSEMBLY, CONTROL, CART ESP/ACQ, 9600 (ITALIAN)	1
3	00-900641-01	PANEL ASSEMBLY, QWERTY KEYBOARD ENGLISH 9600	1
4	00-900771-01	PANEL ASSEMBLY, FRONT, ESP & ESP/4F, 9600 (ENGLISH)	1
4	00-900771-02	PANEL ASSEMBLY, FRONT, ESP & ESP/ACQ, 9600 (GERMAN)	1
4	00-900771-03	PANEL ASSEMBLY, FRONT, ESP & ESP/ACQ, 9600 (FRENCH)	1
4	00-900771-04	PANEL ASSEMBLY, FRONT, ESP & ESP/ACQ, 9600 (SPANISH)	1
4	00-900771-05	PANEL ASSEMBLY, FRONT, ESP & ESP/ACQ, 9600 (ITALIAN)	1
5	00-876767-01	ASSEMBLY, ALARM, AUDIO	1
6	00-877670-01	COVER, BOX, RFI, 9600 WS	
7	00-876613-01	ASSEMBLY, PCB, CONT PANEL PROC, 9600 MC	1
8	00-877671-01	BASE, BOX, RFI., 9600 WS	1
9	00-876224-01	NAMEPLATE, PRODUCT, 9600 (NOT EXPLODED)	1
10	00-876533-01	CABLE ASSEMBLY, QW-CPP (NOT SHOWN)	1
11	00-876693-01	CABLE ASSEMBLY, FRONT PANEL-CPP (NOT SHOWN)	1
12	00-876538-01	CABLE ASSEMBLY, CONTROL PANEL TO CPP (NOT SHOWN)	1

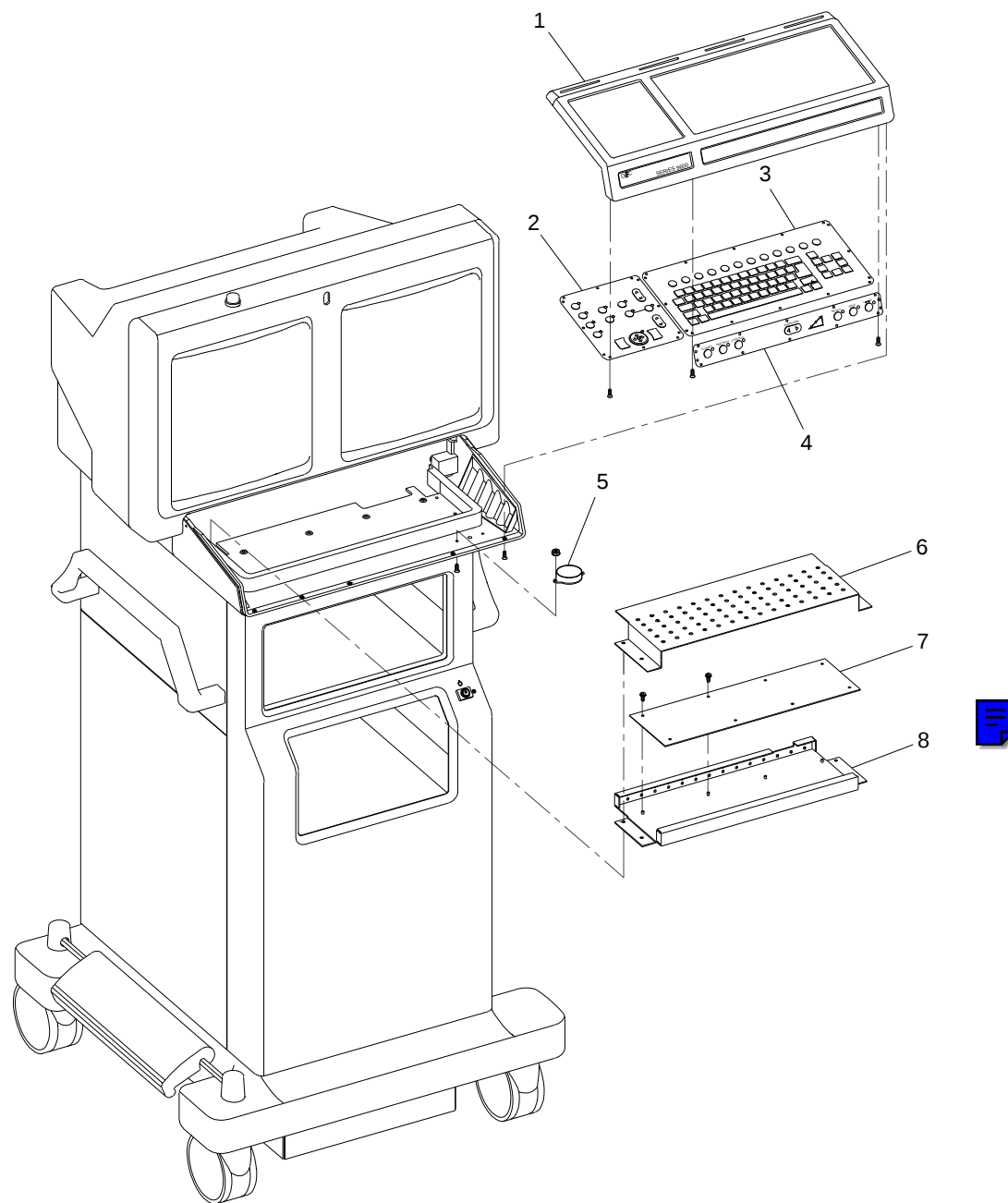


Fig. 2-12 Keyboard, ESP/Acquire Model Exploded View

FIG. 2-13 PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-875988-01	COVER, KEYBOARD, TOP	1
2	00-900766-01	PANEL ASSEMBLY, CONTROL, CART VAS, 9600 (ENGLISH)	1
2	00-900766-02	PANEL ASSEMBLY, CONTROL, CART VAS, 9600 (GERMAN)	1
2	00-900766-03	PANEL ASSEMBLY, CONTROL, CART VAS, 9600 (FRENCH)	1
2	00-900766-04	PANEL ASSEMBLY, CONTROL, CART VAS, 9600 (SPANISH)	1
2	00-900766-05	PANEL ASSEMBLY, CONTROL, CART VAS, 9600 (ITALIAN)	1
3	00-900641-01	PANEL ASSEMBLY, QWERTY KEYBOARD ENGLISH 96	1
4	00-900770-01	PANEL ASSEMBLY, CONTROL, FRONT, VAS, (ENGLISH)	1
4	00-900770-02	PANEL ASSEMBLY, CONTROL, FRONT, VAS, (GERMAN)	1
4	00-900770-03	PANEL ASSEMBLY, CONTROL, FRONT, VAS, (FRENCH)	1
4	00-900770-04	PANEL ASSEMBLY, CONTROL, FRONT, VAS, (SPANISH)	1
4	00-900770-05	PANEL ASSEMBLY, CONTROL, FRONT, VAS, (ITALIAN)	1
5	00-876767-01	ASSEMBLY, ALARM, AUDIO	1
6	00-877670-01	COVER, BOX, RFI, 9600 WS	
7	00-876613-01	ASSEMBLY, PCB, CONT PANEL PROC, 9600 MC	1
8	00-877671-01	BASE, BOX, RFI., 9600 WS	1
9	00-876224-01	NAMEPLATE, PRODUCT, 9600 (NOT EXPLODED)	1
10	00-876533-01	CABLE ASSEMBLY, QW-CPP (NOT SHOWN)	1
11	00-876693-01	CABLE ASSEMBLY, FRONT PANEL-CPP (NOT SHOWN)	1
12	00-876538-01	CABLE ASSEMBLY, CONTROL PANEL TO CPP (NOT SHOWN)	1

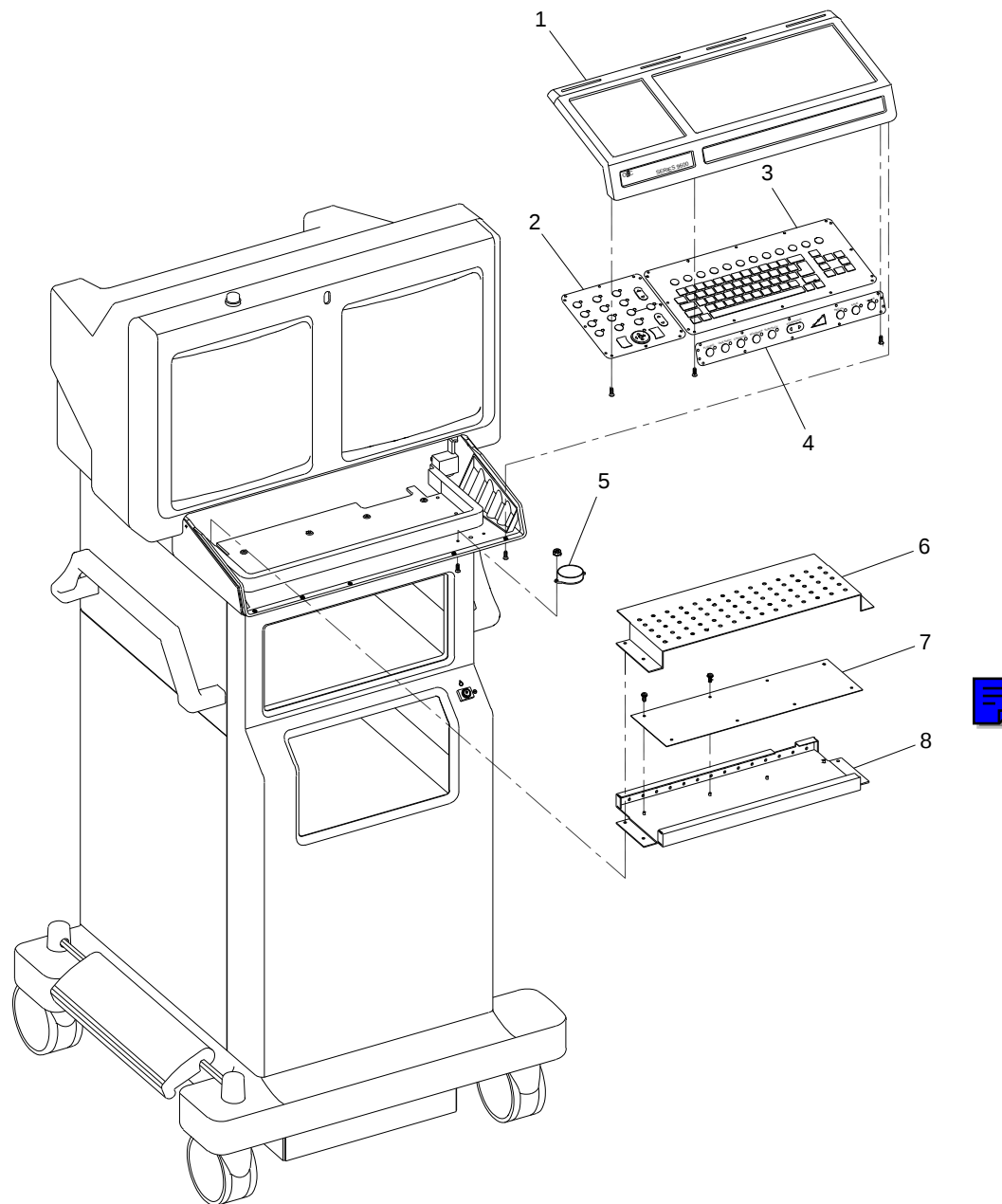


Fig. 2-13 Keyboard, ESP Vascular, Neuro-Vascular Model Exploded View

FIG. 2-14A PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-876538-01	CABLE ASSEMBLY, CONTROL PANEL TO CPP	1
2	00-876533-01	CABLE ASSEMBLY, QW-CPP	1
3	00-876693-01	CABLE ASSEMBLY, FRONT PANEL-CPP	1
4	00-876846-04	CABLE ASSEMBLY, GROUND, CONT PANELS, 9600	1
5	00-876846-05	CABLE ASSEMBLY, GROUND, CONT PANELS, 9600	1
6	00-876846-06	CABLE ASSEMBLY, GROUND, CONT PANELS, 9600	1
7	00-876530-01	CABLE ASSEMBLY, CPP/REM CONT/X-RAY ON	3
8	00-876770-01	CABLE ASSEMBLY, X-RAY ON LAMP	1
9	00-876767-01	ASSEMBLY, ALARM, AUDIO	1
10	00-876541-01	CABLE ASSEMBLY, CONT/BRT POTS	2
11	00-876541-02	CABLE ASSEMBLY, CONT/BRT POTS	2
12	00-876540-01	CABLE ASSEMBLY, CONT/BRT RIGHT	1
13	00-876540-02	CABLE ASSEMBLY, CONT/BRT LEFT	1
14	00-876694-01	CABLE ASSEMBLY, BNC MONITOR RIGHT	1
15	00-876694-02	CABLE ASSEMBLY, BNC MONITOR LEFT	1
16	00-876438-01	HARNESS ASSEMBLY, POWER CONTROL-PS4	2

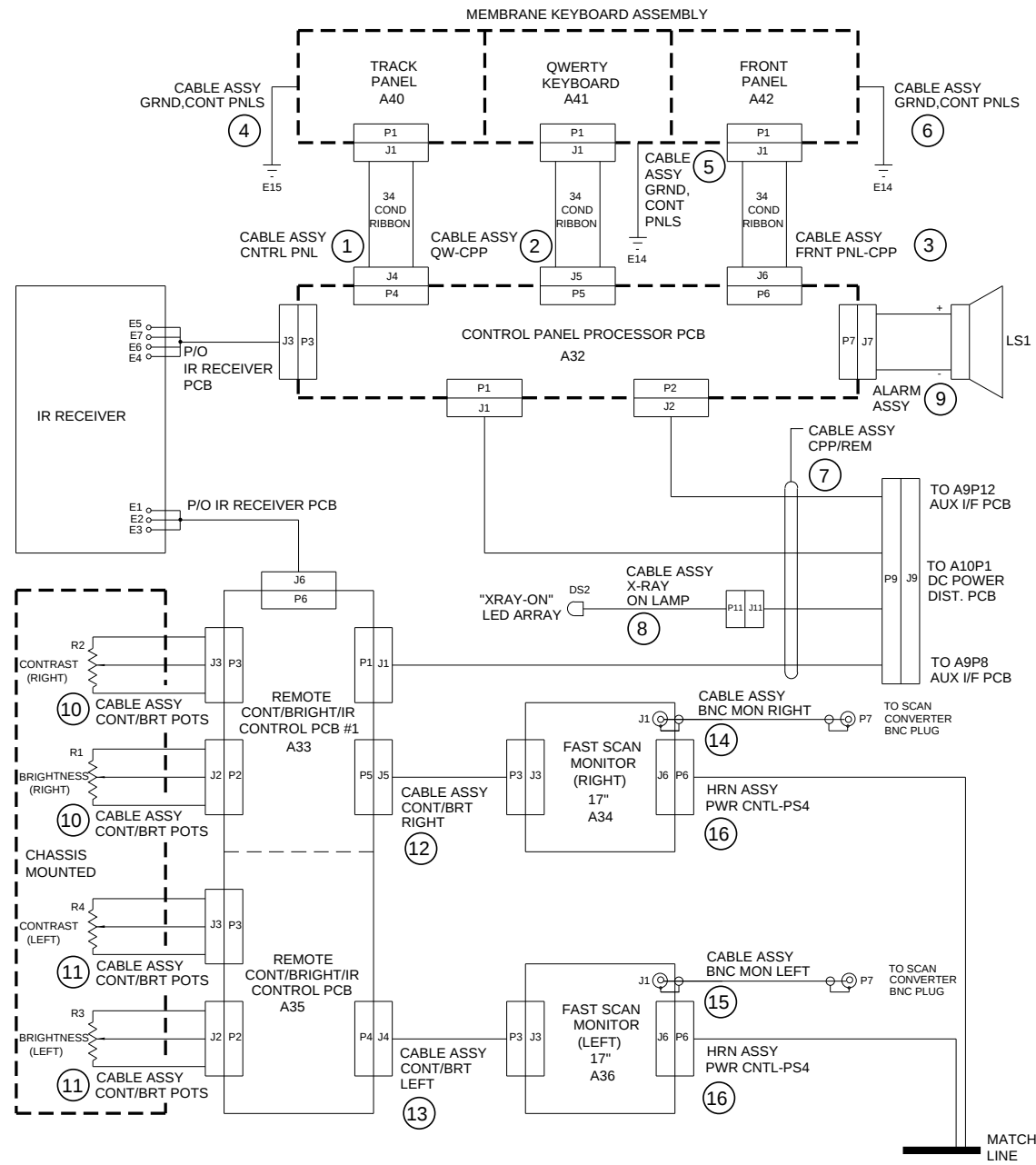


Fig. 2-14A Interface Cable Diagram, Imaging Workstation (Sheet 1 of 5)

FIG. 2-14B PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
16	00-876438-01	HARNESS ASSEMBLY, POWER CONTROL-PS4	1
17	00-876181-01	ASSEMBLY, POWER CORD, 220-250V, 20 FT, 9600	1
18	00-876181-02	ASSEMBLY, POWER CORD, 220-250V, 30 FT, 9600	1
19	00-877318-01	ASSEMBLY, POWER CORD, 220V, 60 HZ, 20', 9600	1
20	00-876180-01	ASSEMBLY, POWER CORD, 100-270V, 20 FT, 9600	2
21	00-876435-02	CABLE ASSEMBLY, SAFETY GROUND STRAP	1
22	00-876435-01	CABLE ASSEMBLY, SAFETY GROUND STRAP	1
23	00-876437-01	CABLE ASSEMBLY, SURGE SUP T1	1
24	00-876542-01	CABLE ASSEMBLY, POWER, POWER CONTROL PCB	1
25	00-877318-02	ASSEMBLY, POWER CORD, WKST, DOM, URO2600, 60'	1
26	00-876183-01	ASSEMBLY, CABLE, INTERCONNECT, 20', 9600	2
27	00-876184-01	ASSEMBLY, CABLE, INTERCONNECT, 30', 9600	2
28	00-876543-01	CABLE ASSEMBLY, POWER CONTROL TO TB2	1
29	00-876998-01	CABLE ASSEMBLY, POWER CONTROL TO J15	1

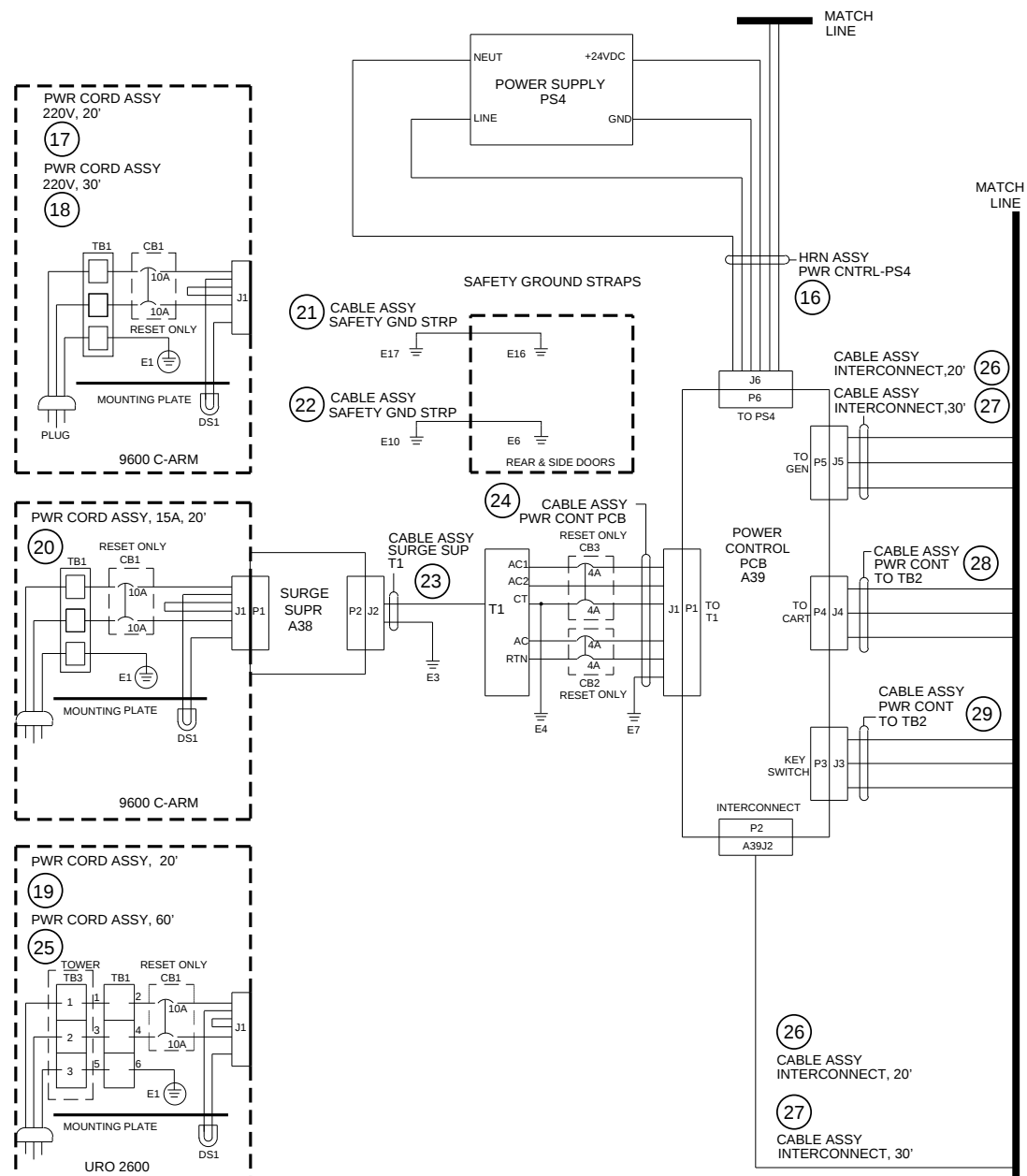


Fig. 2-14B Interface Cable Diagram, Imaging Workstation (Sheet 2 of 5)

FIG. 2-14C PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
26	00-876183-01	CABLE ASSEMBLY, INTERCONNECT, 20', 9600	2
27	00-876183-02	CABLE ASSEMBLY, INTERCONNECT, 30', 9600	2
28	00-876543-01	CABLE ASSEMBLY, POWER CONT TO TB2	1
29	00-876998-01	CABLE ASSEMBLY, POWER CONT TO J15	1
30	00-876544-01	CABLE ASSEMBLY, KEY SWITCH TO P15	1
31	00-876161-01	KIT, LENZAR, 60 HZ, 9600, ENGLISH	3
32	00-876846-07	CABLE ASSEMBLY, GROUND, CONT PANELS, 9600	1
33	00-877009-01	ASSEMBLY, OUTLET STRIP, INTERNATIONAL AC	1
34	00-876803-01	CABLE ASSEMBLY, VIDEO, INT, VCR, 9600	1
35	00-876802-01	CABLE ASSEMBLY, CONT, INT, VCR, 9600	1
36	00-876803-02	CABLE ASSEMBLY, VIDEO, INT, VCR, 9600	1
37	00-876444-01	CABLE ASSEMBLY, TB2-J16	1
38	00-900740-01	CABLE ASSEMBLY, A9P10 TO J12, 13, 14	1

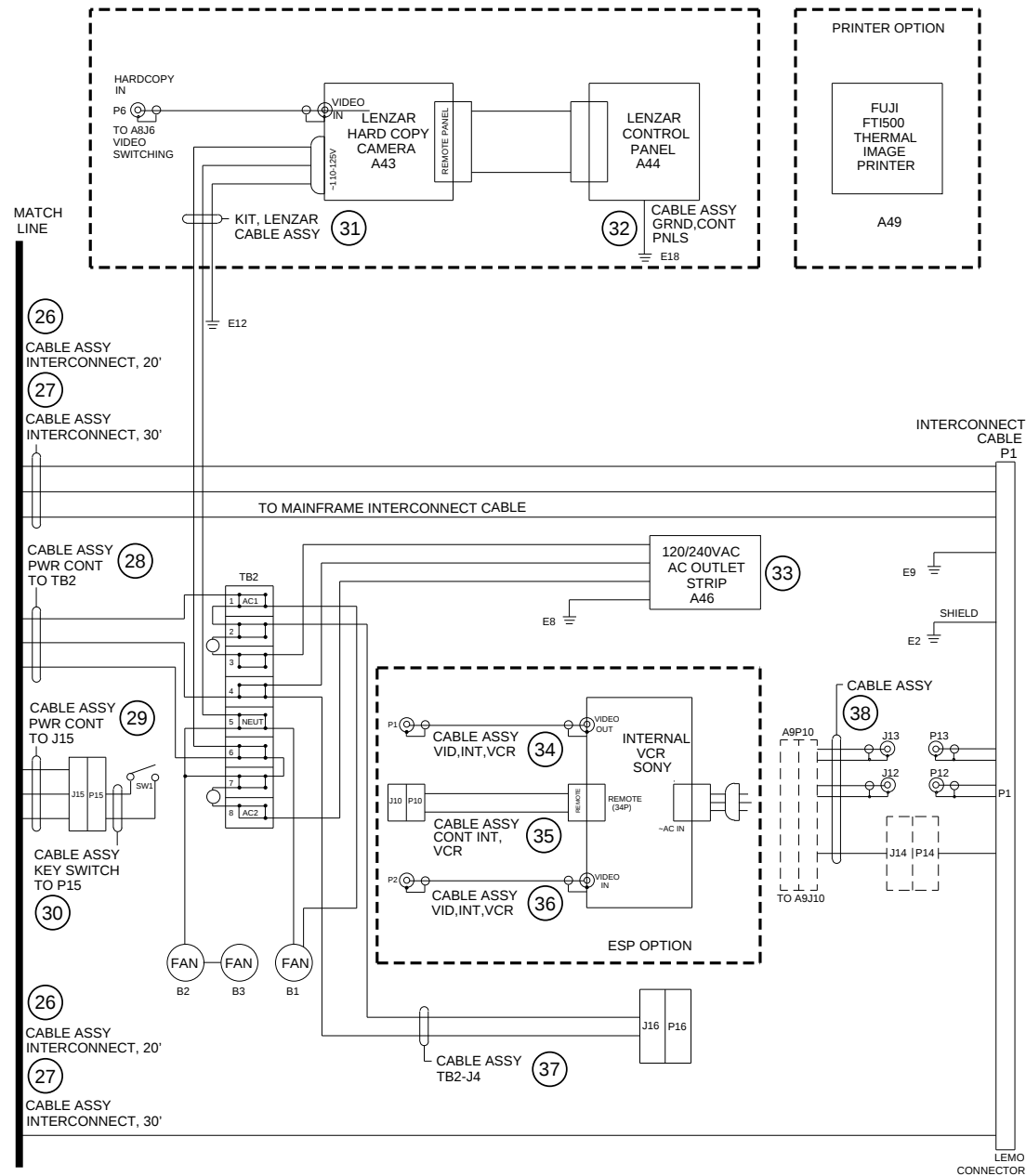


Fig. 2-14C Interface Cable Diagram, Imaging Workstation (Sheet 3 of 5)

FIG. 2-14D PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
39	00-900720-01	CABLE ASSEMBLY, VCR1/VCR2, 9600	2
40	00-900462-01	CABLE, SIGNAL, VIDEO, SW TO AUX	1
41	00-878014-01	CABLE ASSEMBLY, AUX IF/HRS CONV, WORKSTATION	1
42	00-876442-01	CABLE ASSEMBLY, VIDEO, SW/AUX I/F	1
43	00-876449-01	CABLE ASSEMBLY, SIGNAL, AUX INTF/VIDEO SW	1
44	00-900948-01	CABLE ASSEMBLY, VIDEO, 9600	1
45	00-876453-01	CABLE ASSEMBLY, SIGNAL, IP/VIDEO, SW	1
46	00-900721-01	CABLE ASSEMBLY, COAX, VIDEO SW-IP, 9600	1
47	00-876634-01	CABLE ASSEMBLY, POWER, IP, VIDEO SW & AUX I/F	2
48	00-876534-01	CABLE ASSEMBLY, INTERNAL VCR CONTROL	1
49	00-876450-01	CABLE ASSEMBLY, SIGNAL AT COM 26 PIN	1
50	00-876451-01	CABLE ASSEMBLY, SIGNAL AT COM 34 PIN	1
51	00-876452-01	CABLE ASSEMBLY, SIGNAL AT COM 50 PIN	1
52	00-876550-01	CABLE ASSEMBLY, COAX, NETWORK (FIG, 2-15E)	1
53	00-876447-01	CABLE ASSEMBLY, POWER, DISK DRIVES	1
54	00-876446-01	CABLE ASSEMBLY, POWER, MOTHERBOARD	1
55	00-876394-01	CABLE ASSEMBLY, DISK DRIVE & ALTA	1
56	00-877291-01	CABLE ASSEMBLY, 30 FPS HARD DRIVE POWER, 9600	1
57	00-876536-01	CABLE ASSEMBLY, POWER, ETI	1
58	00-876445-01	CABLE ASSEMBLY, POWER SUPPLY	1
59	00-876546-01	CABLE ASSEMBLY, PS1 TO A10	1
60	00-876439-01	CABLE ASSEMBLY, DC POWER & AUX I/F TO J9	1

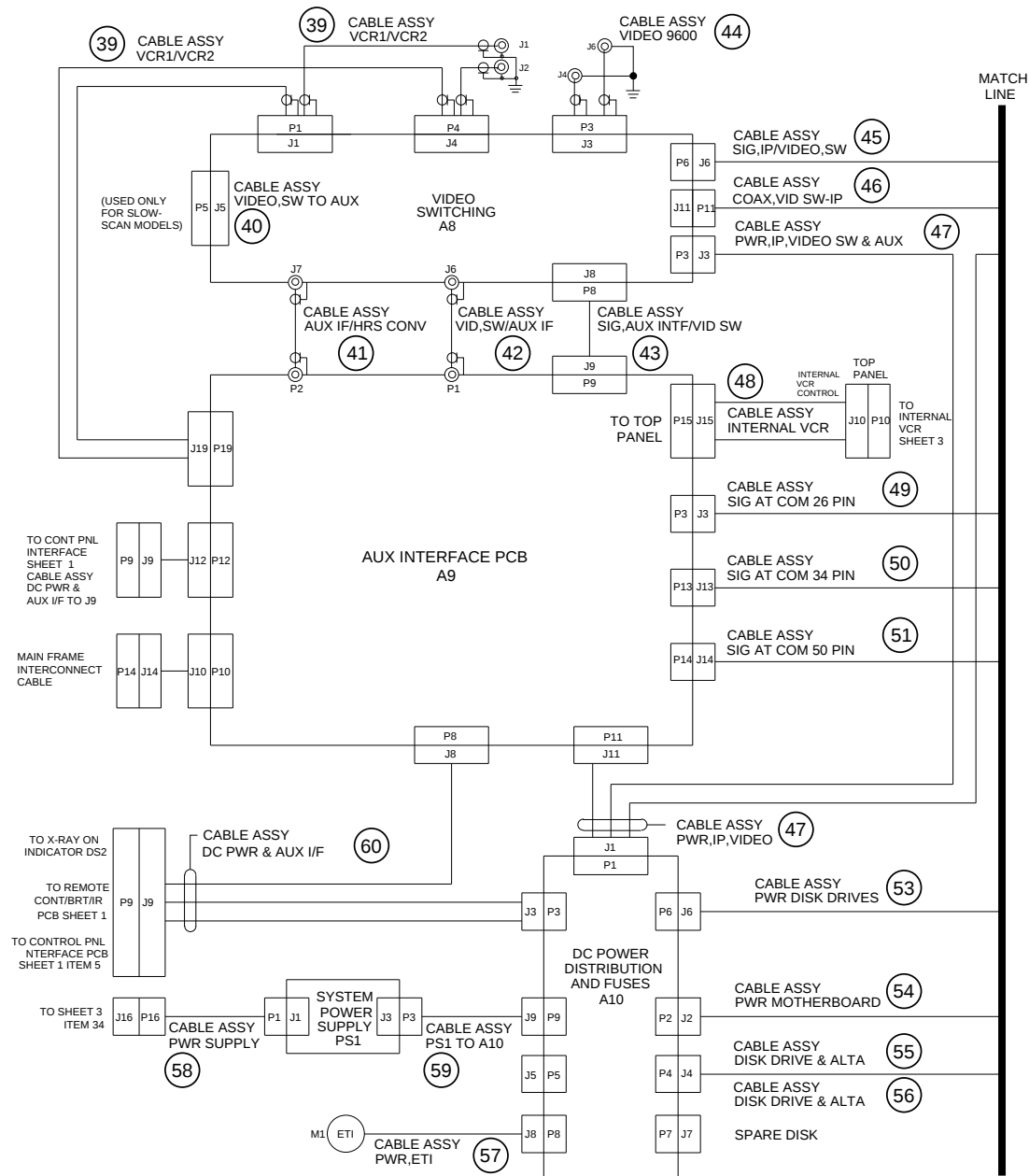


Fig. 2-14D Interface Cable Diagram, Imaging Workstation (Sheet 4 of 5)

FIG. 2-14E PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
45	00-876453-01	CABLE ASSEMBLY, SIGNAL, IP/VIDEO, SW	1
46	00-900721-01	CABLE ASSEMBLY, COAX, VIDEO SW-IP, 9600	1
47	00-876634-01	CABLE ASSEMBLY, POWER, IP, VIDEO SW & AUX I/F	2
48	00-876534-01	CABLE ASSEMBLY, INTERNAL VCR CONT (FIG. 1-14D))	1
49	00-876450-01	CABLE ASSEMBLY, SIGNAL AT COM 26 PIN	1
50	00-876451-01	CABLE ASSEMBLY, SIGNAL AT COM 34 PIN	1
51	00-876452-01	CABLE ASSEMBLY, SIGNAL AT COM 50 PIN	1
52	00-876550-01	CABLE ASSEMBLY, COAX, NETWORK	1
53	00-876447-01	CABLE ASSEMBLY, POWER, DISK DRIVES	1
54	00-876446-01	CABLE ASSEMBLY, POWER, MOTHERBOARD	1
55	00-876394-01	CABLE ASSEMBLY, DISK DRIVE & ALTA	1
56	00-877291-01	CABLE ASSEMBLY, 30 FPS HARD DRIVE POWER, 9600	1
57	00-876536-01	CABLE ASSEMBLY, POWER, ETI (FIG. 1-14D)	1
58	00-876445-01	CABLE ASSEMBLY, POWER SUPPLY (FIG. 1-14D)	1
59	00-876546-01	CABLE ASSEMBLY, PS1 TO A10 (FIG. 1-14D)	1
60	00-876439-01	CABLE ASSEMBLY, DC POWER & AUX I/F TO J9 (FIG. 1-14D)	1
61	00-876455-01	CABLE ASSEMBLY, SIGNAL, FLOPPY DISK	1
62	00-876367-01	CABLE ASSEMBLY, DATA FAST SCAN-IP	1
63	00-877365-01	CABLE ASSEMBLY, SIGNAL, ALTA 2 HD, 2, 30 FPS	1
64	00-877364-01	CABLE ASSEMBLY, SIGNAL, ALTA 1-ALTA, 30 FPS	1
65	00-876395-01	CABLE ASSEMBLY, SIGNAL, ALTA IPI TO IP	1
66	00-877412-01	CABLE ASSEMBLY, SIGNAL, IP-ALTA	1

FIG. 2-14E PARTS LIST (CONT.)

ITEM	PART NUMBER	DESCRIPTION	QTY
67	00-877363-01	CABLE ASSEMBLY, SIGNAL, IP-ALTA, 30 FPS	1
68	00-877977-01	CABLE ASSEMBLY, LASER CAM I/F, 9600	1
69	00-877366-01	CABLE ASSEMBLY, SIGNAL, ALTA-SCSI/HD	1
70	00-876454-01	CABLE ASSEMBLY, SIGNAL, HD, 40 PIN	1
71	00-876366-01	CABLE ASSEMBLY, FAST SCAN-BNC (RIGHT)	1
72	00-876366-02	CABLE ASSEMBLY, FAST SCAN-BNC (LEFT)	1
73	00-877901-01	CABLE ASSEMBLY, VIDEO, FASTSCAN, 9600	1
74	00-878014-01	CABLE ASSEMBLY, AUX IF/HRS CONV	1

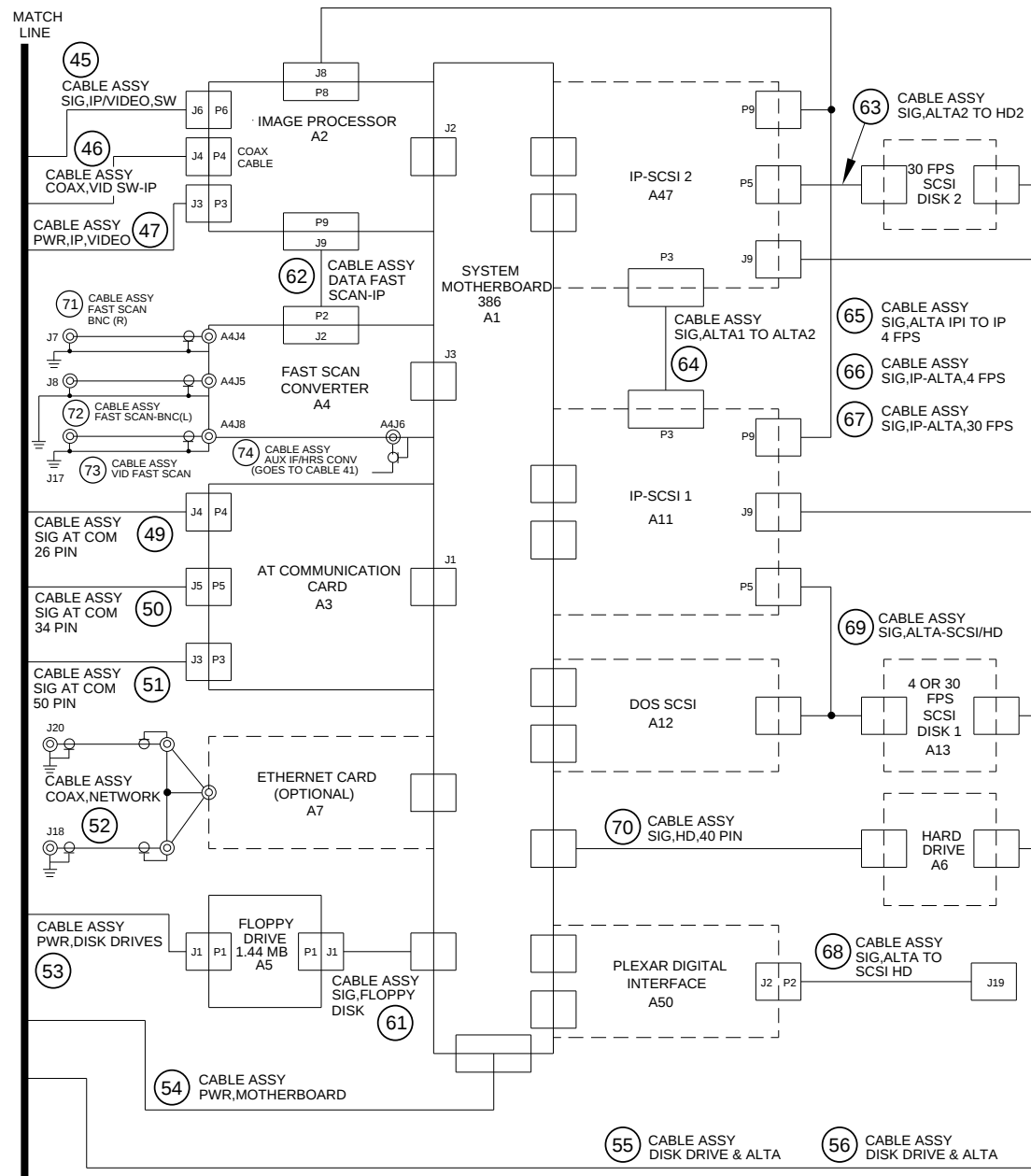


Fig. 2-14E Interface Cable Diagram, Imaging Workstation (Sheet 5 of 5)

ATTACHING HARDWARE

PART NUMBER	DESCRIPTION	QTY
00-876119-01	SPACER, HINGE, SIDE, ELECT BOX	1
00-876990-01	CLIP, SPRING, KEYBOARD	1
00-900407-02	WASHER, COLOR CODED	1
00-900407-03	WASHER, LOCK, SERRATED	1
00-900407-04	NUT, MU 0.8	1
00-900407-05	WASHER, U/M6	1
10-601106-04	SCREW, SLFTAP, PAN, SLT, 6-32, 1/4, SZC	2
12-215004-04	SCREW, MACH, FL, TRX, 4-40, 1/4, SZC	4
12-215006-06	SCREW, MACH, FL, TRX, 6-32, 3/8, SZC	18
12-215006-08	SCREW, MACH, FL, TRX, 6-32, 1/2, SZC	10
12-215006-12	SCREW, MACH, FL, TRX, 6-32, 3/4, SZC	8
12-215008-06	SCREW, MACH, FL, TRX, 8-32, 3/8, SZC	1
12-221104-04	SCREW, MACH, PAN, PHL, 4-40, 1/4, SST	7
12-221104-12	SCREW, MACH, PAN, PHL, 4-40, 3/4, SST	2
12-225104-04	SCREW, SEMS SPRING, PAN, PHL, 4-40, 1/4, SZC	2
12-225104-05	SCREW, SEMS SPRING, PAN, TRX, 4-10, 5/16, SZC	42
12-225104-06	SCREW, SEMS SPRING, PAN, TRX, 4-40, 3/8, SZC	2
12-225106-04	SCREW, SEMS APR, PAN, TRX, 6-32, 1/4, SZC	6
12-225106-06	SCREW, SEMS SPRING, PAN, TRX, 6-32, 3/8, SZC	31
12-225106-08	SCREW, SEMS SPRING, PAN, TRX, 6-32, 1/2, SZC	23
12-225106-12	SCREW, SEMS SPRING, PAN, TRX, 6-32, 3/4, SZC	10

ATTACHING HARDWARE (CONT.)


PART NUMBER	DESCRIPTION	QTY
12-225108-06	SCREW, SEMS SPRING, PAN, TRX, 8-32, 3/8, SZC	41
12-225108-08	SCREW, SEMS SPRING, PAN, TRX, 8/32, 1/2, SZC	14
12-225108-12	SCREW, SEMS SPRING, PAN, TRX, 8/32, 3/4, SZC	2
12-225303-08	SCREW, METRIC SEMS, PAN, TRX, M3X0.5, 6MM, SZ	14
12-312014-10	SCREW, NA, SKT, HEX, 1/4-20, 5/8, SST	14
12-600000-08	SCREW, NA, HEX, CAP, 1/4-20, 1/2, SZY, 8	8
12-900711-04	SCREW, MACH, PAN, SLT, 4-40, 1/4, NYLON	2
12-912008-14	SCREW, METRIC, HEX, CAP, M8X1.25, 12MM, SST	4
13-122104-00	NUT, HEX, 4-40, NYLON, NA, A = .250, C = .094	2
13-311104-00	NUT, HEX-LK-EXT-T, 4-40, SST, NA, A = .250	5
13-311106-00	NUT, HEX-LK-EXT-T, 6-32, SST, NA, A = .313	33
13-311108-00	NUT, HEX-LK-EXT-T, 8-32, SST, NA, A = .344	14
13-311114-00	NUT, HEX-LK-EXT-T, 1/4-20, SST, NA, A = .4	11
13-532011-00	NUT, HEX-LK-W/INSR, 5/16-18, SZC	4
14-101104-00	WASHER, PL, #4, SZC, NA, ID = .125, OD = .312	2
14-102014-00	WASHER, PL, 1/4, SST, NA, ID = .265, OD = .500	8
14-111104-00	WASHER, PL, #4, SST, MS15795-804, ID = .120	5
14-201104-00	WASHER, LK-HLCL-SPR, #4, SST, MS35338-135	8
14-211114-00	WASHER, LK-HLCL-SPR, 1/4, SST, MS35338-	8
14-311112-00	WASHER, LK-EXT-T, 5/16, SST, NA, ID = .320	4

ATTACHING HARDWARE (CONT.)

PART NUMBER	DESCRIPTION	QTY
14-311114-00	WASHER, LK-EXT-T, 1/4, SST, NA, ID = .256	12
14-900002-00	WASHER, PL, #4, SST, NA, ID = .125, OD = .281	2
14-999702-00	WASHER, PL, #4, NYLON, NA, ID = .125, OD = .250	2
21-709008-00	SPRING, CPRSN, .420OD, 3/4LG, .042WD, SST	2
22-012010-00	CLAMP, CABLE, 7/16D, 1/2W, STL	1
22-012101-00	CLAMP, CABLE/W, .437 X .39SQ, .219W, NYLON	18
22-012102-00	CLAMP, CABLE/W, .437 X .75SQ, .219W, NYLON	5
22-012103-00	CLAMP, CABLE/W, .437 X .34SQ, .219W, NYLON	23
22-012104-00	CLAMP, CABLE/W, .437 X 1.02SQ, .219W, NYLON	6
22-119605-00	CLAMP, CABLE TIE MT, NYLON	2
22-119650-00	CLAMP, CABLE TIE, 3/32 X 3-5/8, NYLON	AR
23-620843-00	RETAINER, 1/4TRN, .485OD X .013THK, SST	8
23-621841-00	FASTENER, 1/4TRN STD, PHL, SZC	8
23-621843-00	FASTENER, RCPT, 1/4TRN, CLP-ON, .062/.187	8
23-621844-00	FASTENER, 1/4TRN STD, PHL, MINTR, NYLON	6
23-621845-00	FASTENER, RCPT, 1/4TRN, CLP-ON, .0011/.080	6

Section 3

SERIES9600™ Imaging Workstation Illustrated Parts Manual Updates

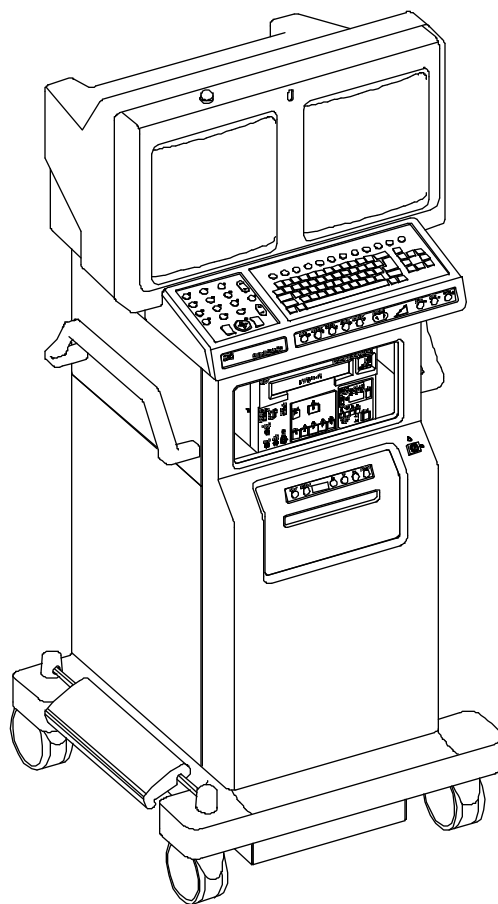

SERIES9600™ Imaging Workstation Illustrated Parts Manual Update July 1995

(P/N 00-878549-01)

This update reflects changes that occurred to the **SERIES9600** Imaging Workstation Illustrated Parts Manual. Review the contents to familiarize yourself with the field replaceable units that have changed. Place this update in the back of your Illustrated Parts Manual, behind the other sections.

© July 1995
OEC Medical Systems, Inc.
384 Wright Brothers Drive
Salt Lake City, Utah 84116
U.S.A.

FIG. 1 JULY 1995 UPDATE PARTS LIST

ITEM	PART NUMBER	DESCRIPTION	QTY
1	00-900968-01	PANEL ASSEMBLY, FRONT ESP (ENGLISH)	1
1	00-900968-02	PANEL ASSEMBLY, FRONT ESP (GERMAN)	1
1	00-900968-03	PANEL ASSEMBLY, FRONT ESP (FRENCH)	1
1	00-900968-04	PANEL ASSEMBLY, FRONT ESP (SPANISH)	1
1	00-900968-05	PANEL ASSEMBLY, FRONT ESP (ITALIAN)	1
2	00-900967-01	PANEL ASSEMBLY, FRONT VAS (ENGLISH)	1
2	00-900967-02	PANEL ASSEMBLY, FRONT VAS (GERMAN)	1
2	00-900967-03	PANEL ASSEMBLY, FRONT VAS (FRENCH)	1
2	00-900967-04	PANEL ASSEMBLY, FRONT VAS (SPANISH)	1
2	00-900967-05	PANEL ASSEMBLY, FRONT VAS (ITALIAN)	1
3	00-875981-03	COVER SIDE, UPPER LEFT (NOT EXPLODED)	1
4	00-875981-04	COVER SIDE, UPPER RIGHT (NOT EXPLODED/SHOWN)	1
5	00-878420-01	COVER, FRONT PANEL	1
6	00-875986-03	COVER, SIDE LOWER LEFT (NOT EXPLODED)	1
7	00-875986-04	COVER, SIDE LOWER RIGHT (NOT EXPLODED)	1

Note: All items listed below are options on the European model.

8	00-878543-01	FAN KIT	1
9	00-900975-01	SONY 910 VIDEO GRAPHIC PRINTER	1

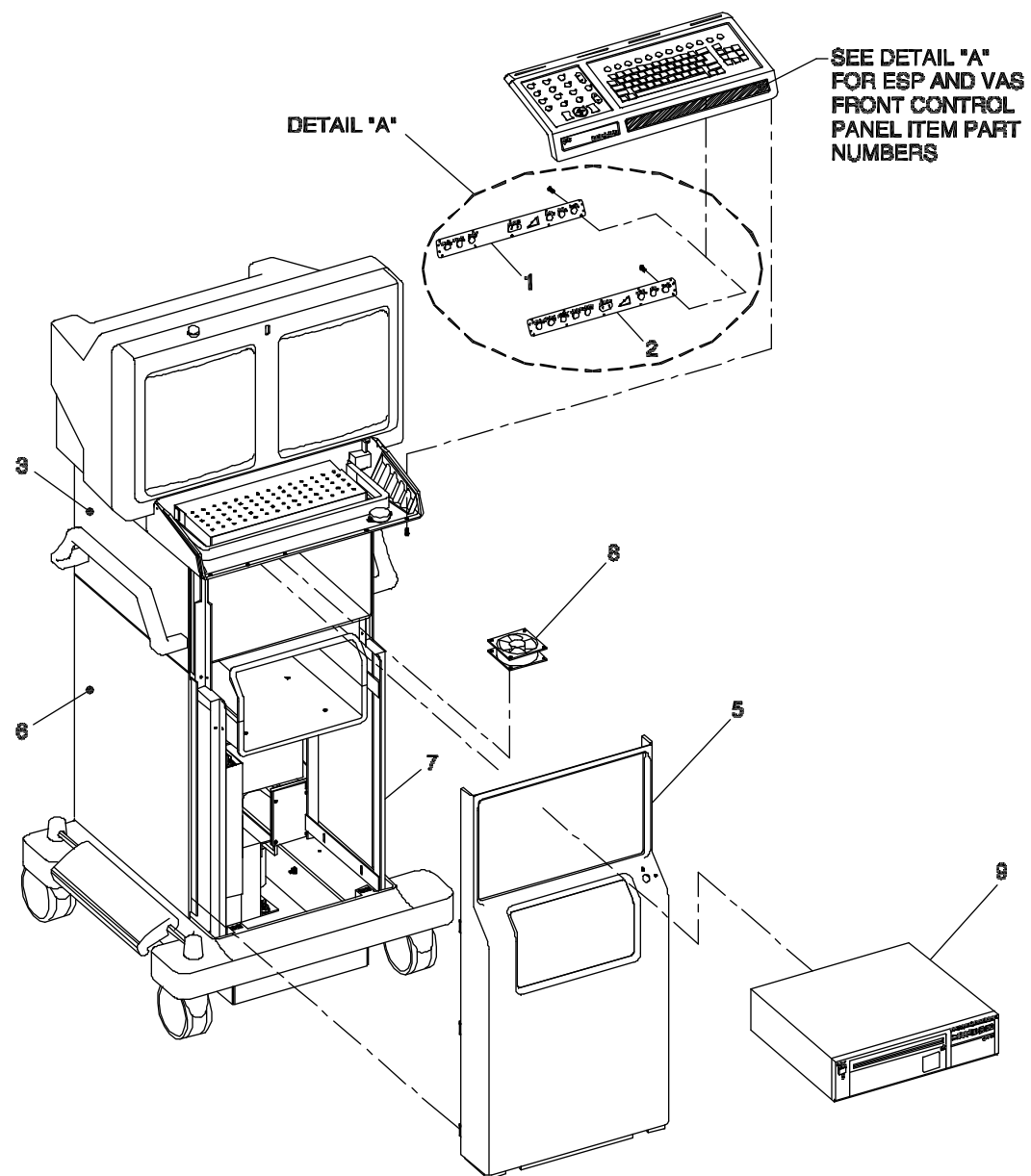


Fig. 1 July 1995 Update Front Panel, Front Control Panel, Sony Printer, and Fan Kit Exploded View